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Additional elements less than 1 %: Fe, Co, Ce
Contains no beryllium

remanium® CSe must not be used for patients suffering from nickel allergy.

Technical data

Yield strength*	R _{0,2}	340 MPa
Tensile strength*	R _m	580 MPa
Hardness*	HV10	195
Fracture elongation*	A ₅	15 %
Modulus of elasticity*	E	170 GPa
Density		8.2 g/cm ³
Solidus		1260 °C / 2300 °F
Liquidus		1350 °C / 2462 °F
CTE (25 – 500 °C / 77 – 932 °F)		14 x 10 ⁻⁶ K ⁻¹
Color		white
Weight per ingot		approx. 6 g

* Variations of +/- 10 % are possible depending on the batch.

Use	NiCrMo bonding alloy compatible with conventional ceramic materials with appropriate CTE range
Melting	Torch, Induction (HF), Light arc
Mirror like melt surface (HF)	Yes
Casting time point (HF)	As soon as the molten mass begins to split
Pressure casting	250 – 450 mbar Observe instructions for use REF 989-448-00!
Welding	Possible using methods such as laser welding and TIG impulse welding
Additive material	NiCr welding wire, REF 528-220-00 (0.5 mm)
Presoldering	NiCr-Sold 1, REF 102-302-00
Flux	rema®-Flux 1, REF 102-304-00
Recommended ceramic	Ceramics with suitable CTE range such as Carmen® and CCS, from Dentaureum
Cooling of ceramics in the furnace	Regular cooling, unless otherwise recommended by the ceramic manufacturer

For further information see:

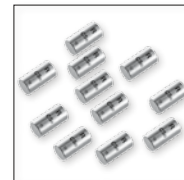
remanium®, non-precious metal alloys for crown and bridgework, instructions for use **REF 989-448-00**
Available free of charge from Dentaureum or in the internet under www.dentaureum.de

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