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Instructions for use

Mode d'emploi

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Instructions for use  
Self-ligating brackets – discovery<sup>®</sup> sl

**Dear Customer**  
Thank you for having chosen a quality product from Dentaurum. In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully. The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise. Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaurum.de.

**Manufacturer**  
Dentaurum J. P. Winkelstroeter KG · Turnstraße 31 · 75228 Ispringen · Germany

**Working Steps**  
Inform the patient of the possible risks and adverse effects accompanying treatment with brackets, especially during debonding.

**1. Tooth preparation**  
Check each tooth carefully for possible cracks or damage so that there are no complications during the debonding stage. Please observe the manufacturer's instructions for the bracket adhesive you intend to use for conditioning the teeth.

**2. Choice of bracket adhesive**  
discovery<sup>®</sup> sl brackets can be bonded using any standard bracket adhesive. We recommend our own adhesives ConTec-, NO-MIX-, MIX-adhesive and SmartBond<sup>®</sup>. Please read the respective instructions for use for any bracket adhesive you choose.

**3. Removal and handling**  
Grasp the discovery<sup>®</sup> sl brackets mesio-distally using bracket tweezers (eg. REF 025-277-00), remove from the packaging, and then observe the following working steps. The patented laser structured base of the self-ligating brackets should not be touched by fingers or contaminated in any way, as this could have an adverse effect on the bonding quality.

**4. Using the bracket adhesive**  
Prepare the bracket adhesive according to the relevant manufacturer's instructions. Apply a small amount of adhesive on the center of the bracket base, enough to cover the entire bracket base. This will guarantee an optimal bond.

**5. Application and alignment of the brackets**  
After having applied the adhesive, place the discovery<sup>®</sup> sl bracket onto the surface of the tooth in the customary manner.

Use the laser markings on the top of the brackets (Fig. 1) as a positioning guide. The vertical line marks the bracket center. The horizontal line marks the middle of the slot. **Please note that the slot does not lie in the center of the bracket, and take this into consideration when aligning the bracket!** It is essential to avoid adverse contact due to the risk of abrasion and/or premature bracket loss. If necessary a bite plate can be used in order to raise the bite. Remove all excess adhesive material from the bracket base before it sets. **It is essential to ensure that no adhesive material enters the bracket fastening mechanism on the gingival and occlusal side of the bracket, as this could restrict or destroy the functionality of the lid.** The removal of excess adhesive material is especially important to facilitate debonding after the treatment has ended.

**6. How the fastening mechanism works / securing the arch wire**  
The slot of the discovery<sup>®</sup> sl brackets is closed off with a lid. All lids, whether they are designed for the upper or lower jaw, open from the occlusal side towards gingival (Fig. 2). This is carried out using the sl instrument (REF 023-276-00) (Fig. 3). The lid must be opened in order to insert the arch wire. Place the pointed end of the sl instrument into the notch in the lid between the occlusal bracket wings (Fig. 4a and 4b) and flip the lid open, using slight leverage towards vestibular-lingual (Fig. 5). The lid will spring open.

The discovery<sup>®</sup> sl brackets include a mechanism (CapSafe<sup>®</sup>), which enables the lid to remain open while the arch wire is being inserted. In order to activate CapSafe<sup>®</sup>, open the lid and press it back in a gingival direction with your finger or the sl instrument until there is a feeling of resistance (Fig. 6). The lid is now secured in the open position (Fig. 7). Place the arch wire into the slot (Fig. 8). If necessary, use the forked end of the sl instrument to aid in this step. To close the lid, use your finger or the sl instrument to press the lid from the gingival to the occlusal position (Fig. 9). The lid should close with a distinct click, which guarantees the lid is now secure (Fig. 10).

In order to prevent an unintentional closing of the lid during arch wire change, we recommend to use lip retractors (REF 075-600-00 or 075-601-00).

**7. Bracket debonding**  
The excellent bonding characteristics of discovery<sup>®</sup> sl brackets means the process of debonding requires special attention: When debonding discovery<sup>®</sup> sl brackets it is advisable to use a Weingart plier (eg. REF 003-120-00) or in the molar region, an angled bracket removing plier (REF 004-347-00). Grasp the bracket mesio-distally on the tie wings, and twist toward the left or the right without pulling away from the tooth.

**8. Note for doctor and patient**  
• Please consider that teeth must never be brushed with the bracket lids open e.g. for arch wire change! This leads to torsion of the delicate lids, which distracts the function of the closing mechanism so that the lid cannot be optimally closed any more.  
• This product contains nickel and chrome. Patient's suffering from allergic reactions to these metals must not undergo treatment with this product. There are no other known risks or side effects regarding the material used in discovery<sup>®</sup> sl.  
• As the material 1.4542 used in the self-ligating discovery<sup>®</sup> sl brackets is one of the slightly magnetizable metals, we recommend to take the brackets off before the magnetic resonance imaging examination.

**Additional Tips**  
• Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog information.  
• Store discovery<sup>®</sup> sl in sealed packaging.  
• There are no particular precautions to be taken when disposing of discovery<sup>®</sup> sl brackets after treatment.  
• If you have any further questions regarding fixed orthodontic techniques, please use our orthodontic hotline Tel. +49 72 31 / 803-550.

**With reference to our quality standards**  
Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Date of information: 10/09  
Subject to modifications.

Fig. 1

Fig. 2

Fig. 3

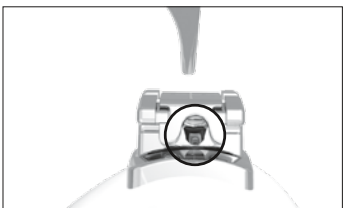
Fig. 4 a

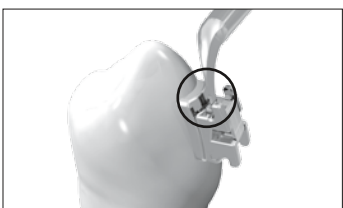
Fig. 4 b

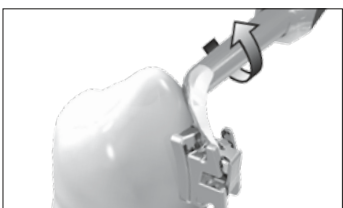
Fig. 5

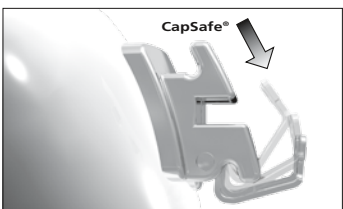
Fig. 6

Fig. 7

Fig. 8

Fig. 9

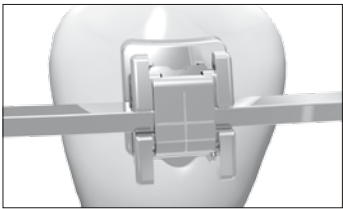
Fig. 10

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