discovery
delight

Gebrauchsanweisung | Instructions for use
Mode d'emploi | Modo de empleo
Modalità d'uso

D
DENTAURUM

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Para más información sobre nuestros productos, visite www.dentaurum.com
- Informazioni sui prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 2023-12

Änderungen vorbehalten | Subject to modifications |
Sous réserve de modifications | Reservado el derecho de modificación |
Con riserva di apportare modifiche

CE 0483

D
DENTAURUM
Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 7231/803-0 | Fax +49 7231/803-295
www.dentaurum.com | info@dentaurum.com

Printed by Dentaurum Germany 2023-12/BR13-3
989-530-00

DE

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EN

Instructions for use discovery® delight lingual brackets

Dear customer,

Thank you for choosing a quality product from Dentaurum. Please read these instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. Should you have any questions or ideas, please contact your local representative.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

discovery® delight is a special bracket system for lingual orthodontic treatment (lingual technique).

3. Intended purpose

For the treatment of tooth misalignments as part of orthodontic treatment.

4. Intended users

All products are intended for dental professionals only.

5. Indications

For tooth and jaw misalignments.

6. Contraindications

Stainless steel brackets must not be used in patients with a nickel allergy. Warnings are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients for whom a fixed appliance with orthodontic braces can be placed as part of orthodontic treatment.

8. Processing

8.1. Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment.

8.2. Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding.

8.3. Selection of bracket adhesive

discovery® delight lingual brackets can be bonded using all commercially available bracket adhesives.

8.4. Removal and handling

Grip the discovery® delight lingual brackets mesially-distally using crimping tweezers to remove them from the packaging. The laser-structured base of the discovery® delight lingual brackets should not be touched with your fingers or contaminated in any way, as this could otherwise have a detrimental effect on the bond.

8.5. Using the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

8.6. Placing the brackets – direct bonding

After applying the adhesive, place the discovery® delight lingual bracket in the usual way onto the tooth. Use a positioning tool (REF 023-278-00) to achieve the best position for the lingual bracket and press it onto the tooth firmly. Completely remove any excess adhesive before it has cured.

8.7. Placing the brackets – indirect bonding

After applying the adhesive, place the transfer tray with the discovery® delight lingual brackets as usual in the mandible and/or maxilla. Completely remove any excess adhesive.

8.8. Ligating archwires

Once the adhesive has cured, ligate the lingual bows as usual with wire or plastic ligatures.

8.9. Bracket debonding

For the debonding of discovery® delight lingual brackets, we recommend using Weingart pliers. Grasp the brackets mesially-distally on the wings and turn the bracket briefly to the left or right, without pulling, to remove it from the tooth.

9. Composition

discovery® delight brackets are made of the following material: DIN 1.4404. The composition is included in the orthodontic materials list; please refer to the catalog or www.dentaurum.com. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

10. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

11. Storage and shelf life

Store discovery® delight lingual brackets in the closed packaging. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

12. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

14. Product range

For the complete product range of discovery® delight lingual brackets, please see our orthodontics catalog.

15. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

16. Explanation of symbols used on the label

Please refer to the label. Additional information and the Instructions for use can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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