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Stand der Information: 02/10

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Dear Customer

Thank you for having chosen a quality product from Dentaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaurum.de.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany

2. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. Fabricating the Herbst I appliance

Pour the impressions, which include the waxed out bands fitted on the 1st premolars and molars, with dental stone (e.g. Rapidur®). Mount the models in the correct position in an articulator with plaster using the activating bite recorded by the operator. Bend and solder (or laser weld) the half-round remaloy® wire (ø 0.75 x 1.5 mm) or round remaloy® wire (ø 1.0 mm) to the bands to form a lingual arch. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization.

Next spot weld and solder the bases of the hinge units to the bands (e.g. using the Master 3000). Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the hinges between the upper and lower jaw operate without friction to ensure smooth functioning of the telescopic mechanism. Friction can damage the appliance – especially the bands.

Then fit the telescope tubes and telescope rods: first shorten the tubes according to the activating bite and then fit the telescope rods, which should be approx. 2 mm longer.

If the telescope rods are too short, they could come out of the telescope tube; if the rods are too long, they could cause irritation of the mucosa (risk of injury!). We therefore recommend that the exact length is always checked again intraorally.

Note: "Double bands" should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the band already fitted (e.g. using the Master 3000).

Generally the second band is 2 to 3 sizes larger than the first band fitted. Ready-welded "double bands" can be ordered from Dentaurum.

Fitting intraorally:

The bands are fitted intraorally using standard band cement (e.g. GlasTec). We recommend covering the bases of the hinges with wax to ensure that the cement does not penetrate into the thread. After the cement has set, attach the telescope tubes and telescope rods (already shortened!) to the bases.

If necessary, the Herbst I appliance can be activated using various spacers (see Product range).

4. Safety instructions – Herbst hinge

The fit of the Herbst I hinge and tightness of the screws should be checked at regular intervals to prevent the appliance spontaneously coming apart and the components being swallowed or aspirated! If necessary, tighten the screws and secure them with a special screw fixing adhesive, e.g. Ceka® Bond.

5. Product range

Herbst I set	REF 607-120-00	1 Set
with hexagon socket screws		
For welding or soldering to bands or cast splints.		
Contents:		
• 1 x Telescope tube, right		
• 1 x Telescope tube, left		
• 2 x Telescope rod		
• 4 x Hexagon socket screw		
• 4 x F-base		
• 1 x Instructions for use		
The hexagon socket screw key is not included in delivery.		
Hexagon socket screw key	REF 607-144-00	1 piece
for Herbst TS/I/II/SUS ² sterilizable		
wrench size 1.5		
Allen screws	REF 607-122-30	10 pieces
for Herbst TS/I		
Herbst F base	REF 607-127-30	10 pieces
for Herbst TS/I		
Telescope tube		
for Herbst I/II,		
left	REF 607-121-00	1 piece
right	REF 607-122-00	1 piece
Telescope rod	REF 607-124-00	1 piece
for Herbst I/II		
Spacers		
for Herbst I/II/IV		
Length: 1 mm	REF 607-103-30	10 pieces
Length: 2 mm	REF 607-104-30	10 pieces
Length: 3 mm	REF 607-105-30	10 pieces
Length: 4 mm	REF 607-107-30	10 pieces
Length: 5 mm	REF 607-112-30	10 pieces

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