Estimado cliente

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Stand der Information: 09/10

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Dear Customer

Thank you for having chosen a quality product from Dentaurum. In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaurum.de.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany

2. Herbst II

For use on the upper and lower arch wire of a multibracket appliance.

The Herbst II retention hinge is fitted with a special connector, which can be used for attaching hinges to the existing arch wire of a multibracket appliance. This produces maximum anchorage without having to rely on the cooperation of the patient (e. g. headgear).

We assume the strongest steel arch possible in this treatment technique has been used with the multibracket appliance.

The connector (nut) of the hinge has a slot that is fitted onto the existing arch wire of the multibracket appliance and then secured to the arch wire using the screw supplied. The hinges are fitted to the arch wire at the mesial of the first molar in the upper jaw and between the canine and first premolar in the lower jaw.

Intraoral technique:

After fitting the nuts to the steel arch wire, measure the distance between the upper and lower nuts and shorten the hinges accordingly. Then secure the hinges to the nuts with the mounting screws. Remove the nuts first if the arch of the multibracket appliance has to be replaced. The hinge is then refitted as described above.

If necessary, the Herbst II appliance can be activated using various spacers (see Product range).

3. Composition

Please refer to the list of materials in the catalog or under www.dentaurum.de.

4. Safety Instructions

If there is a known allergic reaction to one or more constituents, this product must not be used. The fit and strength of the screws must be checked in regular intervals in order to prevent uncontrolled loosening and swallowing or inhalation of the parts! If necessary the screws must be tightened and secured with a screw fixation glue e.g. Ceka® Bond. The eyelets of the Herbst II retention hinge must not be bent! The eyelets were not designed to be ground.

5. Information on single-use products

The Herbst II retention hinge is intended for single use only. Reconditioning of Herbst II that has already been used (recycling) and its reuse on a patient is not permitted.

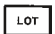
6. Product range

Herbst II set with hexagon socket screws	REF 607-108-00	1 Set
for fixation on arches according to multibracket technique		
Contents:		
• 1 x Telescope tube, right		
• 1 x Telescope tube, left		
• 2 x Telescope rod		
• 2 x Short screw		
• 2 x Long screw		
• 4 x Nut		
• 1 x Instructions for use		
The hexagon socket screw key is not included in delivery.		
Hexagon socket screw key	REF 607-144-00	1 piece
for Herbst TS//II//SUS ² sterilizable		
wrench size 1.5		
Hexagon socket screw		
for Herbst II,		
short	REF 607-108-10	10 pieces
long	REF 607-108-20	10 pieces
Nuts for Herbst II	REF 607-111-00	10 pieces
Telescope tube		
for Herbst I/II,		
left	REF 607-121-00	1 piece
right	REF 607-122-00	1 piece
Telescope rod	REF 607-124-00	1 piece
for Herbst I/II		
Spacers		
for Herbst I/II/IV		
Length: 1 mm	REF 607-103-30	10 pieces
Length: 2 mm	REF 607-104-30	10 pieces
Length: 3 mm	REF 607-105-30	10 pieces
Length: 4 mm	REF 607-107-30	10 pieces
Length: 5 mm	REF 607-112-30	10 pieces

7. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

8. Explanation of symbols used on the label

REF	Order number
 12345	Batch description
 0483	CE Mark MP Class IIa
 → www.dentaurum.de	Instructions for use available in the internet
	Not for reuse
	Codablock barcode
	Matrix code

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