


activeblue®



High performance cleaning unit –
designed especially for orthodontic appliances.



Gentle cleaning – quickly and effectively.

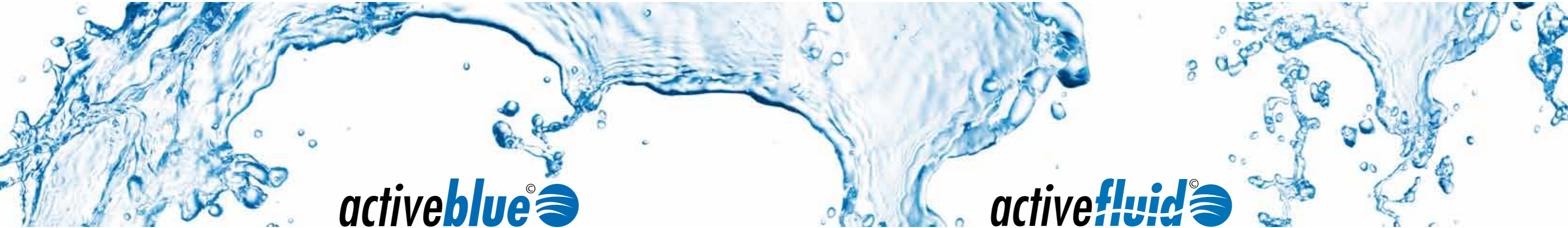

activeblue®

activefluid®



**High performance cleaning unit
REF 098-000-00**

Used for gentle removal of accretion on removable orthodontic appliances.

Contents:

- 1 x **activeblue**® high performance cleaning unit
- 1 x **activefluid**® cleaning fluid, 400 ml
- 1 x cleaning cup, white
- 1 x cleaning pins, 75 g
- 1 x tweezers
- 1 x holding magnet
- 1 x power cable 230 V

Technical data:

Power voltage: 1100-240 V at 50/60 Hz
Power input: 90 VA
Electric fuse: T 1,6AL / 250 V
Rotation speed: 1200 / 1600 / 2000 min⁻¹
Dimensions [B x H x T]: 150 x 240 x 280 mm
Unit net weight: netto 3 kg
Noise level: < 70 dB (A)

activeblue® is a compact, high performance cleaning unit for all types of removable orthodontic appliances. Its function is based on mechanical cleaning using magnetizable needles that rotate in a cleaning fluid.

Quick and effective cleaning of "braces" to remove stubborn plaque and calculus is achieved when used in combination with **activefluid**® cleaning fluid, which has been especially developed for orthodontic appliances. Cleaning is particularly effective in areas where normal mechanical cleaning (with a toothbrush) is not possible.

- **excellent service provision for your patients**
- **perfectly cleaned removable orthodontic appliances**
- **easy, quick, hygienic**
- **compact size, fits anywhere**

Special cleaner for use with activeblue®

activefluid® cleaning fluid is a special cleaner, which has been developed for cleaning orthodontic appliances in the high performance **activeblue**® cleaning unit. Even stubborn accretion, plaque and calculus are removed easily and gently. Since the fluid is especially designed for orthodontic appliances, corrosion and other detrimental effects can be excluded on sensitive expansion screws and wire elements.

No guarantee will be honored for orthodontic appliances, if cleaners other than **activeblue**® cleaning liquid are used!

- **especially developed for sensitive orthodontic appliances**
- **gently and reliably removes even stubborn accretion**
- **harmless for the patient**
- **for use with the activeblue® high performance cleaning unit**



Cleaning fluid REF 150-020-00

activefluid® cleaning fluid is available as an accessory for the **activeblue**® unit.

Chemical and physical property:
Acid cleaning fluid, pH = 2.0

Contents:

1 x 1000 ml **activefluid**® cleaning fluid

Dentaurum Group

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.



DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

➔ For more information on our products and services, please visit www.dentaurum.de

Date of information: 07/15

Subject to modifications



www.dentaurum.de

D
DENTAURUM