# Sandstrahlgerät **Dentablast Duo**

Sandblasting machine **Dentablast Duo**

Sableuse **Dentablast Duo**

Arenadora **Dentablast Duo**

REF 094-540-00  
094-541-00

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S. 2

Instructions for use

P. 12

Mode d'emploi

P. 22

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P. 32

# Sandstrahlgerät *Dentablast Duo*

REF 094-540-00  
094-541-00

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Instructions for use

## Contents

<b>1 Declaration of conformity</b> .....	<b>14</b>
1.1 General Information .....	15
<b>2 Safety issues</b> .....	<b>15</b>
2.1 Proper Use.....	15
2.2 Signs stipulating danger .....	15
2.3 Guarantee and liability.....	15
2.4 General tips on how to use the machine safely .....	15
2.5 Disposal.....	16
2.6 Packaging.....	16
2.7 Delivery .....	16
<b>3 Area of use and description</b> .....	<b>17</b>
3.1 Function .....	17
<b>4 Installation</b> .....	<b>17</b>
4.1 Positioning the machine .....	17
4.2 The power supply.....	17
4.3 The compressed air supply.....	17
4.4 Connection to an extraction unit.....	17
<b>5 Operating the Sandblasting machine <i>Dentablast Duo</i></b> .....	<b>18</b>
5.1 General Information .....	18
5.2 Setting the blasting pressure.....	18
5.3 Re-filling the sandblasting material .....	18
5.4 Sandblasting.....	18
5.5 Emptying the Sandblasting material.....	18
<b>6 Cleaning and maintenance</b> .....	<b>20</b>
6.1 Replacing hand insertion .....	20
6.2 Replacing the sandblasting nozzle .....	20
6.3 Replacing the fluorescent lamp.....	20
6.4 Replacing the viewing window .....	20
6.5 Replacing the hand piece tube .....	20
<b>7 Spare parts</b> .....	<b>21</b>
<b>8 Accessories</b> .....	<b>21</b>
<b>9 Technical Data</b> .....	<b>21</b>
<b>Diagram blasting medium – pressure</b> .....	<b>42</b>

# 1 Declaration of Conformity

## EEC-Declaration of Conformity

Dentaurum GmbH & Co. KG  
Turnstrasse 31  
D 75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EEC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of Machine: REF	Circulatory Sandblasting machine Dentablast Duo 094-540-00 and 094-541-00
From machine No.:	127-001 128-001

EC-Guidelines:	EC-Machine guidelines	89/392/EEC
	i.d.F. 91/368/EEC, i.d.F. 93/ 44/EEC	
	EG-EMV-guidelines	89/336/EEC
	i.d.F. 92/ 31/EEC	
	EG-low voltage guidelines	73/ 23/EEC

Applicatory Norm:	IEC 601-1-1-3
-------------------	---------------

Date / Signature of manufacturer:	18.06.2004
-----------------------------------	------------

Personal details to signatory:

**D**  
**DENTAURUM**  
GmbH & Co. KG  
Turnstraße 31 · 75228 Ispringen · Germany  
Telefon +49 7231 / 803-0 · Fax +49 7231 / 803-295

.....  
- i.V. Dipl.-Ing(FH) K.Merkle -  
Productionmanager

## 1.1 General Information

Thank you for having chosen a Dentaureum Machine. Please study these instructions for use carefully so that you may enjoy the benefits of using this new Sandblasting machine.

## 2 Safety instructions

 **Caution** Read through the instructions for installing and operating the unit carefully before switching it on.

### 2.1 Approved use

The Sandblasting machine **Dentablast Duo** is intended solely for homogeneous mixing of investment, plaster or silicone used for fabricating models in the dental laboratory. The unit should only be used for the purpose for which it is intended. Dentaureum GmbH & Co. KG, is not liable for damages resulting from any other use of the unit. Approved use also includes adhering to the instructions for use as well as servicing and carrying out maintenance work at regular intervals.

When a Dentaureum product is finally no longer in use the disposal regulations of that particular country apply. Dentaureum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

### 2.2 Symbols indicating risk

The following symbols are used in the instructions to indicate risks:

 **Warning** Indicates a potential danger to the life and health of the person. Disregarding this sign can critically affect a person's health and may even result in life-threatening injuries.

 **Caution** Indicates a potentially dangerous situation. Disregarding this sign can result in minor injury or damage to the equipment.

 **Note** Indicates general information on operating the unit

### 2.3 Guarantee and liability

Our general conditions of sale and delivery apply. Guarantee and liability claims for damages to persons or equipment are invalid if the damage is found to have resulted from any of the following:

- improper installation, operation, assembly or maintenance of the unit
- use of the unit other than for the purpose intended
- operation with faulty safety equipment or improperly installed or non-functioning safety and protective precautions
- unauthorised structural alterations to the unit.

### 2.4 General Information on the safe use of this machine

- Only use the machine together with an extractor.
- Keep your workplace clean. Untidiness increases the risk of accidents.
- For your own safety, please use the accessories and materials recommended in the operating instructions.
- Before servicing the machine, always remove the power supply.

## 2.5 Disposal

Note:

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaurum or the dental trade.

## 2.6 Packing

The packing provides only limited protection against wet, heat and any mechanical force. The packing should be disposed of in the proper manner.

## 2.7 Form of delivery

- 1 Sandblasting Machine **Dentablast Duo** - 230 V..... REF 094-540-00
- 115 V..... REF 094-541-00
- 1 Operating instructions and declaration of conformity
- 1 metal sandblasting nozzle  $\varnothing$  1.0 mm (yellow) ..... REF 094-552-00
- 1 metal sandblasting nozzle  $\varnothing$  1.2 mm (blue) ..... REF 094-562-00
- 1 mains cable
- 2 hand pieces
- 1 compressed air tube
- 1 connection attachment
- 1 foot control

### 3. Areas of Use and Description

#### 3.1 Function

The sandblasting machine **Dentablast Duo** is a fine sandblasting machine (a whirlpool style pressure blaster) for universal application. It is suitable for all conventional abrasive material up to a grain size of 250 µm. Both sandblasting machines can be filled with a variety of different abrasives; the pressure can be adjusted separately. All types of regular cast objects in the dental laboratory can be cleaned and superficially conditioned.

Other qualities include:

- easy handling
- doesn't take up much space
- high safety due to robust construction
- friendly service

### 4. Setting up the machine

#### 4.1 Positioning the machine

The machine can be placed onto a normal laboratory workbench.

#### 4.2 The power supply

The voltage information on the machine type sign must correspond with the mains voltage.

- This machine operates on a 230V power supply. (the 110V machine must be connected to a 110...120V power supply)
- The socket must be secured with a fuse.

 **Caution:** All electrical servicing either on the socket or the supply line must be carried out by an electrician.

#### 4.3 The compressed air supply

This machine can be connected to a compressor or a compressed air unit. The amount of pressure must not exceed max. 10 bar. The consumption is 136 – 160 l/min.

#### 4.4 Connection to an extraction unit

The machine is supplied with an extension socket for a 230V extraction unit, which runs automatically when the machine is in use. The extraction is essential for work involving a large amount of dust accumulation, eg. Devesting and cleaning.

The machine's max. performance of 1000W must not be exceeded. An extractor is not suitable as the filter capacity is too low.

The connection outlet on the sandblaster has a diameter of  $\varnothing$  40 mm. From here the sandblasting machine can be connected to an external extraction unit.

**Note** If an extraction unit is connected, inside the sandblasting machine must be removed.

## 5 Operating the sandblasting machine *Dentablast Duo*

### 5.1 General Information

 **Caution:** The machine must only be used when the sandblaster itself and the sandblasting material have reached room temperature. Otherwise there is a danger of condensation and subsequently lumpy sand.

### 5.2 Setting the blasting pressure

The Sandblasting machine is equipped with two pressure adjustment dials (black dials). The pressure can be set by pulling and turning the dial. The set pressure is shown on the pressure display. These show the actual abrasive container input pressure.

### 5.3 Re-filling the abrasive material

When the blasting pressure reduces, the abrasive material must be re-filled. Unscrew the cap of the abrasive material container. The vertical tube in the middle of the central abrasive container must be visible by at least 10 mm. Close the cap by screwing shut by hand.

### 5.4 Sandblasting

Hold the object to be sandblasted in the chamber under the sandblasting head and use the foot control to begin sandblasting. By letting the foot control go, the compressed air is halted and the sandblasting stops.

The following sandblasting nozzles may be used:

Metal sandblasting nozzle $\varnothing$ 1.0 mm	(yellow)	for a grain size of approx. 30 – 100 $\mu$ m
Metal sandblasting nozzle $\varnothing$ 1.2 mm	(blue)	for a grain size of approx. 75 – 250 $\mu$ m

Available as an accessory:

Metal sandblasting nozzle $\varnothing$ 0.8 mm	(black)	for a grain size smaller than 50 $\mu$ m (corundum)
--	---------	---

### 5.5 Emptying the abrasive material

The machine is equipped with a drawer which can be pulled out to empty the abrasive.

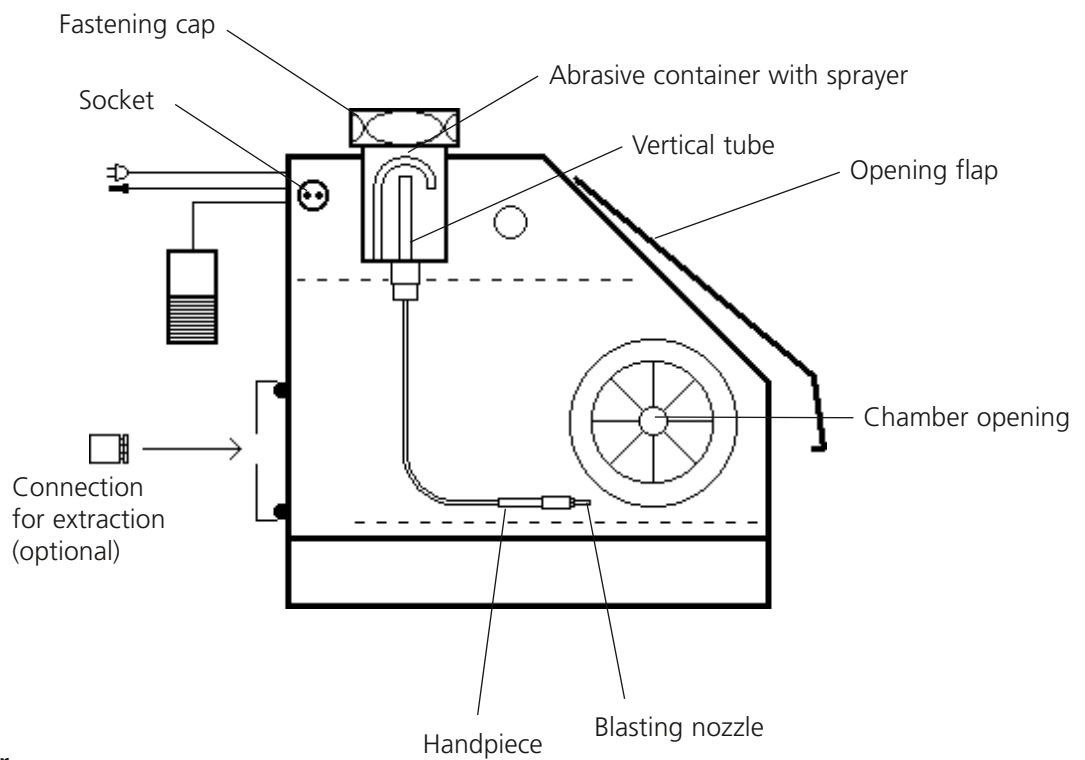
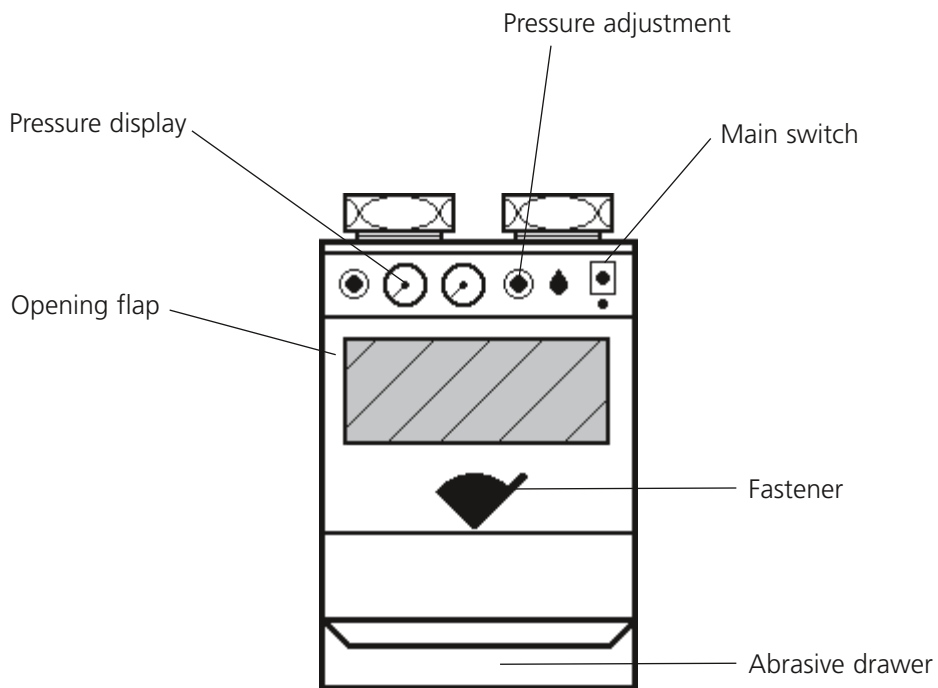
Fig 1: Sandblaster

## 6 Servicing

 **Warning:** Always switch off the machine and remove the power supply prior to all types of servicing and maintenance. If you have any technical questions regarding the machine, please contact our customer support team directly under +49 72 31 / 803-211.

### 6.1 Replacing the glove

Remove the screws from the insertion ring and pull the glove off from the metal ring. Pull the new glove from the inside towards the outside and over the metal ring. Place the glove with ring back into the chamber opening and attach the insertion ring with the screws lightly tightened. Adjust the glove and then carefully tighten the screws alternately.

### 6.2 Replacing the sandblasting nozzle

Loosen the nozzle from the hand piece using a wrench no. 12 or a ring spanner. Remove the nozzle and replace with a new one. Screw the hand piece and nozzle back together. The nozzle must sit straight in the tube.

### 6.3 Replacing the fluorescent lamp

Remove the round black lid on the left hand side of the machine by loosening the screws. Pull the fluorescent lamp (G23 11W) from its socket and replace with a new one.

### 6.4 Replacing the viewing window

Unscrew the support splints from the base of the frontal flap. Remove the glass pane and replace with a new one. Screw the support splints back on.

### 6.5 Replacing the hand piece tube

Loosen the brass cap at the end of the tube inside the blasting chamber, remove the tube and replace with a new one. Turn the brass cap just slightly. The end of the tube must be widened beforehand, eg. using pointed pliers or heat.

## 7. Spare parts

Viewing window.....	REF 094-533-00
1 metal sandblasting nozzle $\varnothing$ 1.0 mm (yellow).....	REF 094-552-00
1 metal sandblasting nozzle $\varnothing$ 1.2 mm (blue).....	REF 094-562-00
Hand piece tube.....	REF 094-543-00
Gloves .....	REF 094-537-50
Fluorescent lamp (G23-11W).....	REF 094-534-00

## 8. Accessories

1 metal sandblasting nozzle $\varnothing$ 0.8 mm (black) .....	REF 094-542-00
(for a grain size smaller than 50 $\mu\text{m}$ – corundum )	
Extraction unit .....	REF 094-550-00
Blasting medium, white: aluminium oxide 99.6 %:	
Grain size 50 (44 – 74 $\mu\text{m}$ ) 4.5 kg.....	REF 128-017-00
Grain size 100 (88 – 125 $\mu\text{m}$ ) 4.5 kg.....	REF 128-020-00
Grain size 250 (210 – 297 $\mu\text{m}$ ) 10 kg.....	REF 128-016-00
Blasting medium, brown:	
Grain size 250 (210 – 297 $\mu\text{m}$ ) 10 kg.....	REF 128-011-00
Polishing beads	
Grain size 75 (50 – 105 $\mu\text{m}$ ) 4.5 kg.....	REF 128-211-00

## 9 Technical Data

Voltage	094-540-00	230 V
	094-541-00	115 V
Frequency		50/60Hz
Average power consumption		30 – 100 W
Compressed air connection		0...10 bar
Air consumption per hour		136 – 160 l
Measurements [width x height x depth]		340 x 470 x 480 mm
Weight		24 kg
Container for blasting medium, capacity		0.5 l each

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