Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

Dentablast Duo

Sandstrahlgerät

Sandblaster

Sableuse

Arenadora

REF 094-540-00

REF 094-541-00

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Druckdatum: 05.10.15

Contents

1. Manufacturer	11
2. Safety instructions.....	11
2.1 Appropriate handling.....	11
2.2 Danger symbols and markings.....	11
2.3 Warranty and liability.....	11
3. General notes and disposal	11
4. Packing.....	12
5. Delivery	12
6. Areas of use and description.....	12
4.1 Function	12
7. Installation.....	12
7.1 Positioning the machine	12
7.2 The power supply.....	13
7.3 The compressed air supply.....	13
7.4 Connection to an extraction unit	13
8. Operating the sandblasting machine Dentablast.....	13
8.1 General information	13
8.2 Setting the blasting pressure.....	13
8.3 Re-filling the abrasive material	13
8.4 Sandblasting.....	13
9. Maintenance.....	15
9.1 Replacing the gloves.....	15
9.2 Replacing the sandblasting nozzle	15
9.3 Replacing the lamp.....	15
9.4 Replacing the viewing window	15
9.5 Wechseln des Handstückschlauches.....	15
9.6. Blasting performance too low.....	15
9.7 Emptying the abrasive drawer.....	15
10. Spare parts.....	16
11. Accessories.....	16
12. Technical data.....	16
13. Quality information	16
14. EEC-Declaration of Conformity	17

Dear customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. Safety information

 **Read through the instructions for installing and operating the unit carefully before switching it on.**

2.1 Intended use

The sandblasting machine Dentablast Duo is intended solely for homogeneous mixing of investment, plaster or silicone used for fabricating models in the dental laboratory. The unit should only be used for the purpose for which it is intended. Dentaureum GmbH & Co. KG, is not liable for damages resulting from any other use of the unit. Approved use also includes adhering to the instructions for use as well as servicing and carrying out maintenance work at regular intervals.

When a Dentaureum product is finally no longer in use the disposal regulations of that particular country apply. Dentaureum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

2.2 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

 **This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.**

 **This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and / or other property.**

 **This symbol gives important information on the correct use of the unit. Disregarding this information may cause the unit to malfunction.**

2.3 Guarantee and liability

Our general conditions of sale and delivery apply. Guarantee and liability claims for damages to persons or equipment are invalid if the damage is found to have resulted from any of the following:

- improper installation, operation, assembly or maintenance of the unit
- use of the unit other than for the purpose intended
- operation with faulty safety equipment or improperly installed or non-functioning safety precautions
- unauthorized structural alterations to the unit.

3. General notes and disposal

- Only use the machine together with an extractor.
- Keep your workplace clean. Untidiness increases the risk of accidents.
- For your own safety, please use the accessories and materials recommended in the operating instructions.
- Before servicing the machine, always remove the power supply.

When a Denta-urum product is finally no longer in use, the disposal regulations of that particular country apply. Denta-urum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

**Note:**

This symbol indicates products marked with it should not be disposed off together with household garbage. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Denta-urum or the dental trade.

4. Packaging

The packaging provides only limited protection against wet, heat and any mechanical force. The packaging should be disposed of in the proper manner.

5. Delivery

1	Sandblasting machine Dentablast Duo	- 230 VREF 094-540-00
		- 115 VREF 094-541-00
1	Operating instructions and declaration of conformity	
1	Metal sandblasting nozzle \varnothing 1.0 mm (yellow)	REF 094-552-00
1	Metal sandblasting nozzle \varnothing 1.2 mm (blue).....	REF 094-562-00
1	Mains cable	
2	Hand pieces	
1	Compressed air tube	
1	Connection attachment	
1	Foot control	

6. Areas of use and description

6.1 Function

The sandblasting machine Dentablast Duo is a fine sandblasting machine (a whirlpool style pressure blaster) for universal application. It is suitable for all conventional abrasive material up to a grain size of 250 μm . Both sandblasting machines can be filled with a variety of different abrasives; the pressure can be adjusted separately. All types of regular cast objects in the dental laboratory can be cleaned and superficially conditioned.

Other qualities include:

- easy handling
- doesn't take up much space
- high safety due to robust construction
- friendly service

7. Installation

7.1 Positioning the machine

The machine can be placed onto a normal laboratory workbench.

7.2 The power supply

The voltage information on the machine type sign must correspond with the mains voltage.

- This machine operates on a 230 V power supply. (the 110 V machine must be connected to a 110...120 V power supply)
- The socket must be secured with a fuse.

Caution: All electrical servicing either on the socket or the supply line must be carried out by an electrician.

7.3 The compressed air supply

This machine can be connected to a compressor or a compressed air unit. The amount of pressure must not exceed max. 10 bar. The consumption is 136 – 160 l/min. We recommend the use of a maintenance unit with water separator.

7.4 Connection to an extraction unit

The machine is supplied with an extension socket for a 230 V extraction unit, which runs automatically when the machine is in use. The extraction is essential for work involving a large amount of dust accumulation, e.g. de-vesting and cleaning.

The machine's max. performance of 1000 W must not be exceeded. An extractor is not suitable as the filter capacity is too low. The connection outlet on the sandblaster has a diameter of \varnothing 40 mm. From here the sandblasting machine can be connected to an external extraction unit.

Note: If an extraction unit is connected, the particle filter inside the sandblasting machine must be removed.

8. Operating the sandblasting machine Dentablast Duo

8.1 General Information

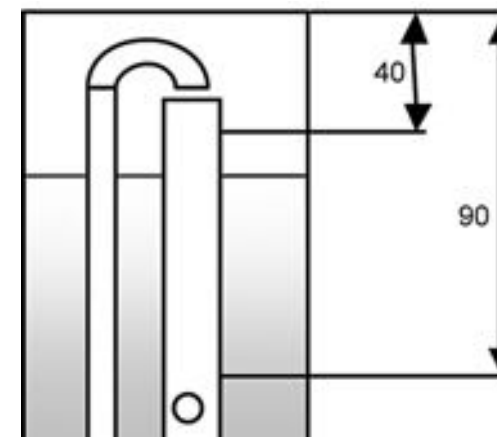
Caution: The machine must only be used when the sandblaster itself and the blasting abrasive have reached room temperature. Otherwise there is a danger of condensation and subsequently lumpy sand.

8.2 Setting the blasting pressure

The sandblasting machine is equipped with two pressure adjustment dials (black dials). The pressure can be set by pulling and turning the dial. The set pressure is shown on the pressure display. These show the actual abrasive container input pressure.

8.3 Refilling the blasting abrasive

To fill with sandblasting material, rotate the fastening cap of the abrasive container counterclockwise and lift it upwards. Fill with sandblasting material at a maximum of 40 mm below the container edge (see illustration). Replace the fastening cap on the container and close by rotating clockwise.

Filling level of the sandblasting material
(in mm measured from the upper edge)

minimum: 90 mm

maximum: 40 mm

8.4 Blasting

Hold the object to be blasted in the chamber under the blasting head and use the foot control to begin blasting. By letting the foot control go, the compressed air is halted and the blasting stops. A bit of compressed air still flows in.

The following blasting nozzles may be used:


Metal blasting nozzle \varnothing 1.0 mm (yellow) for a grain size of approx. 30 – 110 μm

Metal blasting nozzle \varnothing 1.2 mm (blue) for a grain size of approx. 75 – 250 μm

Available as an accessory:

Metal blasting nozzle \varnothing 0.6 mm (black) for a grain size smaller than 50 μm (corundum)

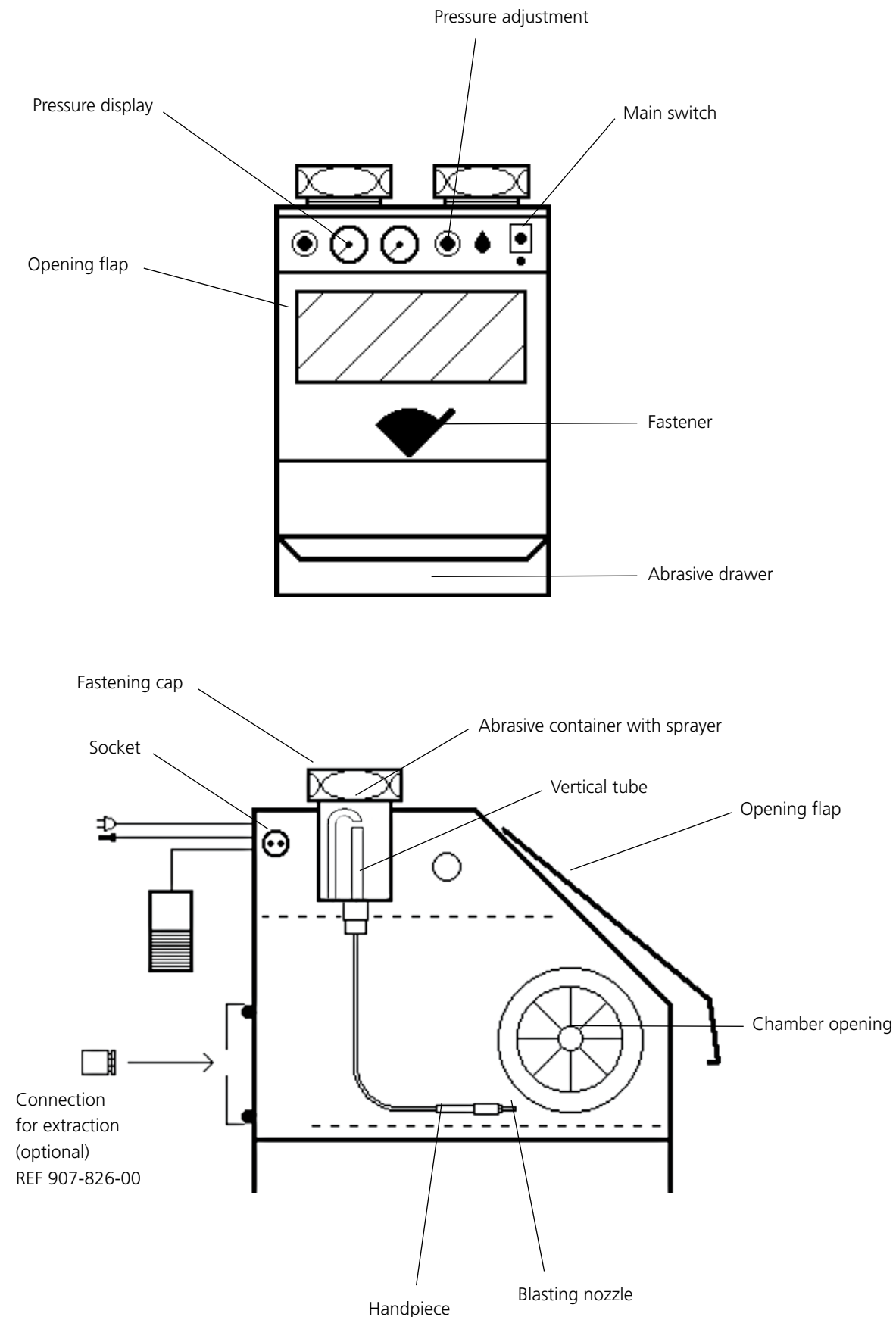Fig 1: Sandblaster

9. Service

⚠ Warning: Always switch off the machine and remove the power supply prior to all types of servicing and maintenance. If you have any technical questions regarding the machine, please contact your local representative.

9.1 Replacing the gloves

Remove the screws from the insertion ring and pull the gloves/cuffs off from the metal ring. Pull the new gloves/cuffs from the inside towards the outside and over the metal ring. Place the gloves/cuffs with ring back into the chamber opening and attach the insertion ring with the screws lightly tightened. Adjust the gloves/cuffs and then carefully tighten the screws alternately.

9.2 Replacing the sandblasting nozzle

Loosen the nozzle from the hand piece using a wrench no. 12 or a ring spanner. Remove the nozzle and replace with a new one. Screw the hand piece and nozzle back together. The nozzle must sit straight in the tube.

9.3 Replacing the lamp

⚠ Turn off the machine and pull the plug. Unscrew the two fastening caps. Take off the upper cover by unscrewing the upper two screws using a screw driver. Unscrew the screws of the white reflector and lift the reflector. Now, you can access the lamp. Unscrew the lamp (LED 65W, E27) anti-clockwise and insert new lamp.

9.4 Replacing the viewing window

Unscrew the support splints from the base of the frontal flap. Remove the glass pane and replace with a new one. Screw the support splints back on.

9.5 Replacing the hand piece tube

Loosen the brass cap at the end of the tube inside the blasting chamber, remove the tube and replace with a new one. Turn the brass cap just slightly. The end of the tube must be widened beforehand, e.g. using pointed pliers or heat.

9.6 Blasting performance too low

With reduced blasting performance, check the filling level of the abrasive container and refill it, if necessary (see chapter 8.3). If the blasting performance is still too low, check the permeability of the blasting nozzle.

9.7 Emptying the abrasive drawer

The unit is equipped with an abrasive drawer, which is pulled out forward for emptying.

10. Spare parts

Viewing window.....	REF 094-533-00
1 metal sandblasting nozzle \varnothing 1.0 mm (yellow)	REF 094-552-00
1 metal sandblasting nozzle \varnothing 1.2 mm (blue)	REF 094-562-00
Hand piece tube	REF 094-543-00
Cuffs	REF 094-537-50

11. Accessories

1 metal sandblasting nozzle \varnothing 0.6 mm (black)	REF 094-542-00
(for a grain size smaller than 50 μ m – corundum)	
Extraction unit	REF 094-550-00
Blasting medium, white: aluminium oxide 99.6 %:	
Grain size 50 (44 – 74 μ m).....	4.5 kg..... REF 128-017-00
Grain size 100 (88 – 125 μ m).....	4.5 kg..... REF 128-020-00
Grain size 250 (210 – 297 μ m).....	10 kg..... REF 128-016-00
Blasting medium, brown:	
Grain size 250 (210 – 297 μ m).....	10 kg..... REF 128-011-00
Polishing beads	
Grain size 75 (50 – 105 μ m).....	4.5 kg..... REF 128-211-00
Connection for extraction	REF 907-826-00

12. Technical data

Voltage	REF 094-540-00.....230 V
	REF 094-541-00.....115 V
Frequency50/60Hz
Average power consumption30 – 100 W
Compressed air connection0...10 bar
Air consumption per hour136 – 160 l
Measurements [width x height x depth]340 x 470 x 480 mm
Weight24 kg
Container for blasting medium, capacity0.5 l each

Subject to change!

13. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

14.

EC-Declaration of Conformity

DENTAURUM GmbH & Co. KG
Turnstr. 31
75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of unit:	Dentablast Duo REF 094-540-00 / 094-541-00
Type of unit:	sandblast unit
Start with Unit No.:	127-001 128-001
EC guidelines:	Guideline for machine 89/392/EEC
	Electromagnetic compatibility 89/336/EEC
	Electrical equipment used within certain voltage limits 73/23/EEC
	Modified Guideline 93/68/EEC
Applied unified standards:	IEC 601-1-1-3

Date and manufacturers signature: 01.01.2010
Signatory:

D
DENTAURUM
GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany
Telefon +49 7231 / 803-0 · Fax +49 7231 / 803-295

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