

Our technical advisors would be pleased to help you with any questions regarding the use of our products.

**Hotline +49 72 31 / 803-410**

Information and instructions for using Dentaaurum products are available on the Internet at:

[www.dentaaurum.de](http://www.dentaaurum.de)

Date of Information: 02/07

# Power Liquid



## Power Liquid

Simple and accurately controlled expansion

989-872-20 Printed by Dentaaurum Germany 02/07

**D**  
DENTAURUM

**D**  
DENTAURUM

# Control the expansion simply and accurately

# Power Liquid

## Extremely flexible expansion

### To produce a wider fitting

Power Liquid can be used for various investment materials from Dentaureum in order to achieve an accurate degree of expansion. The highly concentrated mixing liquid is diluted according to the desired level of expansion. Power liquid can be optimally adjusted to suit each individual working environment.

## Power in the bottle

### Economical and space saving

The concentrated liquid reduces up to 40% of bottle volume during packaging. This is environmentally friendly, reduces transport costs and saves space in your storage room. The mixing liquid concentrate can be quickly restored with the addition of distilled water.

## Versatile use

### For use with various investment materials

Whether rema® Exakt, rema® Exakt F, rema® dynamic S, Castorit® super C or Trivest - all these investment materials can be used together with the Power Liquid mixing concentrate.

## Dilution recommendation

### From concentrate to mixing liquid

*Investment material*    *Mixing ratio*

rema® Exakt:	60% Power Liquid + 40% Aqua dest.
rema® Exakt F:	60% Power Liquid + 40% Aqua dest.
rema® dynamic S:	55% Power Liquid + 45% Aqua dest.
Castorit® super C:	60% Power Liquid + 40% Aqua dest.
Trivest:	60% Power Liquid + 40% Aqua dest.

---

**Power Liquid**

1000 ml

**REF 105-502-00**