remanium® star MD II

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EN

remanium® star MD II

CoCrW – Dental alloy based on cobalt for metal/ceramic bonding, type 3 according to DIN EN ISO 22674 / DIN EN ISO 9693 for milling

Composition (% by mass)

Co	Cr	W	Si
60,5	28,0	9,0	1,5

Additional elements < 1%: Mn, N, Nb

This product contains cobalt.

Free from nickel, beryllium, lead, cadmium and iron.

Technical data

Yield strength*	R _{p,0,2}	320 MPa
Tensile strength*	R _m	506 MPa
Hardness*	H	281 HV10
Elongation at rupture*	A _s	5,9%
Modulus of elasticity*	E	202 GPa
Density	ρ	8,5 g/cm ³
Solidus temperature	T _s	1320 °C / 2408 °F
Liquidus temperature	T _l	1420 °C / 2588 °F
CTE (25 °C - 500 °C / 77 - 932 °F)	α	14,1 x 10 ⁻⁶ K ⁻¹

* Variations of ± 10% are possible depending on the batch.

Dear customer,

Thank you for choosing a quality product from Dentaurum. It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative. As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstraße 31 | 75228 Ispringen | Germany

2. General product description

Non-precious alloy for the manufacture of medical devices by means of the subtractive manufacturing process.

3. Area of application

remanium® milling blanks for subtractive manufacturing are cobalt-based dental alloys. remanium® star MD II is particularly suitable for the manufacture of constructions to be produced by milling in the field of crown and bridge technology, such as crowns, bridges and abutments, in the form of fixed and removable restorations, and for metal ceramics.

4. Digital modeling

The dental products are modeled using a CAD software which has been approved for this purpose. Construction must be in line with dental regulations in order to fulfil mechanical requirements.

5. Milling

Milling can be carried out with most commercially available CAD/CAM milling machines or hand-guided copy milling machines. Suitable tools should be selected according to the machine manufacturer's operating instructions. Before starting the milling process, ensure that the tools are clamped correctly and securely. Machining should be carried out with the milling parameters recommended by the machine manufacturer.

6. Post treatment

After completion of the milling operation, the framework must be removed from the blank. Support residues can be removed by means of finely toothed hard metal burs. Any particles or dust remaining on the surface must be removed with compressed air.

7. Finishing

The finishing of the framework and the surfaces should be carried out by means of finely-toothed hard metal burs, ceramic-bonded grinding points or diamond grinding points.

8. Welding

Suitable welding processes such as laser welding and TIG impulse welding produce a solder-free, mechanically strong and corrosion-resistant joint. It is important to observe the framework design, the surface structures, the welding sequence and the welding parameters of each individual laser machine. Suitable welding wires are available as filler material for all remanium® alloys, e.g. CoCr welding wire 0,25 mm (REF 528-215-10), 0,35 mm (REF 528-210-10) and 0,5 mm (REF 528-200-10).

9. Soldering

Soldering should be avoided if possible to reduce the number of materials. If, however, it is necessary to solder, please use a suitable solder and flux which are appropriate for the composition and melting temperature of the alloy used, e.g. CoCrMo-Sold 1 (REF 102-306-00) in conjunction with rema® Flux 1 (REF 102-304-00).

10. Ceramic veneering

remanium® bonding alloys can be veneered with all suitable ceramic masses, e.g. ceraMotion® Me (Dentaurum), as long as the CTE is suitable for the alloy. The highest recommended firing temperature is 980 °C / 1796 °F. The ceramic should cool down over a long period in the furnace unless the ceramic manufacturer has other specifications.

11. Contraindications and adverse reactions

Signs of intolerance to non-precious remanium® alloys are extremely rare if the manufacturer's Instructions for use are adhered to. If the patient has a proven allergy against any component within the alloy, this alloy must not be used for safety reasons. There have been individual reportings of local irritations which were electrochemically induced. If various alloys have been used, it is possible that galvanic effects may occur.

Exposure to dust and vapors may cause irritation to the eyes and/or respiratory tract. Individual alloy components may have a carcinogenic effect. Please adhere to warnings and safety data sheets. In the State of California elements of this alloy are known to be carcinogenic.

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