- dinamique

m

dinamique

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Self-ligating brackets –
dinamique® m, dinamique® c
- Dear customer**

Thank you for choosing a quality product from Dentaurum. In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. Instructions for use cannot describe every eventuality and possible application.

In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.
1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General description

dinamique® m self-ligating metal brackets (Fig. 1) and dinamique® c self-ligating ceramic brackets (Fig. 2) meet the requirements of modern orthodontics.

3. Processing

3.1. Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding.

3.2. Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding. For tooth conditioning, adhere to the manufacturer's instructions of the bracket adhesive.

3.3. Selecting bracket adhesive

dinamique® brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC Ic (REF 163-111-00), ConTec Go! (REF 163-321-00) and SmartBond® (REF 163-506-00). Please follow the instructions of the bracket adhesive manufacturer and observe the table „Application for adhesives“ in our Orthodontics Catalog.

3.4. Handling brackets

Grip the dinamique® m/dinamique® c brackets mesially-distally using clamping tweezers (e.g. REF 025-277-00) to remove them from the packaging and handle according to the following steps. The base of the dinamique® m/dinamique® c brackets should not be touched with your fingers or contaminated in any way, as this could otherwise detrimentally affect the bond.

3.5. Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

3.6. Placing and aligning brackets

After applying the adhesive, place the dinamique® m/dinamique® c bracket in the usual way onto the tooth.
- ⚠ Caution:

Avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary, a palatal bite raise can be used. Completely remove any excess adhesive before it has cured.

⚠ Caution:

Ensure that no adhesive material enters the closing mechanism (clip) on the gingival and occlusal side of the bracket, as this could restrict or destroy the functionality of the lid. Removing the excess adhesive is particularly important to ensure easy debonding at the end of treatment.

3.7. Using the clip/ligating the archwire

dinamique® m/dinamique® c brackets are delivered in open form so that you can ligate the archwire directly after the bonding. Press the archwire with one finger into the slot of the bracket and close the clip with another finger. To close the clip, press the occlusal surface of the clip with your finger (Fig. 3). Alternatively, you can press the archwire into the bracket slot with the slot-like end of the sl-instrument (REF 023-276-00) and close the clip with your finger. To open the dinamique® m/dinamique® c brackets, insert the tip of the sl-instrument in the vestibular opening of the clip and carefully pull towards occlusal. The clip opens (Fig. 4). Always use the sl-Instrument (REF 023-276-00) to open the brackets!

⚠ Caution:

Open the clip just wide enough to allow access to the slot. Do not open further, as it can lead to deformation and fracture of the clip!

3.8. Bracket debonding

We recommend removing the ligatures and archwires before debonding. Let the patient bite on a cotton swab and remove the brackets with straight or angled debonding pliers (e.g. REF 004-346-00 or REF 004-347-00). Grab into the space between tooth and bracket. You can grab the bracket diagonally or mesial-distally. Carefully press the pliers together and keep increasing the pressure to debond the bracket from the tooth (Fig. 5 and 6).

⚠ Caution:

Do not turn or pull the bracket from the tooth! dinamique® m/dinamique® c brackets will come off of the tooth without the adhesive, leaving a layer of adhesive on the tooth. The adhesive layer can be removed with carbide burs (e.g. REF 123-603-00 or REF 123-604-00) and/or adhesive residue remover (e.g. REF 123-700-00 – REF 123-703-00).
4. Composition

Information about the composition is included in the material list, see catalog or www.dentaurum.com.

5. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components.

6. Storage and shelf life

Store dinamique® m and dinamique® c brackets in closed packaging.

7. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

8. Availability

For the complete range of dinamique® m/dinamique® c brackets, please see our catalog or www.dentaurum.com.

9. Information for the doctor/patient

■ This product contains nickel and chrome. If the patient has a known allergy to one of the metals, this product should not be used. Further risks or side effects of the material of dinamique® m and dinamique® c are not known.

■ As the material 1.4542 of the dinamique® m has a low magnetizability, we recommend removing the brackets prior to an MRT.

10. Additional tips

■ Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog information.

■ Store dinamique® m and dinamique® c brackets in closed packaging.

■ There are no particular precautions to be taken when disposing of dinamique® m and dinamique® c brackets after treatment.

11. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon Dentaurum's own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

12. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

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