Transluzenter Idealbogen pearl
Translucent archwires pearl

CE 0483

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Matrixcode

Dear Customer

Thank you for choosing this high-quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaaurum.de.

1. Manufacturer

Dentaaurum GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany

2. General product description / field of application

The translucent archwires pearl are used in conjunction with aesthetic brackets, e.g. discovery® pearl, to provide force to the teeth to effect movement in orthodontic treatment.

2.1. Mechanical properties

The mechanical properties of the translucent archwires pearl are comparable to those of modern nickel titanium archwires. Please consider that the force behavior of the 0,46 mm/18 translucent archwires pearl complies with the 0,40 mm/16 nickel-titanium archwire.

2.2. Instruments

In general, standard orthodontic instruments can be used. Please avoid any serrated or sharp-edged instruments, as these can destroy the mechanical properties of the translucent archwires pearl and lead to breakages.

2.3. Brackets

The translucent archwires pearl can be used with most aesthetic ceramic brackets. An optimal aesthetic effect can be achieved in combination with discovery® pearl brackets and Dentalastics® pearl plastic ligatures.

Exceptions: Self-ligating brackets with a clamping lock, e.g. Clarity™ SL (3M Unitek) Ligating the translucent archwires pearl into this bracket type leads to destruction of the mechanical properties!

Ceramic and plastic brackets with a metal slot should not be used as the metal slot is visible due to the translucent nature of the archwires pearl and thus nullifies the optimal aesthetic effect.

2.4. Ligating the translucent archwires pearl

Attention – risk of breakage:

Ligating the translucent archwires pearl must be done with extra care, to prevent breakage. We recommend pre-stretching our Dentalastics® pearl before ligating and in case of certain malpositions of the teeth to initially place the ligature only on two wings. Crack formation or white discoloration of the archwire is a clear indication that it has been bent beyond its limits.

2.5. Formability

Attention – risk of breakage:

The translucent archwires pearl come preformed. They cannot be bent into shape. The formation of loops, crimps and bends leads to the destruction of the mechanical properties. Crack formation or white discoloration of the archwire is a clear indication that it has been bent beyond its limits.

2.6. Deflection

Attention – risk of breakage:

The translucent archwires pearl are limited by how much they can be deflected before they break. The bending angle should not exceed 60°. Crack formation or white discoloration of the archwire is a clear indication that it has been bent beyond its limits.

If – due to the malocclusion of the teeth – larger bends are necessary, we recommend a partial fixation of the archwire to the bracket slots or rebonding the brackets.

2.7. Protection of the archwire

In areas of long, non-supported archwire segments or distinct overbite/crossbite, where the translucent archwires pearl, can come into contact with the teeth during mastication, we recommend the use of plastic sleeves.

2.8. Interval of control

The translucent archwires pearl tend to break due to high compression during the tooth movement. Therefore it is recommended to shorten the interval of intraoral control to four weeks.

3. Insertion of the archwire

Ligating the translucent archwires pearl is carried out in a similar way to the metal archwires, except for the ligation of the archwire ends. At the archwire end, it is recommended that the orthodontic practitioner proceeds as follows:

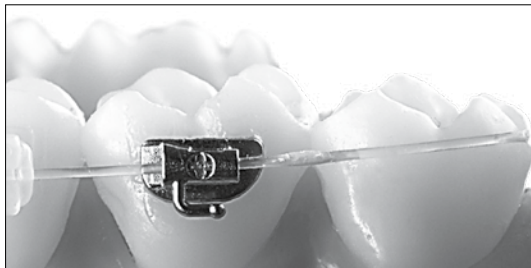
Fig.1: Place the archwire in the buccal molar tube.

- Pull the translucent archwires pearl distal to the buccal tube (Fig. 2).

Cutting and fixing the translucent archwire pearl:

- After having placed the archwire end in the buccal molar tubes cut it off slightly protruding (Fig. 3), using a ligature cutter (e.g. REF 014-151-00), not a distal cutter.

Instead of a cinch-back bend place a drop of adhesive at the archwire end or anywhere along the length of the archwire to prevent unwanted sliding (Fig. 4, black arrow).

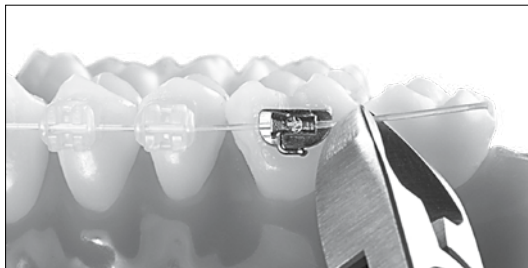
Fig. 3: Cut off the archwire slightly protruding.

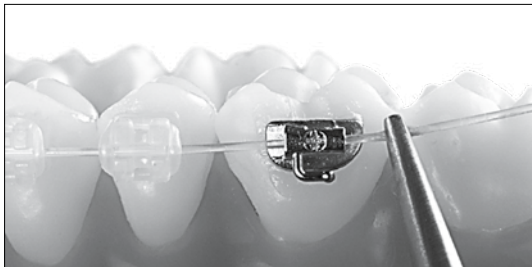
Fig. 2: Pull the archwire distal to the molar tube.

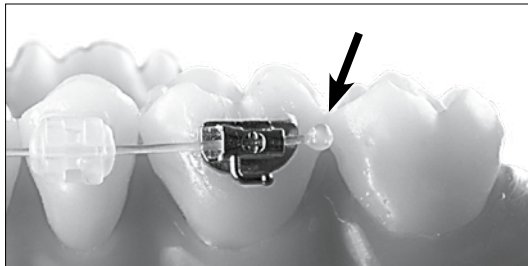
Fig. 4: Place a drop of adhesive at the archwire end (black arrow).

4. Information regarding composition

The composition is available in the materials list, see catalog or www.dentaurum.de.

5. Safety information

The translucent archwires pearl is made from a fiber-reinforced polymer plastic material. The novel material necessitates the above mentioned measures to be observed when using the wire.

The product should not be used, if there is a known allergic reaction to one or more of the material components.

6. Information regarding single-use products

The translucent archwires pearl is intended for single use only. Reconditioning of a translucent archwires pearl that has already been used (recycling) and its reuse on a patient is not permitted.

7. Sterilization

The translucent archwires pearl may be sterilized using the standard cold sterilization procedure or autoclaving at a standard autoclaving temperature of 121 °C/250 °F.

8. Availability

∅ 0.46 mm / 18	5 pieces	REF 766-906-00
----------------	----------	-----------------------

9. Quality information

Dentaurum guarantees the perfect quality of its products. The contents of these instructions for use are based on our experience. The user is solely responsible for processing the product. As Dentaurum cannot influence the use of the product by the user, no liability can be accepted for inferior results caused by improper use of the product.

10. Explanation of symbols used on the label

Order number

Batch description

CE Mark MP Class IIa

→ www.dentaurum.de

Instructions for use available in the internet

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