CE 0483

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Instructions for use

Mode d'emploi

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Instruction for use discovery® pearl brackets

Dear Customer

Thank you for choosing this high-quality product from Dentaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise. Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaurum.de.

1. Manufacturer

Dentaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

2. General description

discovery® pearl is the aesthetic alternative to our discovery® metal brackets. This ceramic bracket – manufactured using a CIM (Ceramic Injection Molding) process – consists of highly pure polycrystalline aluminum oxide. Due to its translucency, discovery® pearl assumes the natural shade of the tooth and is therefore virtually invisible. discovery® pearl meets all the requirements of modern orthodontic treatment.

discovery® pearl ceramic brackets are supplied with an identification system (positioning guides) for the "McLaughlin-Bennett-Trevisi* 22 straight wire technique". The bracket base also has laser marking with the respective FDI notation. This allows for easy identification and placement of the brackets.

Information on the use of the identification system (positioning guides) can be found at the end of these instructions for use (Section 8).

* The Dentaurum version of this prescription is not claimed to be a duplication of any other, nor does Dentaurum imply that it is endorsed in any way by Drs. McLaughlin, Bennett and Trevisi.

3. Working stages

3.1. Patient consultation

Inform the patient about the possible consequences and risks involved in treatment using brackets, in particular during debonding. This applies especially in the case of teeth with enamel that already exhibits signs of initial damage (cracks, crazing, splitting off etc.).

3.2. Tooth conditioning

Adhere to the bracket adhesive manufacturer's instructions for conditioning the teeth.

3.3. Bracket adhesive selection

discovery® pearl brackets can be bonded using all commercially available bracket adhesives. We recommend our ConTec SE adhesive (REF 163-550-00).

Adhere to the manufacturer's instructions for the bracket adhesive as well as the Adhesive System Table in our orthodontic catalog.

3.4. Removal and handling

Grip the discovery® pearl brackets using clamping tweezers (e.g. REF 025-277-00) mesio-distally and at the bar of the positioning guide, to remove them from the packaging and handle according to the following steps. The base of the discovery® pearl brackets should not be touched with your fingers or contaminated in any way, as this could otherwise detrimentally affect the bond.

3.5. Application of the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, but enough to cover the entire bracket base. This guarantees an optimum bond.

3.6. Placement and alignment of the bracket

After applying the adhesive, place, align and press the bracket in the usual way onto the tooth until the adhesive exudes from the sides.

It is essential to avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary a palatal bite raise can be used e.g. Bite-Bumper™ (REF 705-000-00).

Completely remove the exuded excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

3.7. Ligating the archwires

For the initial treatment phase we recommend the use of super elastic Tensic®, rematitan® LITE or translucent pearl archwires.

Do not use force to ligate the archwires. Do not torque the archwires when they are ligated.

Once the adhesive has cured (curing time according to the adhesive manufacturer's instructions) and the placement aids have been removed, ligating can be completed in the usual way using wire or plastic ligatures (e.g. Dentalastics® pearl REF 774-563-00).

Note: Careful ligation avoids damage to the brackets!

3.8. Bracket debonding

For debonding the discovery® pearl brackets we recommend the special discovery® pearl debonding instrument, REF 019-001-00, which grips the body of the bracket directly at the base. The bracket is gently removed from the bracket adhesive by slightly tipping it sideways in the direction of the slot and archwire.

Alternatively, the straight bracket removal pliers (REF 004-346-00) can be used for debonding the discovery® pearl brackets. Place the blades of the bracket removal pliers mesio-distally on the bracket base, **directly on the surface of the tooth**. The bracket is gently removed from the tooth by slightly twisting the pliers. The archwire can be removed beforehand – though it is not essential.

Note:

Despite their small, delicate shape there is no risk of fracture with discovery® pearl brackets provided they are used correctly. Use of other instruments can result in bracket fracture. If the bracket fractures during incorrect debonding, we recommend carefully removing the fractured segments and the adhesive from the enamel using bracket debonding pliers (e.g. REF 003-349-00) or a scaler (e.g. REF 027-349-00).

Use care when debonding the segments, as there is a risk of the enamel chipping. Use an aspirator and wear protective glasses.

4. Information about the composition

Information about the composition is included in the materials list, see catalog or www.dentaurum.de.

5. Safety information

The product should not be used if there is a known allergy to one or more of the ingredients.

6. Storage and shelf-life

Store discovery® pearl brackets in closed packaging. discovery® pearl brackets do not have a limited shelf life.

7. Information for single-use products

discovery® pearl brackets are intended for single use only. Reconditioning of discovery® pearl brackets that have already been used (recycling) and reuse on patients is not permitted. Micro cracks may result from loading during treatment and in particular bracket removal that could cause fractures in the ceramic during reuse!

8. Notes about the identification system

For the "McLaughlin-Bennett-Trevisi* 22 straight wire technique" (Positioning guides).

* The Dentaurum version of this prescription is not claimed to be a duplication of any other, nor does Dentaurum imply that it is endorsed in any way by Drs. McLaughlin, Bennett and Trevisi.

8.1. The color of the positioning guides and the tooth symbol indicate the tooth/bracket combination.

8.2. Maxilla: right = red
left = green
Mandible: right = yellow
left = blue

8.3. The occlusal-gingival distance can be determined by means of the occlusal indicator. The distance between the occlusal end and the center of the slot is 6 mm.

8.4. The tooth symbol always points from occlusal to distal.

8.5. Removing the positioning guides

Once the adhesive has cured (curing time according to the adhesive manufacturer's instructions), grip the mid-section of the positioning guides using tweezers or Weingart pliers (e.g. REF 003-120-00) and remove.

The practical FDI notation on the base also aids clear identification.

9. Additional tips

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog information.
- There are no particular precautions to be taken when disposing of discovery® pearl brackets after treatment.

10. Delivery program

For the complete range of discovery® pearl products please see our orthodontics catalog.

11. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

12. Explanation of symbols used on the label

Order number

12345 Batch description

0483 CE Mark MP Class IIa

→ www.dentaurum.de Instructions for use available in the internet

Not for reuse

Codablock barcode

Matrix code

Date of information: 08/12
Subject to modifications

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Dentaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germania