Gebrauchsanweisung | Instructions for use  
Mode d'emploi | Modo de empleo | Modalità d'uso

CE 0483

tomas®-auxiliaries

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## **Dear customer**

Thank you for choosing a quality product from Dentaaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at [www.dentaaurum.com](http://www.dentaaurum.com), even though you frequently use the same product.

### **1. Manufacturer**

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

### **2. General description**

A temporary, reliable skeletal anchorage option is created for orthodontic treatment with the aid of an endosteally anchored mini-implant (tomas<sup>®</sup>-pins). With the tomas<sup>®</sup>-auxiliaries as well as further Dentaaurum articles that are not included in the tomas<sup>®</sup>-auxiliary kit (see [www.dentaaurum.com](http://www.dentaaurum.com)), various orthodontic appliances can be fabricated, e.g. for skeletal anchored distalization, mesialization, intrusion.

### **3. Product overview and indications**

#### **3.1 General information**

tomas<sup>®</sup>-auxiliaries are designed to fabricate different skeletal anchoring appliances. The following product overview lists some of the possible indications.

- Direct coupling means that the orthodontic appliance is directly connected to the tomas®-pin via the tomas®-auxiliary. The tomas®-pin serves as counter bearing for the force exerted by the appliance.
- Indirect coupling means that the orthodontic appliance is fixated on the teeth, just as in the conventional techniques. The tooth or the group of teeth that serve as counter bearing are coupled with the tomas®-pin using a sufficiently dimensioned wire.

### 3.2 Product overview

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>tomas®-coil spring,</b> light (8 mm) Code blue (nickel titanium tension spring)	<b>302-012-00</b>	10 pieces	✓		Tension spring for direct coupling between the tomas®-pin and a multi-bracket appliance, e.g. for distalization, en masse retraction and mesialization.
<b>tomas®-coil spring,</b> medium (8 mm) Code yellow (nickel titanium tension spring)	<b>302-012-10</b>	10 pieces	✓		Tension spring for direct coupling between the tomas®-pin and a multi-bracket appliance, e.g. for distalization, en masse retraction and mesialization.
<b>tomas®-coil spring,</b> heavy (8 mm) Code red (nickel titanium tension spring)	<b>302-012-20</b>	10 pieces	✓		Tension spring for direct coupling between the tomas®-pin and a multi-bracket appliance, e.g. for distalization, en masse retraction and mesialization.

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>Super Elasto-Force</b> (plastic chain)	<b>774-221-01</b>	1 piece	–		Elastic chain for direct coupling between the tomas®-pin and a multi-bracket appliance, e.g. for distalization, en masse retraction and mesialization.
<b>tomas®-uprighting spring</b>	<b>302-009-00</b>	10 pieces	✓		Spring for molar uprighting with and without intrusion or extrusion. For details, please refer to the instructions for use of the tomas®-uprighting spring (REF 989-651-00).
<b>rematitan® LITE compression spring</b>	<b>758-361-00</b>	3 pieces	✓		Compression spring for horizontal tooth movement (mesialization, distalization).
<b>tomas®-stop screw</b> (stop screw, without hexagon socket key)	<b>302-013-01</b>	10 pieces	✓		The stop screw is fixed to the arch and can be used to repeatedly put a compression spring under tension without the arch having to be removed. The hexagon socket screw is secured against unscrewing.

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>Hexagon socket key 0.9</b>	<b>607-129-00</b>	1 piece	✓		Key for activating the tomas <sup>®</sup> -stop screw and the stop screws for amda <sup>®</sup> (advanced molar distalization appliance).
<b>tomas<sup>®</sup>-slotted stops 18</b> (stop tube, slotted, for the slot 18 technique)	<b>302-021-18</b>	100 pieces	–		As stop for compression spring in the slot 18 technique.
<b>tomas<sup>®</sup>-slotted stops 22</b> (stop tube, slotted, for the slot 22 technique)	<b>302-021-22</b>	100 pieces	–		As stop for compression spring in the slot 22 technique.
<b>tomas<sup>®</sup>-hook</b> (wire hook)	<b>302-009-10</b>	10 pieces	–		If the tomas <sup>®</sup> -pin SD is initially meant for other anchorage purposes and should later be used to hang the compression spring, use the tomas <sup>®</sup> -hook (see also the Instructions for use REF 989-656-00). If it is clear from the outset that only compression springs will be used for the treatment, a tomas <sup>®</sup> -pin EP should be inserted immediately.

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>tomas®-T-wire</b>	<b>302-024-00</b>	10 pieces	✓		Multifunctional rectangular wire, e.g. for fabricating an indirect anchorage, individual hook, etc.
<b>tomas®-cross tube 18</b> (cross tube, slot 18 technique)	<b>302-014-18</b>	10 pieces	–		For direct connection between the tomas®-pin SD and the main arch for the slot 18 technique.
<b>tomas®-cross tube 22</b> (cross tube, slot 22 technique)	<b>302-014-22</b>	10 pieces	–		For direct connection between the tomas®-pin SD and the main arch for the 22 technique.
<b>tomas®-power arm, square</b> (crimping tube, with wire, square)	<b>302-015-00</b>	10 pieces	✓		Torsion-resistant rectangular wire for the fabrication of customized hooks, e.g. for bringing the point of force application of a compression spring near the center of resistance. Coupling to the tomas®-pin SD is performed directly, e.g. for distalization, en masse retraction and mesialization. Indirect coupling between the tomas®-pin SD and the main arch.

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>tomas®-power arm, round</b> (crimping tube, with wire, round)	<b>302-019-00</b>	10 pieces	✓		Round wire for the fabrication of customized hooks, e.g. for bringing the point of force application of a compression spring near the center of resistance. Coupling to the tomas®-pin SD is performed directly, e.g. for distalization, en masse retraction and mesialization.
<b>tomas®-crimp hook</b> Upper left/lower right	<b>302-020-00</b>	10 pieces	✓		Prefabricated hook, can be shortened, e.g. for varying the point of force application of a compression spring in relation to the center of resistance. Coupling to the tomas®-pin SD is performed directly, e.g. for distalization, en masse retraction and mesialization.
<b>tomas®-crimp hook</b> Upper right/lower left	<b>302-020-10</b>	10 pieces	✓		Prefabricated hook, can be shortened, e.g. for varying the point of force application of a compression spring in relation to the center of resistance. Coupling to the tomas®-pin SD is performed directly, e.g. for distalization, en masse retraction and mesialization.

Designation	REF	Quantity	Included in the auxiliary kit	Figure	Indication
<b>tomas®-aligner hook</b> Upper left/lower right (adhesive hook)	<b>302-021-00</b>	10 pieces	✓		Hook to be bonded on teeth or Aligner for direct coupling with the tomas®-pin EP. For fixing the hook on the acrylic of the Aligner, special adhesive is required (Bond Aligner™ from Reliance).
<b>tomas®-aligner hook</b> Upper right/lower left (adhesive hook)	<b>302-021-10</b>	10 pieces	✓		Hook to be bonded on teeth or Aligner for direct coupling with the tomas®-pin EP. For fixing the hook on the acrylic of the Aligner, special adhesive is required (Bond Aligner™ from Reliance).

Find further information on the Dentaaurum website ([www.dentaaurum.com](http://www.dentaaurum.com)).

#### 4. Information about the composition

Information about the composition is included in the materials list, see catalog or [www.dentaaurum.com](http://www.dentaaurum.com).

#### 5. Safety information

The product should not be used if there is a known allergy to one or more of the ingredients.

## 6. Information for single use products

The tomas®-auxiliaries are designed for single use only. Reconditioning of products that have been inserted previously or reuse on patients is not permitted.

## 7. Additional tips

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

## 8. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

## 9. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of the symbols used on the label REF 989-313-00).

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