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Instructions for use FKO-Split-Post Fixator

Dear Customer

Thank you for choosing a quality product from Dentaforum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaforum.com, even though you frequently use the same product.

1. Manufacturer

Dentaforum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. Description

The FKO-Split-Post Fixator (Fig. 1) consists of:

- o upper and lower fixation arms for the plaster models
- o bite lock with fixation screw
- o two round, parallel telescopes

The FKO-Split-Post Fixator is designed for precise occlusal positioning and holding and has a vertically adjustable upper fixation arm. The two round telescopes prevent rotation in the horizontal axis. The conical fixation arms with retention grooves allow for removal and exact return of the plaster models. The central screws enable the models to be held securely in position. The movable bite lock secures the vertical occlusal position given by the bite registration.

3. How to use the FKO-Split-Post Fixator

1. The central screws on the fixation arms must be screwed right the way in.
2. The plaster models should be fixed with the aid of a bite registration. Apply articulating plaster to the base of the upper and lower models and place midway between the ends of the central screws.
3. Complete the model articulation in the FKO-Split-Post Fixator and ensure that the plaster on the upper model is flush with the upper fixation arm (Fig. 2).
4. Before it finally sets, remove any excess plaster from the upper surface of the fixator arm and from the screw slits.
5. After having fixed with plaster, move the bite lock all the way up to the upper fixator arm and screw in tightly (Fig. 3).

4. Removing the models

1. Separate the upper fixator arm from the lower.
2. Unscrew and remove the central screws using a screwdriver.
3. Remove the models with their plaster bases from the fixator arms, taking care not to tilt to one side or the other.
4. When replacing the models, check that the fixator arms are clean (free of plaster and wax residues). If possible, clean the fixator arms and model base using a steam cleaner.
5. Replace the models onto the fixator arms carefully without tilting.
6. Using a screwdriver, turn the central screws all the way back inside and subsequently secure the models.

5. Tips

- o Seal the slits in the central screws with wax to avoid plaster from entering.
- o Spray the FKO-Split-Post Fixator with a thin layer of silicon spray to allow easy removal of the models later.
- o Any excess plaster on the sides of the upper and lower model can be easily removed using the plaster trimmer.
- o Placing the plaster models together with their bases into the archives saves time later. Repeating articulation is unnecessary should FKO appliance repairs be required. Simply slide the old models back onto the FKO-Split Post Fixator arms and secure with the central screws.

Caution: never strike the FKO-Split-Post Fixator with a hammer or similar instrument in order to remove the models and plaster bases. This will distort the FKO-Split-Post Fixator and damage its function.

6. Availability

FKO-Split-Post Fixator 1 piece REF 072-004-00

7. Quality information

Dentaforum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

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Dentaforum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Allemagne