D
DENTAURUM
1886

tiologic[®]
TWINFIT

tiologic[®]ST

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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

Prosthetic abutments and components replace missing teeth, restore chewing function and thus improve the quality of life. The implants of the tioLogic® implant family are used to fixate corresponding abutment components, depending on the indication. Prosthetic abutments and components serve to produce prosthetic meso-constructions, which are then restored with a prosthetic superstructure.

Abutment components						
Series of abutments*	S		M		L	
Implant diameters in mm	3.3	3.7	4.2	4.8	5.5	

*all of the abutment components listed in these Instructions for use are available in the S, M and L series of abutments.

tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium bases, CEREC, Sirona serve as an adhesive base for customized, implant-supported dental restorations or for the production of two-part abutments. Digital recording, planning, and implementation of these abutments is performed exclusively using the Sirona software and the associated scan bodies.

The CAD/CAM titanium bases, CEREC, Sirona are available for the tioLogic® ST and the tioLogic® TWINFIT implant systems. The tioLogic® ST and the tioLogic® TWINFIT CAD/CAM titanium bases, CEREC, Sirona are inscribed with S, M or L corresponding to the series of abutments. In addition, the tioLogic® TWINFIT titanium bases, CEREC, Sirona are available with a conical or platform connector geometry.

Titanium base, CEREC, Sirona			
Implant system	tioLogic® ST		
Series of abutments	S	M	L
GH in mm	0.5		
Implant system	tioLogic® TWINFIT		
Series of abutments	S	M	L
Conical, GH in mm	1.5		
Platform, GH in mm	0.5		

Abutments are intended to connect the implants with the prosthetic restoration.

3. Intended purpose

Abutments are intended to connect the implants with the prosthetic restoration.

4. Intended users

The tioLogic® ST and tioLogic® TWINFIT product ranges are supplied exclusively to qualified personnel such as surgeons, dentists and dental technicians. It should only be used by qualified personnel who are familiar with dental implantology, including diagnosis, preoperative planning, surgical technique and prosthetic treatment.

5. Medical indications / contraindications

5.1 Indications

tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium bases, CEREC, Sirona

- Abutments or directly screw-retained crowns are intended for fixation on implants in the upper or lower jaw

5.2 Contraindications

- In patients with a known intolerance to individual components.
- The contact surface between the implant and abutment must not be blasted or otherwise mechanically processed
- Insufficient oral hygiene
- Bruxism
- Insufficient space

tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium bases, CEREC, Sirona

- Primary bracing of titanium bases
- Correction of angulation exceeding 20°, or exceeding 15° in the case of implants \varnothing 3.3 mm
- Cast-on technique
- Direct laser welding
- Processing (shortening, grinding, etc.) of the adhesive bond
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Single crowns with a cantilever bridge unit
- Abutment on implant \varnothing 3.3 mm only for replacement in the area of the upper and lower posterior teeth

6. Patient target group

All skeletally mature patients undergoing implant treatment. Please consider the indications / contraindications.

7. Information on use, processing, fitting and insertion

Prior to use, dental professionals should ensure that they have carefully read and understood the full tioLogic® ST and tioLogic® TWINFIT Instructions for use.

We strongly recommend that, prior to using Dentaurum products, dental professionals attend a Dentaurum training course to familiarize themselves with the Dentaurum products.

Carry out thorough radiological, psychological and physical examinations before implantation and inform patients of the process in detail. Dentaurum recommends full clinical, radiological, photographic and statistical documentation.

The tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium base, CEREC, Sirona can be documented, e.g. in the patient file or PatientPass, using the additional labels.

Precautions:

Before surgical implantation, it is important to carry out a careful patient anamnesis and, if required, a third-party anamnesis by the patient's doctor (GP/PCP) in order to clarify if 1) implantation may be difficult due to an anatomical situation, 2) there is the risk of a serious surgical problem or a general risk, 3) wound healing/osseointegration may be difficult, or 4) good hygiene and care of the implant, abutment and restoration may worsen. Do not fit dental implants if any of the above cases or a combination hereof is severe or uncontrollable.

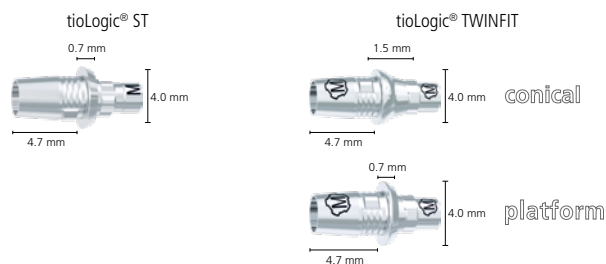
Prosthetic restoration of the implant must not be performed until the soft tissue has healed without irritation. Furthermore, a radiographic check-up should be performed before the prosthetic restoration.

Important:

The implantologist should ensure the products cannot be aspirated during intra-oral use.

tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium bases, CEREC, Sirona

The geometry of the tioLogic® ST and tioLogic® TWINFIT CAD/CAM titanium base, CEREC, Sirona is designed to ensure a reliable, aesthetic bond with the ceramic mesostructure. CAD/CAM titanium bases, CEREC, Sirona are available in all the three prosthetic abutment lines S, M, and L and, within the tioLogic® TWINFIT implant system, and additionally with conical and platform connector geometries.

The following scanners and software products can be used for the production process:

Scanner	inEos Blue
	inEos X5
	CEREC 3
	Bluacam
	Omniscam
Software	Primescan
	CEREC SW
	inLab SW
	Connect SW

The following products may be combined for scanning and bonding with mesostructures.

Overview of compatible scan bodies:

Implant types	Implant- \varnothing in mm	CAD/CAM titanium base, CEREC, Sirona by Dentaureum	TiBase Scanbody by Sirona
tioLogic® ST	3.3	386-195-00	Scanbody for Bluescan, S Scanbody for Omniscan, S
	3.7	386-295-00	
	4.2		
	4.8	386-395-00	Scanbody for Bluescan, L Scanbody for Omniscan, L
	5.5		
tioLogic® TWINFIT	3.3	346-197-01 346-197-02	Scanbody for Bluescan, S Scanbody for Omniscan, S
	3.7	346-297-01	
	4.2	346-297-02	
	4.8	346-397-01	Scanbody for Bluescan, L Scanbody for Omniscan, L
	5.5	346-397-02	

7.1 Scanning

- Place the CAD/CAM titanium base, CEREC, Sirona on the corresponding laboratory analog in the master model or on the implant in the patient's mouth and screw in place using the included AnoTite screw. The direction of the small cam on the top of the titanium base is inconsequential here.
- Place a suitable scan body (see section "Overview of compatible scan bodies") on the CAD/CAM titanium base, CEREC, Sirona until it fits without gaps and ensure that the intended guide groove in the scan body is guided over the small cam on the CAD/CAM titanium base, CEREC, Sirona. The scan body can be scanned without powder or scan spray.
- Record the situation using a scanner from the CEREC or inLab CAD/CAM system.

CAUTION – intra-oral application

Only use the CAD/CAM titanium base, CEREC, Sirona for scanning in the mouth if the correct fit of the scan body can be reliably verified. This is hardly feasible with deeply seated implants.

- Use the inLab SW, Connect SW, or CEREC SW software to design the individual shape of the mesostructure, taking the following information into account:

As the tioLogic® implant system is not currently integrated into the Sirona CEREC® software, the camlog® implant system can be selected as an alternative for processing in the software.

Please observe the following in this case:

Check the correct position of the construction before the grinding process. If you choose a camlog® implant system with three positioning options, a correction may be necessary. If this is not observed, the mesostructure may possibly be positioned incorrectly. Correct the position with a corresponding recess in the corrected position in the unsintered mesostructure. Mill the shape from one of the materials approved for this purpose (e.g. inCoris ZI meso or CEREC Zirconia meso). To this end, it is essential to follow the instructions for design, finishing, and bonding with the CAD/CAM titanium base, CEREC, Sirona in the Instructions for use/processing instructions for the corresponding material.

7.2 Processing the CAD/CAM titanium base, CEREC, Sirona

Reducing the diameter of the CAD/CAM titanium base, CEREC, Sirona, e.g. by grinding, is not permitted. The CAD/CAM titanium base, CEREC, Sirona must not be shortened. The contact surfaces between the CAD/CAM titanium base, CEREC, Sirona and the implant must not be blasted or processed in any other way!

Only the parts of the CAD/CAM titanium base, CEREC, Sirona surfaces that are to be bonded to a mesostructure should be blasted (50 μ m aluminum oxide, max. 2.0 bar) and then cleaned (with alcohol or steam). Use PANAVIA™ F 2.0 extra-orally as an adhesive

for bonding CAD/CAM titanium base, CEREC, Sirona and sintered mesostructures made of inCoris ZI meso or CEREC Zirconia meso.

Other adhesives may be required for bonding other materials. Observe the Instructions for use for the material used.

- Screw the CAD/CAM titanium base, CEREC, Sirona into a laboratory implant or polishing aid to facilitate handling during bonding.
- Cover the hex head of the AnoTite screw with wax.
- Blast the bonding surfaces of the CAD/CAM titanium base, CEREC, Sirona with 50 μ m aluminum oxide at max. 2.0 bar and clean the surfaces with alcohol or steam.
- Ensure that the mesostructure can be pushed completely onto the CAD/CAM titanium base, CEREC, Sirona.
- Apply metal primer and adhesive to the CAD/CAM titanium base, CEREC, Sirona in accordance with the manufacturer's instructions.
- Slide the sintered mesostructure onto the base as far as it will go. Make sure it latches into the rotation and position stop.
- Remove excess adhesive immediately. Ensure that no adhesive residues remain in the screw aperture!
- Follow the adhesive manufacturer's recommendations for final curing of the adhesive.
- After curing, remove the residue adhesive with a rubber polisher.

Tightening torque	
Tightening torque	15 Ncm
Prosthetic screw on the model	30 Ncm
AnoTite screw intra-orally	30 Ncm

To achieve optimum preloading of the AnoTite screw, the AnoTite screw must be retightened with the same torque approx. 5 minutes after insertion.

7.3 Additional components

The prosthetic screw (not included in scope of delivery) serves only to help the dental technician fix the abutment on the working cast temporarily.

A new AnoTite screw should always be used for the final fitting and fixation of the abutment in the mouth at the prescribed torque.

Torque ratchet
Hex key
Adapter, ISO shank, hexagon/ratchet
Tightening torque
Prosthetic screw on the model

Caution:

Do not confuse the prosthetic screw with the AnoTite screw. The prosthetic screw is colored silver and the AnoTite screw is anodized gold.

8. Technical specification

Composition

Product	Composition
CAD/CAM titanium base, CEREC, Sirona	3.7165
AnoTite screw	3.7165

Information about the composition is included in the material list for our implantology products (REF 989-801-06), or see www.dentaureum.com/ifu. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaureum.com.

9. Adverse reactions and residual risks

Possible complications and risks during prosthetic treatment (temporary adverse reactions) are:

Pain, swellings, difficulties speaking, inflammation of the gums.

Possible complications and risks during prosthetic treatment (permanent adverse reactions) are:

fractured dentures, aesthetic problems.

10. Safety information and precautions

- Do not use the product if there is a known allergic reaction to one or more of the material components.
- Due to the small size, the article could be swallowed or aspirated. Aspiration could lead to difficulty in breathing or death due to asphyxiation. Therefore, secure all articles used intra-orally against swallowing and/or aspiration.
- Different types of alloy in the oral cavity can lead to galvanic reactions.

- If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.
- All tioLogic® ST prosthetic components may only be used in combination with tioLogic® ST implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaureum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- All tioLogic® TWINFIT prosthetic components may only be used in combination with tioLogic® TWINFIT implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaureum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- Observe the indications for the individual products. Consider all indications that are not listed explicitly to be contraindications.
- Disinfection, cleaning and sterilizing should not be done in the original packaging.
- Products must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.
- Technical advice (oral and written) is based on the scientific and technical knowledge available at the time the product is put on the market. It does not release the dental professional from the responsibility of personally checking the suitability of the products for the intended indications and applications. Advice is only given as a non-binding recommendation and cannot be assumed to provide any form of assurance, liability or guarantee.

Severe incidents:

Report all severe incidents arising from the use of the product to the manufacturer and the competent authority in the country in which the dental professional and/or the patient are resident.

Severe incidents are identified as malfunctions and a drop in performance or deterioration of the properties of a medical device which resulted or may have resulted in a direct or indirect temporary or permanent deterioration in a patient's state of health, in the death of a patient, a dental professional or other persons or as posing a serious danger to public health.

11. Storage and shelf life

Store the tioLogic® TWINFIT implants, abutments and components always in a dry, dark place at room temperature in the original packaging. If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

12. Disposal

Adhere strictly to the rules that apply for your region and the details outlined in the safety data sheets, if available. Products to be disposed of should be treated as surgical waste and be decontaminated in compliance with the relevant regulations for medical waste disposal.

13. Information for single use products

Prosthetic abutments and components are supplied non-sterile and are intended for single use on a single patient only. The product must be disinfected, cleaned and sterilized before first use.

Reconditioning of abutments and components (recycling) as well as reuse on patients are not permitted as safe reconditioning and/or functional safety cannot be guaranteed and may lead to a loss in mechanical, chemical and/or biological properties. Reuse may cause infection. Do not use implants after the expiry date stated on the label.

14. Information on disinfection / cleaning / sterilizing

The tioLogic® ST and tioLogic® TWINFIT prosthetic abutments and components are supplied non-sterile. They are for single use on one patient only, unless explicitly intended for multiple use. They must be cleaned and disinfected before and after being used on a patient (e.g. transfer to dental laboratory). Dentaureum generally recommends that the product be sterilized in addition, as the use of non-sterile components may cause infections. Only sterilized prosthetic components may be used if bleeding occurs.

15. Product overview and scope of delivery

Please refer to the tioLogic® ST product catalog for the tioLogic® ST product range.

Please refer to the tioLogic® TWINFIT product catalog for the tioLogic® TWINFIT product range

16. Quality information

Dentaureum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaureum has no influence on the processing on site.

17. Explanation of symbols

Please refer to the label. See chapter "Additional information".

18. Trademarks and copyright

CEREC® is a registered trademark from Sirona.
camlog® is a registered trademark of CAMLOG.

19. Information on MRT safety

The products within these Instructions for use are MRT safe.

20. Additional information

Additional information can be found at www.dentaureum.com/ifu

Instructions for use	REF
Screw	989-804-98
tioLogic® ST Additional components	989-805-87
tioLogic® TWINFIT Additional components	989-805-88
Torque ratchet	989-719-00
Preparation instructions for the implant system components	989-801-07
Explanation of symbols used on the label	989-313-00
MRI safety of Dentaureum products	989-804-43

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