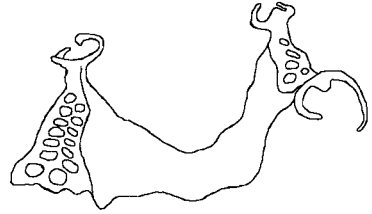
Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

---

rema<sup>®</sup>  
**blue** 

Phosphatgebundene Präzisionseinbettmasse für Dublier- und CAD/Vest Verfahren  
Phosphate bonded precision investment material for duplicating procedures and CAD/Vest  
Revêtement de précision à liant phosphate pour la duplication et le procédé CAD/Vest  
Revestimiento de precisión ligado con fosfato para procesos de duplicado y colado CAD/Vest  
Rivestimento di precisione a legante fosfatico per tecnica di duplicazione e CAD/Vest

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## Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at [www.dentaurum.com](http://www.dentaurum.com), even though you frequently use the same product.

## Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

## Safety information

**When using the speed heating method, ensure that the furnace door remains closed continuously for at least 15 min after the casting ring has been inserted (Danger of burning due to flames escaping).**

**Investments contain quartz. Avoid inhalation of dust as this may cause lung diseases (silicosis / lung cancer).**

**Wearing of face mask type FFP 2 – EN 149:2001 is recommended.**

**Open pouch with scissors and avoid dust development when pouring into mixing bowl.**

**Rinse empty pouch with water before disposal.**

**Remove dust from working area only with a wet cloth.**

**To avoid dust when devesting place ring into water until it is completely soaked after ring has reached room temperature.**

**When sandblasting use suction with fine dust filter.**

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EN

## Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

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# Dentaurum Group

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA  
and in more than 130 countries worldwide.

DENTAURUM  
QUALITY  
WORLDWIDE  
UNIQUE

- ➔ Informationen zu Produkten und Serviceleistungen finden Sie unter [www.dentaurum.com](http://www.dentaurum.com)
- ➔ For more information on our products and services, please visit [www.dentaurum.com](http://www.dentaurum.com)
- ➔ Vous trouverez toutes les informations sur nos produits et services sur [www.dentaurum.com](http://www.dentaurum.com)
- ➔ Descubra nuestros productos y servicios en [www.dentaurum.com](http://www.dentaurum.com)
- ➔ Informazioni su prodotti e servizi sono disponibili nel sito [www.dentaurum.com](http://www.dentaurum.com)

Stand der Information | Date of information | Mise à jour |  
Fecha de la información | Data dell'informazione: 01/17

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications |  
Reservado el derecho de modificación | Con riserva di apportare modifiche

**D**  
DENTAURUM