StarWax Blank

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StarWax Blank

Dear customer,

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the Internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General description

The wax blanks are made of high-quality synthetic wax, guaranteed to burn-out completely.

3. Application

StarWax Blanks are not suitable for use in the mouth!

For the manufacture of crowns, bridges and cast partial dentures for further processing by means of casting or pressing.

3.1 Suitable milling machines

All milling machines that are suitable for the processing of wax with holders for standard blank dimensions of ø 98.5 mm with collar. **Use cutters that are suitable for wax.**

3.2 Processing

- Prepare and mill according to the milling machine manufacturer's instructions.
- Remove objects from wax blanks carefully.
- Attach sprues to objects (cast/press) as for the conventional technique.
- Investment and casting/pressing according to the instructions for use from the manufacturer of the investment material or the alloy or the ceramics.

3.3 Recommendation for investment material:

- Use rema® dynamic S or rema® exakt F for cast partial dentures
- Use ceraMotion® press invest for press ceramics
- Use Castorit all speed for crowns and bridges

4. Technical data:

Composition: synthetic wax

Color: grey, blue

Dripping point: 100 - 130 °C / 212 - 266 °F

Density: 0.92- 0.97 g/cm³

Solubility in water: insoluble

5. Storage

Protect from damp and exposure to sunlight.

6. Disposal

StarWax Blanks are insoluble and can be disposed of with normal household garbage.

7. Availability

Blanks ø 98.5 mm with collar:

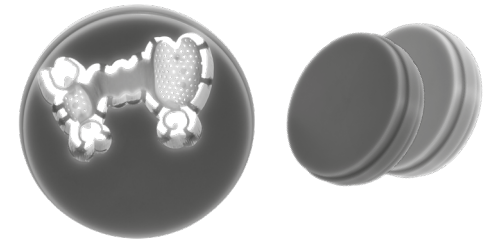
Blank	REF	Thickness	for
StarWax Blank grey	120-230-00	20 mm	crowns and bridges
StarWax Blank blue	120-235-00	25 mm	cast partial dentures

8. Quality information

Dentaurum ensures faultless quality of the products manufactured by us. These recommendations are based upon Dentaurum's own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken by Dentaurum as we have no influence on the processing on site.

9. Explanation of symbols used on the label

 Refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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