Estimado cliente

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EN

Dear customer

Thank you for choosing a quality product from Dentaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer
Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General product description/ area of application

Variety click/Variety click SP are special universal expansion screws. They have a patented screw locking feature (Fig. 1) which prevents unwanted reversing and can be used in the lower jaw to create space and control tipping of the anterior and posterior teeth as well as to open and close spaces in both the upper and the lower jaw. They can also be used as distraction screws, or for early palatal expansion of the upper jaw. The screws are welded or soldered to preformed bands or cap splints using the retention legs. The bands/cap splints are then usually cemented into place.

- Variety click (Fig. 1)**
- small dimensions with maximum expansion
 - solder-free
 - break-resistant retention legs and stable guidance throughout the entire expansion
 - a locking device prevents unwanted reversing
 - permanently laser marked arrow on the body of the screw indicates the opening direction and the maximum expansion
 - screw can be fixed with wire once desired expansion has been reached

Variety click SP (small palate, Fig. 2)

- same characteristics as the Variety click expansion screw
- especially suitable for very narrow jaws
- the retention legs can be bent close to the screw body

3. Fabricating an appliance with the Variety click/ Variety click SP expansion screw

Preformed bands or cast cap splints are fitted on the mandibular first premolars and the first molars, each left and right. The bands/cap splints are adjusted to fit on a hard plaster model. The legs of the Variety click/Variety click SP screw are then bent to fit the four bands/cap splints so that they can be soldered or welded into place (Fig. 3). The Variety click/Variety click SP screw should "float" freely 1-2 mm above the gingiva (Fig. 4). To avoid any leverage effect, ensure that the direction of force runs parallel to the occlusal plane.

4. Tips for working with Variety click/Variety click SP expansion screws

In order to bend the retention legs of the Variety click/Variety click SP screw gently and easily, we recommend using the universal bending key (REF 611-122-00, Fig. 5 + 9) without the use of additional pliers (lever principle). If this is not available, the retention legs must be stabilized by holding the expansion screw body with a pair of sturdy pliers, e.g. pliers Maxi (REF 013-522-00), in order to prevent the retention legs from breaking (Fig. 6).

When using the soldering technique please protect the body of the screw against excessive heat, e.g. with heat screen (REF 380-070-00).

5. Composition

Information about the composition is included in the material list, see catalog or www.dentaurum.com.

6. Safety information

Always use the blue safety key supplied to prevent swallowing!
The product should not be used if there is a known allergic reaction to one or more of the material components.

7. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

8. Availability

Description	REF	Quantity
Variety click Mini-8	602-836-10 602-836-30	1 piece 10 pieces
Variety click SP (small palate) Mini-8	602-838-10 602-838-30	1 piece 10 pieces
Variety click Maxi-12	602-837-10 602-837-30	1 piece 10 pieces
Variety click SP (small palate) Maxi-12	602-839-10 602-839-30	1 piece 10 pieces

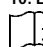
with safety key

Safety key	611-120-01	10 pieces
Universal bending key	611-122-00	1 piece

9. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon Dentaurum's own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

10. Explanation of symbols used on the label

 Refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

Information for the doctor/patient

Before the Variety click/Variety click SP appliance is cemented into position, open the spindle (screw) by making 2-3 quarter turns in the direction of the arrow to test the mobility. Then turn back to the original position.

Turn the Variety click/Variety click SP screw only in the direction of the arrow. Opening and closing the screw several times during treatment can cause it to loosen and no longer function properly.

The finished Variety click/Variety click SP appliance is cemented in place in the conventional manner. Follow the instructions from the cement manufacturer.

Following this, the screw is turned four times 90° using the key (REF 611-120-01, Fig. 8 + 10) supplied for this purpose. One full 360° turn of the spindle = 0.8 mm expansion.

The patient should turn the screw two times 90° both in the morning and evening (according to your instructions) until the desired movement is achieved.

Note: Always use the blue safety key (Fig. 10) supplied to avoid accidental swallowing.

The safety key is designed to take a maximum force of 35 N ± 3.5 kg. Please note that the force exerted on the safety key wire should always be exerted on the closed side of the wire clamp so that the clamp cannot loosen from the axis. Improper use may lead to overload and consequently to failure of the safety key thus potentially excluding any guarantee and liability.

Regular visits are recommended to monitor progress.

The Variety click/Variety click SP appliance is secured against unwanted reversing, after the desired degree of movement has been achieved, by fitting a ligature or by blocking it with acrylic. For stabilization purposes, the appliance should remain in the mouth for approximately 2-3 months. A retainer should be worn following removal of the appliance.

 The product should not be used if there is a known allergic reaction to one or more of the material components.

Information for the doctor/patient

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