DE

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Abbreviated instructions

Dear customer

Thank you for choosing a quality product from Dentaurum Implants. It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum Implants GmbH | Turnstr. 31 | 75228 Ispringen | Germany

2. General description of product, intended use and characteristics, technical description

The tiologic® TWINFIT implant system contains specially coordinated instruments, abutments and accessories for the fabrication of tiologic® TWINFIT implants. Only original components of the tiologic® TWINFIT implant system should be combined in accordance with the instructions for use / user manuals.

The abutment is selected by the dentist/dental technician to suit the individual patient's needs.

The diameter of the implant and the gingival thickness determine the choice of abutment.

Selection aids with conical and platform connector geometry for the abutment series S, M and L are available to help choose the right abutment on the model (gingival height and angulation).

3. Area of application

The 4Base abutments tiologic® TWINFIT are prosthetic components for bridge and bar restorations on tiologic® TWINFIT implants.

4. Medical indications/contraindications

4Base abutments tiologic® TWINFIT

Indications

- Anchorage of implant-supported, splinted bridges and bars in the maxilla in combination with at least 6 tiologic® TWINFIT implants and, in the mandible, in combination with at least 4 or more tiologic® TWINFIT implants.

Contraindications

- For biomechanical reasons we recommend that the following angulations are not exceeded:

4Base abutment with 0°: -20° - 20°

4Base abutment with 20°: 0° - 40°

4Base abutment with 30°: 10° - 50°

- Restorations with mixed retention (tooth/4Base)

- Combination with other anchorage elements

- Grinding and shortening 4Base abutments

5. Information on use, processing, initial implementation and handling

5.1 4Base abutments tiologic® TWINFIT

4Base abutments with conical and platform connector geometry are available for the S, M and L series of abutments. The 4Base abutments are provided in 3 angulations: 0°, 18° and 30°. The seating surface of the 4Base caps (ø 4.6 mm) and the cone (20°) of 4Base abutments are always identical for the S, M and L series of abutments, so that only one size of impression posts, closure caps etc. is required. The gingival heights of the 4Base abutments are 1.0 mm, 2.5 mm and 4.0 mm, that of the angulated 4Base abutments is 3.2 mm.

In case of steep inclination of the implants, splinting the denture is absolutely essential.

When fitting the angulated 4Base abutments, keep in mind that the abutments could have a larger diameter than the implants. As part of the abutments may be below the bone line, the bone may have to be removed in this region, if required, so that the abutment sits flush on the implant. A prerequisite for successful use of the 4Base system is the best possible accurate angular position of the implant. The more accurate this angle can be maintained, the easier the prosthetic treatment, as the abutments are then positioned parallel to one another. The mounted insertion guide facilitates the insertion of the angulated 4Base abutments. At the same time, they secure the pre-assembled AnoTite screw in the 4Base abutment. The insertion guide can be bent sideways to tighten the AnoTite screw.

The 4Base abutments should **NOT** be grounded or shortened.

When handling the 4Base abutments, ensure that the occlusal screw is not damaged by tweezers etc.

Every construction must be checked for passive fit prior to placing it intra-orally. (Sheffield test). This involves placing the construction on the 4Base abutment and fixing it to the 4Base abutment with only one prosthetic screw (not included in delivery). If this raises the construction to create a gap between the construction and 4Base abutments, stresses are present and must be eliminated.

5.2 4Base plastic cap tiologic® TWINFIT

The plastic cap, that burns residue-free, can be individually adapted to the space available. The processing instructions given by the the investment material manufacturers must be adhered to for investing and preheating. After casting, the casting ring must be cooled to room temperature to allow devestment and to achieve stress free casting objects. The cast object should never be blasted with abrasive agents since these can damage the inner configuration. Devestment is carried out either by steam cleaning or an ultrasonic bath.

5.3 4Base titanium caps tiologic® TWINFIT – lasering

The 4Base titanium cap is lasered to the prefabricated bar segments made of titanium (e.g. rematitan®, Dentaurum). The prefabricated titanium bars are shortened and fixed in the correct horizontal axis for laser-welding with a parallelometer / fixator (e.g. Paralas, Dentaurum). The bar components and titanium caps are then lasered together. The only filler material which can be used is pure titanium wire (e.g. Dentaurum). Should further questions arise on lasering and processing titanium, please call the Dentaurum Hotline at: + 49 72 31 / 803 -410.

5.4 4Base titanium cap tiologic® TWINFIT - bonding

The 4Base titanium cap is available for the passive fit technique. Once a cast bar has been produced, it is bonded intra-orally to the titanium bonding base. This helps compensate possible inaccuracies of fit that could create tensions.

The straight 4Base abutments (0°) are screwed into the tiologic® TWINFIT implant with the PentaGrip insertion key tiologic® TWINFIT. The angulated 4Base abutments (20° and 30°) have a rotationally secure fit in the tiologic® TWINFIT implants and are fixed with the AnoTite screw 9.0 mm. We recommend using a new AnoTite screw for the final fitting in the mouth.

To fix bridge and bar constructions on 4Base abutments, special prosthetic and AnoTite screws are available.

Screws with hexagon screw head geometry are tightened manually with the hex key ratchet SW 1.3 or mechanically with the hex key ISO shank SW 1.3.

Tightening torque

- 4Base abutments, straight on model: manually

- 4Base abutments, straight intra-orally: 35 Ncm

- Prothetic screw on model: manually

- AnoTite screw intra-orally: 30 Ncm

- Prothetic screw for 4Base caps on model: manually

- AnoTite screw for 4 Base caps intra-orally: 25 Ncm

- Sure-grip screw impression post on 4Base abutment intra-orally: 15 Ncm

- Closure screw on 4Base abutment intra-orally: 15 Ncm

Attention: Do not confuse the prosthetic screw with the AnoTite screw. The prosthetic screw serves only to help the dental technician fix the abutment on the working cast. The prosthetic screw is colored silver and the AnoTite screw is anodized gold.

6. Composition

- 4Base abutments: titanium alloy (Ti6Al4V)

- 4Base caps:

– 4Base plastic cap: polycarbote

– 4Base titanium cap, laser-weldable: titanium alloy (Ti6Al4V)

– 4Base titanium cap, adhesive technique: titanium alloy (Ti6Al4V)

– Closure screw: titanium alloy (Ti6Al4V)

– 4Base impression post, open: titanium allow (Ti6Al4V) / screw: 1.4305

– 4Base impression post, closed:

– impression post and screw: titanium alloy (Ti6Al4V) / impression cap: POM

– 4Base scan cap: PEEK (polyether ether ketone)

– 4Base insertion guide: PEEK (polyether ether ketone)

- 4Base laboratory implant: titanium alloy (Ti6Al4V)

- Prosthetic screws:

– AnoTite screws: titanium allow (Ti6Al4V)

– prosthetic screw: titanium alloy (Ti6Al4V)

– prosthetic screw for 4Base caps: titanium alloy (Ti6Al4V)

– AnoTite screw for 4 Base caps: titanium alloy (Ti6Al4V

7. Safety instructions

- The product should not be used, if there is a known allergic reaction to one or more of the material components.

- Different types of alloy in the mouth can lead to galvanic reactions.

- Only use the tiologic® TWINFIT prosthetic components in combination with tiologic® TWINFIT implants!

8. Storage and shelf life

Store in a dry place.

9. Information for single use products

4Base abutments and 4Base caps are designed for single use only. Reconditioning of 4Base abutments and 4Base caps that have been used previously (recycling) or reuse on patients is not permitted.

10. Information on disinfection / cleaning / sterilizing

tiologic® TWINFIT prosthetic components are delivered non-sterile. They are for single use on one patient only. They must be disinfected, cleaned and, where applicable, sterilized before and after being used on a patient.

11. Product overview / Scope of delivery, incl. REF

Please refer to the tiologic® TWINFIT product catalog for the product range for tiologic® TWINFIT.

12. Quality information

Dentaurum Implants ensures faultless quality of its products. These recommendations are based upon Dentaurum’s own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum Implants has no influence on the processing on site.

13. Explanation of symbols

 Refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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