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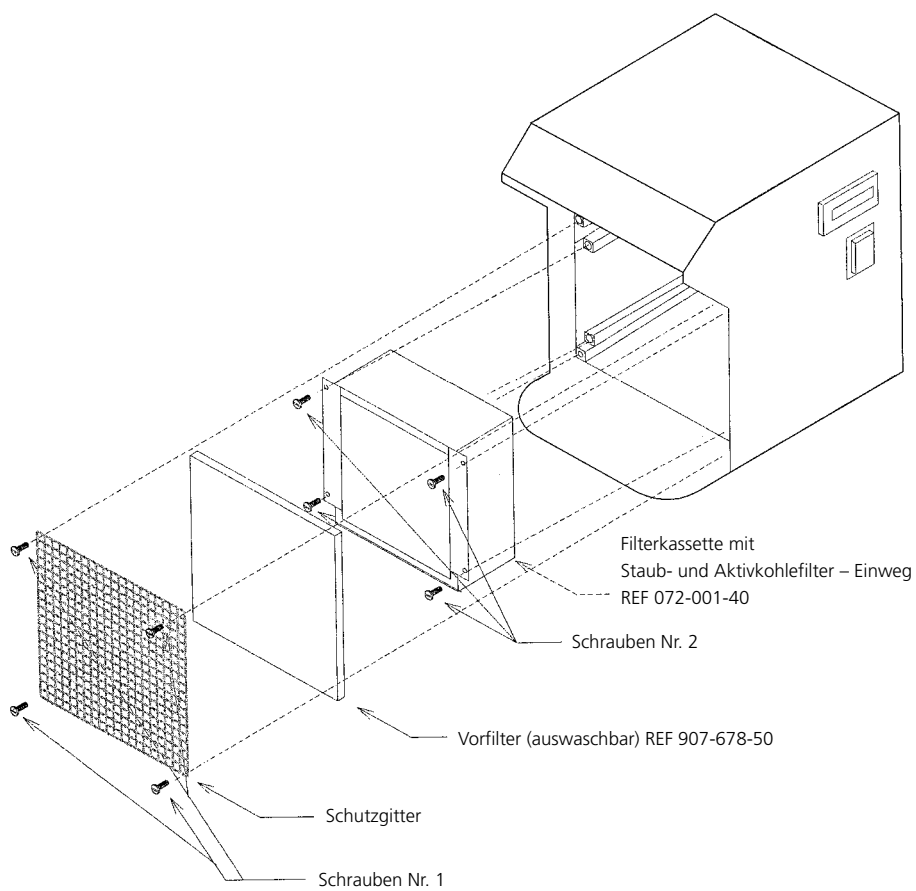
Dear customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. Safety information

Read through the instructions for installing and operating the unit carefully before switching it on.

2.1 Intended use

The suction unit is only intended for sucking out vapors when working with hazardous materials in the dental/orthodontic laboratory. The use of the unit for any other purpose is prohibited. The use of the unit for its correct purpose also includes compliance with all the instructions contained in this handbook.

2.2 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

This symbol gives important information on the correct use of the unit. Disregarding this information may cause the unit to malfunction.

3. General notes and disposal

When a Dentaureum product is finally no longer in use, the disposal regulations of that particular country apply. Dentaureum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

Note:

This symbol indicates that products with this marking should not be disposed of together with household garbage. The legislator does not allow commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaureum or the dental trade.

4. Description and usage

The suction unit helps to protect against vapors, when working with hazardous materials in the dental/orthodontic laboratory. A potentially harmful situation can be avoided from the start. The suction unit is user-friendly and helps prevent dangers arising from chemical substances.

Please note that optimal performance of suction can only be assured with a periodically maintained device. Especially the filter cartridge has to be replaced at regular intervals.

Also follow national regulations on compliance with emission limits of harmful substances, at the workplace.

5. Technical data

Mains connection	230 V	50/60 Hz	REF 072-001-00
	115 V	50/60 Hz	REF 072-002-00
Rated power	100 W		
Dimensions (W x H x D)	225 x 260 x 260 mm		

6. Commissioning of the suction unit

Should you notice any damage which may have occurred in transit, notify the dealer you got the unit from without delay.

⚠ Before using the unit, ensure that the power supply voltage corresponds to the operating voltage stated on the rating plate on the back of the unit.

Before you use the device for the first time, remove the protective foil on the plate. After connection to the mains (230 V) the unit is ready for operation. After turning the main switch the light is switched on automatically and the suction operation runs for about 5 sec.

The suction operation is started automatically during works in the working area via photoelectric switch. On completion of the working procedure the suction operation is stopped after about 5 sec. The operating hours counter is reset using the red button. After the suction unit has been switched on, the entire operating time is displayed for approx. 10 sec. Change the filter cartridge after 50 operating hours or once a year.

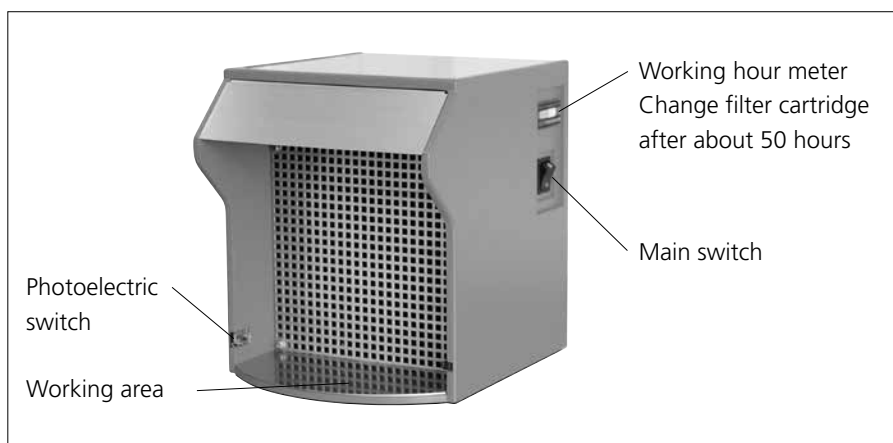
Fig. 1

⚠ If any item lies in the working area, it will be identified by the photoelectric switch pin; the unit will not be switched off automatically.

⚠ For safety reasons, it is not allowed to operate with an open flame in the working area (Fire hazard).

7. Care and maintenance

The unit is maintenance-free. If required, wipe case and working area with dry or moistened cloth. Do not use any aggressive detergents!

8. Filter change

1. Disconnect the unit from the mains.
2. After releasing the 4 screws (Fig. 3) the protective grating and the preliminary filter (Fig. 4) can be removed.

The preliminary filter should be cleaned depending on the level of contamination.

3. After releasing the 4 screws (Fig. 5) the filter cartridge can be changed.

The filter cartridge must be changed after about 50 hours of operation.

Fig. 2

Fig. 3

Fig. 4

Fig. 5

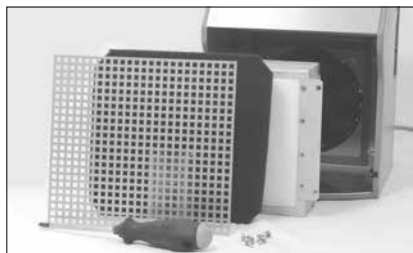
Fig. 6

9. Availability

Suction unit with charcoal filter

230 V **REF 072-001-00**

115 V **REF 072-002-00**

Filter cartridge – for single use **REF 072-001-40**

10. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

11. Explanation of symbols used on the label

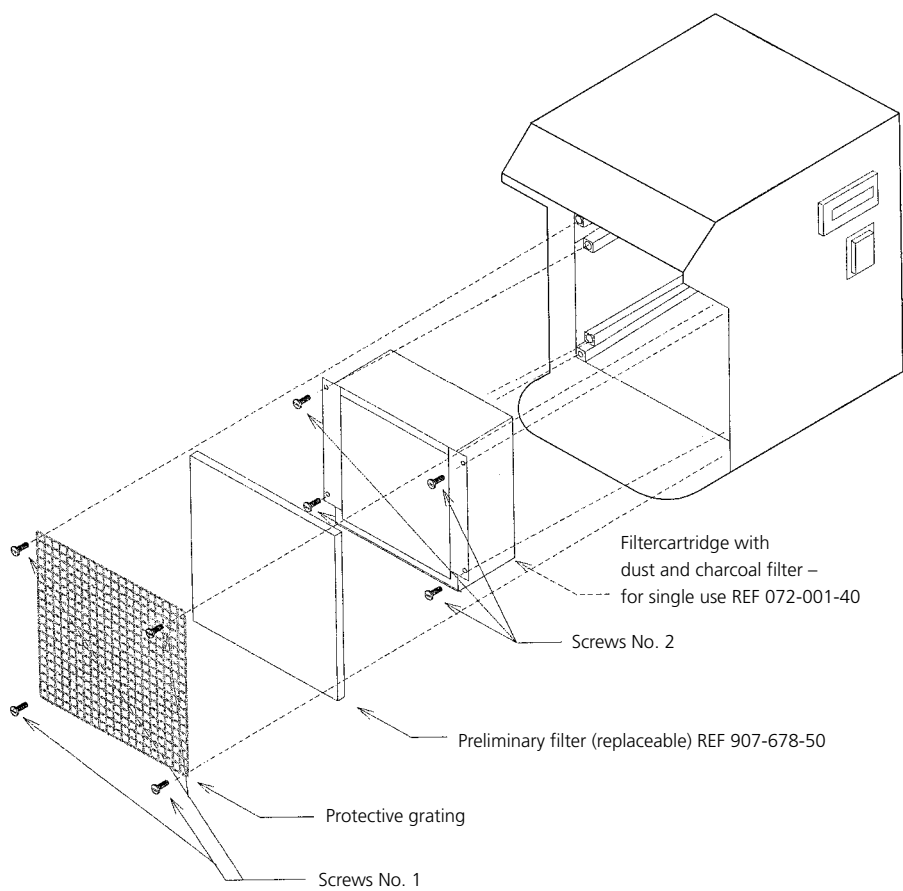
Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

12. CE-Declaration of conformity

We confirm through the CE marking on the rating plate compliance with the EMV Directives 89/336/EEC, Low-voltage Directive 73/23/EEC and CE Marking Mandatory Directive 93/68/EEC.

In the event of any modification of the product without our approval this declaration will become invalid and the warranty will extinct.

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