discovery®

D
DENTAURUM

- ➔ Informationen zu Produkten finden Sie unter www.dentaurum.com
- ➔ For more information on our products, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- ➔ Descubra nuestros productos en www.dentaurum.com
- ➔ Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 04/21

D
DENTAURUM

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31/803-0 | Fax +49 72 31/803-295
www.dentaurum.com | info@dentaurum.com

989-459-00 Printed by Dentaurum Germany 04/21/BR1

DE

discovery® Metallbrackets

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discovery® metal brackets

Dear customer,

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General description

discovery® brackets belong to the premium brackets in the Dentaurum metal bracket range and are the result of many years of experience. discovery® metal brackets are designed as a one-piece bracket and are manufactured on state-of-the-art machines using the MIM technique (Metal Injection Molding) with a special dental alloy. The brackets are optimally adapted to the tooth contour using CAD (Computer Aided Design) and are easy to handle. The laser-structured base ensures the best possible bonding and guarantees easy debonding.

3. Processing

3.1. Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding.

3.2. Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding. Follow the instructions from the bracket adhesive manufacturer for conditioning the teeth.

3.3. Selecting bracket adhesive

Our brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC lc (REF 163-110-00), ConTec Go! (REF 163-307-00) and SmartBond® (REF 163-506-00). Please follow the instructions of the bracket adhesive manufacturer and observe the table „Adhesives - areas of application“ in our Orthodontics Catalog.

3.4. Removal and handling

3.4.1 Removal from 1 case blister packaging

Remove the bracket blister from the packaging. The discovery® metal brackets are protected by two separate foils.

Remove the outer protective foil to remove the bracket from its blister packaging, grasp the bracket through the perforation in the foil above each bracket compartment using crimping tweezers (e.g. REF 025-277-00) mesially-distally.

3.4.2 Removal from 1, 10 and 50 case assortments

Grip the discovery® metal brackets mesially-distally or on the mesial or distal tie wings using crimping tweezers (e.g. REF 025-277-00) to remove them from the packaging.

3.4.3 Handling

The laser-structured base of the discovery® metal brackets should not be touched with your fingers or contaminated in any way, as this could otherwise have a detrimental effect on the bond.

3.5. Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. **This guarantees an optimum bond.**

3.6. Placing and aligning brackets

After applying the adhesive, place, align and press the discovery® metal bracket in the usual way onto the tooth. Place the discovery® metal bracket in the best possible position using the angulation aid on the crimping tweezers. Press the bracket onto the tooth so that excess adhesive is pushed out to the sides.

Avoid contact with the occluding teeth due to the risk of abrasion and/or premature bracket loss. If necessary, a raised bite can be used e.g. with Bite-Bumper™ (REF 705-000-00).

Remove completely any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

3.7. Ligating archwires

Once the adhesive has cured (curing time according to the adhesive manufacturer's instructions), ligating can be completed in the usual way using wire or plastic ligatures.

3.8. Bracket debonding

For the debonding of discovery® metal brackets, we recommend using Weingart pliers (e.g. REF 003-120-00). Remove ligation Grasp the bracket mesially-distally / diagonally by the wings, press the pliers together gently and disengage the base of the discovery® metal bracket carefully from the tooth using the debonding channel. Turn the bracket briefly to the left or right, without pulling, to remove it from the tooth. The angled bracket removing pliers (REF 004-347-00) are particularly suitable for the posterior region as they shear the bracket off between the bracket base and the adhesive.

4. Composition

The composition is included in the materials list; please refer to the catalog or www.dentaurum.com.

5. Safety instructions

There are no known risks or side effects (allergic reaction) for discovery® metal brackets. The product should not be used if there is a known allergic reaction to one or more of the material components.

6. Storage and shelf life

Store discovery® metal brackets in the closed packaging.

7. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted. Due to the mechanical stress during treatment and debonding, neither the dimensions nor the bonding strength can be guaranteed should the brackets be recycled.

8. Additional information

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the product catalog.
- There are no particular precautions to be taken when disposing of discovery® metal brackets after treatment.
- Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident is informed accordingly.

9. Delivery program

For the complete range of discovery® metal brackets, please see our catalog or www.dentaurum.com.

10. Quality

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

11. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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