Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

CE 0483

***topic***

**D**  
DENTAURUM

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**Dear customer,**

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at [www.dentaureum.com](http://www.dentaureum.com), even though you may frequently use the same product.

## **1. Manufacturer**

Dentaureum GmbH & Co. KG

Turnstr. 31 | 75228 Ispringen | Germany

## 2. General description

topic brackets are one of the premium brackets in the Dentaaurum metal bracket range and are the result of many years of experience in the field of brackets. topic brackets are nickel-free and are designed as a one-piece bracket. They are manufactured on state-of-the-art machines using the MIM technique (Metal Injection Molding) with a nickel-free dental alloy. The brackets are optimally adapted to the tooth contour using CAD (Computer Aided Design) and are easy to handle. The laser-structured base ensures the best possible bonding and guarantees easy debonding.

## 3. Processing

### 3.1 Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding.

### 3.2 Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid unexpected damage to the tooth during debonding. Follow the instructions from the bracket adhesive manufacturer for conditioning the teeth.

### 3.3 Selecting bracket adhesive

topic brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC lc (REF 163-111-00), ConTec Go! (REF 163-321-00) and SmartBond® (REF 163-506-00). Please follow the instructions of the bracket adhesive manufacturer.

### **3.4 Handling brackets**

Grip the topic brackets mesially-distally using crimping tweezers (e.g. REF 025-280-00) to remove them from the packaging and handle according to the following steps. The laser-structured base of the topic brackets should not be touched with your fingers or contaminated in any way, as this could otherwise have a detrimental effect on the bond.

### **3.5 Applying bracket adhesive**

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

### **3.6 Placing the brackets - direct bonding**

After applying the adhesive, place the topic bracket in the usual way onto the tooth. Place the topic bracket in the best possible position and press it onto the tooth so that excess adhesive is pushed out to the sides.

Avoid contact with antagonist teeth due to the risk of abrasion and/or premature bracket loss. If necessary, a palatal bite raise can be used.

Completely remove any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

### 3.7 Placing the brackets - indirect bonding

After applying the adhesive, place the transfer tray with the topic brackets as usual in the mandible and/or maxilla.

Completely remove any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

### 3.8 Ligating archwires

Once the adhesive has cured (observe adhesive hardening times in the manufacturer's instructions for use), ligate as usual with wire or plastic ligatures.

We recommend using rematitan® LITE or Tensic® archwires at the beginning of treatment.

### 3.9 Debonding brackets

The excellent bonding properties of topic brackets require special attention during debonding:

For the debonding of topic brackets, we recommend using Weingart pliers (e.g. REF 003-120-00). Grasp the bracket mesio-distally by the wings and carefully remove the topic bracket from the tooth by pressing the pliers together and bending the bracket base away from the tooth. If necessary, turn the bracket briefly to the right or the left, without pulling, to remove it from the tooth.

## 4. Composition

The composition is included in the materials list; please refer to the catalog or [www.dentaurum.com](http://www.dentaurum.com).

## 5. Safety information

There are no known risks or side effects (allergic reaction) for topic brackets. The product should not be used if there is a known allergic reaction to one or more of the material components.

topic brackets (CoCr alloy) must not be ground in the patient's mouth or without a suction apparatus to prevent the release of cobalt.

## 6. Storage and shelf life

Store topic brackets in the closed packaging.

## 7. Information for single use products

topic brackets are designed for single use only.

Reconditioning of topic brackets that have already been used (recycling) and their reuse on a patient is not permitted.

Due to the mechanical stress during treatment and debonding, neither the dimensions nor the bonding strength can be guaranteed should the brackets be recycled.

## 8. Additional information

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog.
- There are no particular precautions to be taken when disposing of topic brackets after treatment.
- Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident, are informed accordingly.

## 9. Availability

See our Orthodontics Catalog or visit [www.dentaurum.com](http://www.dentaurum.com) for information on the products available in the topic range.

## 10. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

## 11. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols on labels REF 989-313-00).

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