SUS3
SUS3
Sabbagh Universal Spring

CE 0483

Gebrauchsanweisung
Instructions for use
Mode d'emploi
Modo de empleo
Modalità d'uso

D
DENTAURUM

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Dear customer,

Thank you for choosing a quality product from Dentaureum.

Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit.

In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

The SUS³ – Sabbagh Universal Spring – is a telescope element with spring for universal intermaxillary use when treating class II. It provides a perfect alternative for patients with compliance issues. The SUS³ acts with constant, gentle forces on the temporomandibular joint, and can help to avoid the need for some extractions and dysgnathic surgery.

3. Intended purpose

For universal intermaxillary treatment of class II.

Fig. 1 SUS³ in position.

4. Intended users

All products are intended for dental professionals only.

5. Indications

- Unilateral and bilateral distoclusion/class II (dentoalveolar and skeletal).
- Molar distalization in the maxilla (headgear substitute).
- Dentoalveolar compensation of the occlusion (elastics substitute).
- Midline displacement.
- Molar mesialization in mandible and gap closure in cases of aplasia or loss of tooth (alternative to mini implants).
- Alternative to dysgnathia operation.
- Temporomandibular dysfunction and craniomandibular dysfunction (reposition effect).
- Therapy for snoring and sleep apnea.

6. Contraindications

The SUS appliance made of stainless steel must not be used for patients with a nickel allergy. Warnings are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients for whom a fixed appliance can be placed as part of orthodontic treatment.

8. Patient target group

SUS³ is supplied with a slightly preloaded inner spring. The preload is approx. 0.4 N. The complete spring travel of the SUS³ is 8.5 mm. At full compression, this represents a force of approx. 3.8 N (without SUS³ turbo spring) and a force of approx. 5.8 N (with SUS³ turbo spring).

8.1. Insert the SUS³ ball retainer clasp distally through the eyelet of the SUS³ telescope element (Fig. 2).

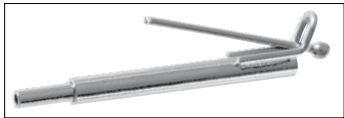
Fig. 2 Inserting the SUS³ ball retainer clasp into the eyelet of the SUS³ telescope element.

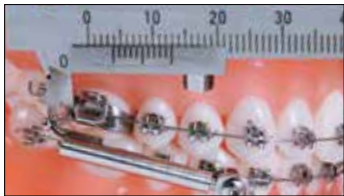
Fig. 3 SUS³ ball retainer clasp in position.

Fig. 4 SUS³ ball retainer clasp bent between the band and hook of buccal tube.

Fig. 5 SUS³ ball retainer clasp bent in buccal and caudal direction by 20 – 40 degrees.

- 8.2. Insert the SUS³ ball retainer clasp together with the SUS³ telescope element in the (headgear/facebow) tube from distal and bend in mesial direction. It is essential to maintain a distance of approx. 4 mm between the head of the ball retainer clasp and the distal end of the (headgear/facebow) tube (Fig. 3). This ensures that there is enough room for the SUS³.
- 8.3. Bend the SUS³ ball retainer clasp between the band and hook of the buccal tube, shorten it if necessary and secure in position using composite/acrylic (**important!**) (Fig. 4).
- 8.4. Bend the SUS³ ball retainer clasp at the distal end of the (headgear/facebow) tube by approx. 20° – 40° in the direction buccal and caudal and finely adjust. (Fig. 5).
- 8.5. Insert the SUS³ telescope rod into the SUS³ telescope element (guidance tube and middle telescope tube) which is fixed in position in the maxilla.

8.6. Insert the preassembled SUS³ arch adapter (closed side facing upwards) into the stainless steel archwire from above (minimum 0.43 mm x 0.56 mm/17 x 22 with slot 18 or 0.48 mm x 0.64 mm/19 x 25 with slot 22) between the lower canine and the first lower premolar (Fig. 6). The SUS³ arch adapter can be also fixed in position between the lower lateral incisor and lower canine if required, e.g. in the case of a missing premolar. A L-shaped bend is necessary to achieve this (Fig. 7), or the SUS³ ball retainer clasp with the telescope element can be fixed onto the upper second molar. Fully tighten the hexagon socket screw using the hexagon socket key (REF 607-144-00). The arch adapter should not come into contact with the bracket should only fix the arch in the arch adapter. Use a screw securing adhesive , e.g. NOBIL GRIP, if necessary.

9. Activating the SUS³

9.1. Spacer rings. Standard activation is completed using the open 1 mm and 2 mm spacer rings supplied (REF 607-103-11 and

Fig. 6 Inserting the SUS³ arch adapter on the stainless steel archwire.

Fig. 7 L bend.

Fig. 8 Additional activation with spacer ring.

Fig. 9 Additional activation with SUS³ turbo spring.

REF 607-104-11). Depending on requirements, one or several spacer rings can be clamped on the SUS³ telescope rod using special crimping pliers (REF 003-710-00) or Weingart universal pliers (REF 003-120-00) (Fig. 8).

- 9.2. SUS³ distance springs.** Alternatively, SUS³ distance springs (REF 607-152-00) can be shortened as required and placed on the SUS³ telescope rod.
- 9.3. SUS³ turbo spring.** The SUS³ turbo spring, with an additional force of approx. 2 N, is available if greater activation (spring force) is required. The turbo spring is placed on the SUS³ telescope element so that the three closed coils are pointing mesially towards the SUS³ telescope rod (Fig. 9).
- 9.4. SUS³ compression spring.** The SUS³ compression spring can replace/supplement the spring that is integrated in the SUS³ telescope element towards the end of the active treatment phase, if necessary

10. Tips and tricks

| Problem | Possible cause | Remedy |
|--|--|---|
| Patient bites on the metal or telescope element catches on the premolar brackets | SUS ³ ball retainer clasp not long enough distally, not bent or not adjusted | <ul style="list-style-type: none">- SUS³ ball retainer clasp should be approx. 4 mm long distally from the (headgear/facebow) tube and should be bent (see Fig. 4 and 5)- Adjust the SUS³ ball retainer clasp buccally or, if required, caudally |
| Patient cannot bite together properly/SUS ³ is too long | SUS ³ too long due to extraction or very narrow premolars, SUS ³ ball retainer clasp is not long enough distally | <ul style="list-style-type: none">- Place the SUS³ telescope rod/arch adapter between lower lateral incisor and lower canine instead of between the lower canine and the first premolar. L-shaped bend necessary- SUS³ ball retainer clasp should be approx. 4 - 5 mm long distally from the (headgear/facebow) tube and bent (see Fig. 4 and 5) |
| Middle telescope tube jams in the guidance tube (no spring function) | Calculus in the SUS ³ telescope element, patient with a heavy build-up of calculus, possibly broken spring | <ul style="list-style-type: none">- Compensate by using the SUS³ compression spring or SUS³ turbo spring- Replace the SUS³ telescope element (Vaseline in the guidance tube helps reduce deposit build-up) |
| The SUS ³ telescope rod slides out of the SUS ³ telescope element when the mouth is fully open | Patient's mouth opens wider than average, condylar hypermobility | <ul style="list-style-type: none">- Place the SUS³ arch adapter further distally- Explain to the patient how the parts can be reinserted- Physiotherapy/rotation exercises |

| | | |
|--|---|---|
| Lower anterior teeth are severely protruded | Too little anchorage and/or overactivation | <ul style="list-style-type: none"> - Insert a strong rectangular steel archwire, min. 0.43 mm x 0.56 mm/17 x 22 in slot 18 or 0.53 mm x 0.64 mm/21 x 25 in slot 22 with lingual crown torque, McLaughlin-Bennett-Trevisi* brackets or low torque brackets and bend it distally - SUPER Elasto-Force plastic chain (power chain) from 6 – 6 - temporary anchorage with tomas®-pin |
| SUS ³ telescope rod catches / rubs as mouth is closed | SUS ³ telescope rod has bent after uncontrolled mouth closure (e.g. after yawning) | Replace the SUS ³ telescope rod (can be ordered separately) |
| Fixing screw on the SUS ³ arch adapter comes loose | Fixing screw too loose or not tightened enough | Tighten the fixing screw and fix in position using a screw securing adhesive (e.g. NOBIL GRIP) |

* The Dentaurem version of this prescription is not claimed to be a duplication of any other, nor does Dentaurem imply that it is endorsed in any way by Drs. McLaughlin, Bennett and Trevisi.

11. Composition

SUS³ is made of the following materials: DIN 1.4305, DIN 1.4310, DIN 1.4301 and DIN 1.4568. The composition is included in the orthodontics materials list; please refer to the catalog or www.dentaurum.com. The SSCP is available at <https://ec.europa.eu/tools/eudamed>.

12. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

13. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

14. Further information:

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

15. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

16. Product range

For the complete product range of SUS³, please see our orthodontics catalog or www.dentaurum.com.

17. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

18. Explanation of symbols used on the label

Please refer to the label. The Instructions for use and additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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