
CE 0483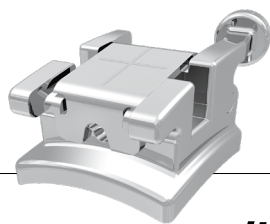

discovery® sl

Selbstligierende Brackets
Self-ligating brackets
Brackets autoligaturants
Brackets autoligables
Brackets autoleganti

D
DENTAURUM

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Descubra nuestros productos en www.dentaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour | Fecha de la información |
Data dell'informazione: 05/19

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications |
Reservado el derecho de modificación | Con riserva di apportare modifiche

D
DENTAURUM

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 7231/803-0 | Fax +49 7231/803-295
www.dentaurum.com | info@dentaurum.com

989-505-00 Printed by Dentaurum Germany 05/19/BR1-5

Fig. 1

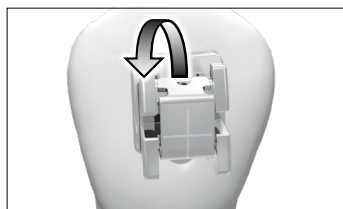
Fig. 2

Fig. 3

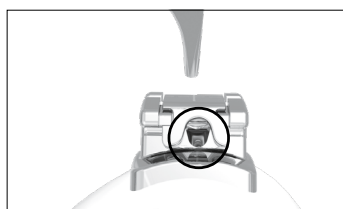
Fig. 4 a

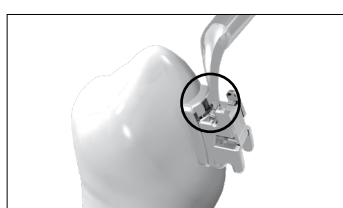
Fig. 4 b

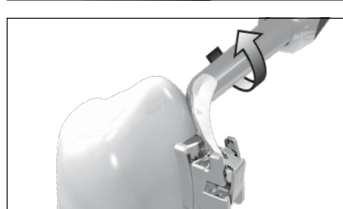
Fig. 5

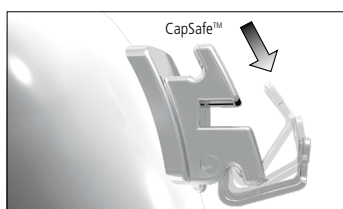
Fig. 6

Fig. 7

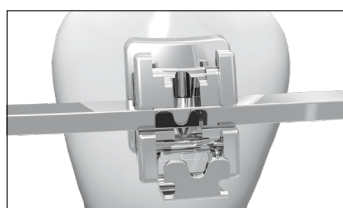
Fig. 8

Fig. 9

Fig. 10

DE

Selbstligierende Brackets – discovery® sl

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EN

Self-ligating brackets – discovery® sl

Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. Processing

2.1 Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding.

2.2 Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding.

Follow the instructions from the bracket adhesive manufacturer for conditioning the teeth.

2.3 Selecting bracket adhesive

Our brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC lc (REF 163-110-00), ConTec Go! (REF 163-307-00) and SmartBond® (REF 163-506-00). Please follow the instructions of the bracket adhesive manufacturer and observe the table „Adhesives – areas of application“ in our Orthodontics Catalog.

2.4 Removing and handling

Grip the discovery® sl brackets mesially-distally using clamping tweezers (e.g. REF 025-277-00) to remove them from the packaging and handle according to the following steps. The laser-structured base of the self-ligating brackets should not be touched with your fingers or contaminated in any way, as this could otherwise have a detrimental effect on the bond.

2.5 Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. **This guarantees an optimum bond.**

2.6 Placing and aligning brackets

After applying the adhesive, place the discovery® sl bracket in the usual way onto the tooth.

Laser-markings on the lid of the bracket can help orientation when placing the bracket (Fig. 1). The vertical line marks the middle of the bracket. The horizontal line is identical to the middle of the slot. **The slot is not in the middle of the bracket! This should be taken into account when aligning the bracket.**

Avoid contact with the occluding teeth due to the risk of abrasion and/or premature bracket loss. If necessary, a raised bite can be used e.g. with Bite-Bumper™ (REF 705-000-00). Ensure that no adhesive material enters the closing mechanism (bracket lid) on the gingival and occlusal side of the bracket, as this could restrict or destroy the functionality of the lid. Remove completely any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

2.7 Using the closure mechanism/ligating the archwires

The slot of the self-ligating bracket discovery® sl is closed with a lid. All lids, both in the maxilla and in the mandible, are opened from occlusal to gingival (Fig. 2). Always use the sl-instrument (REF 023-276-00) to open the brackets (Fig. 3).

The lids should be opened for the archwire to be inserted. Insert the pointed end of the sl-instrument into the opening on the lid between the occlusal bracket tie wings (Fig. 4a and 4b). Open the lid using the sl-instrument, levering it slightly towards vestibular/gingival (Fig. 5). The lid opens.

discovery® sl brackets have a mechanism (CapSafe™) which holds the lid open while the archwire is inserted. To activate the CapSafe™, press the opened lid using either a finger or the sl-instrument towards the gingiva until a resistance is felt (Fig. 6). The lid is now fixed securely in open position (Fig. 7).

Insert the archwire into the slot (Fig. 8) using the fork-shaped end of the sl-instrument if necessary. Close the lid from gingival to occlusal using either a finger or the sl-instrument (Fig. 9). The lid must be felt to click into place. This ensures the slot is closed securely (Fig. 10).

We recommend you use a lip retractor (REF 075-600-00 or 075-601-00) to avoid irritation to the lips or cheeks while the archwire is being changed.

2.8 Bracket debonding

The excellent bonding properties of discovery® sl brackets require special attention during debonding: for the debonding of discovery® sl brackets, we recommend using Weingart pliers (e.g. REF 003-120-00). Grasp the bracket from mesial-distal on the tie wings. Turn the bracket briefly to the right or the left, without pulling, to remove it from the tooth. The angled bracket removing pliers (REF 004-347-00) are particularly suitable for the posterior region as they shear the bracket off between the bracket base and the adhesive.

3. Information for the doctor/patient

- Please note that the teeth should not be cleaned while the archwire is being changed as long as the bracket lids are open. This would cause distortion to the delicate lid. This would in turn destroy the function of the closure mechanism and it would no longer be possible to close the lid.
- This product contains nickel and chrome. If the patient has a known allergy to one of the metals, this product should not be used. Further risks or side effects of the material of discovery® sl brackets are not known.

4. Composition

The composition is included in the materials list; please refer to the catalog or www.dentaurum.com.

5. Safety instructions

- There are no known risks or side effects (allergic reaction) for discovery® sl brackets.
- The product should not be used if there is a known allergic reaction to one or more of the material components.
- As the material 1.4542 of the self-ligating discovery® sl brackets has a low magnetizability, we recommend removing the brackets prior to an MRT examination.

6. Storage and shelf life

Store discovery® sl brackets in closed packaging.

7. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

Due to the mechanical stress during treatment and debonding, neither the dimensions nor the bonding strength can be guaranteed should the brackets be recycled.

8. Additional information

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog information.
- There are no particular precautions to be taken when disposing of discovery® sl brackets after treatment.

9. Availability

For the complete range of discovery® sl brackets, please see our catalog or www.dentaurum.com.

10. Quality

Dentaurum ensures faultless quality of its products. The recommendations in these instructions are based upon Dentaurum Implant's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

11. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

Fig. 1

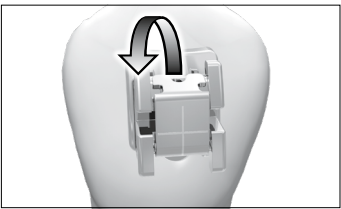
Fig. 2

Fig. 3

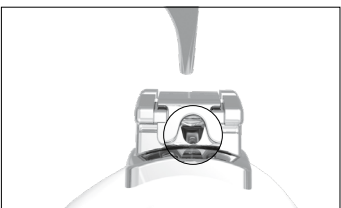
Fig. 4 a

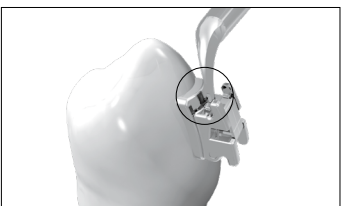
Fig. 4 b

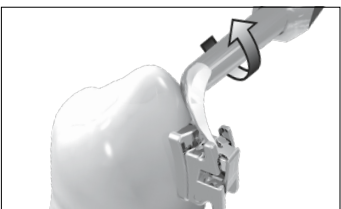
Fig. 5

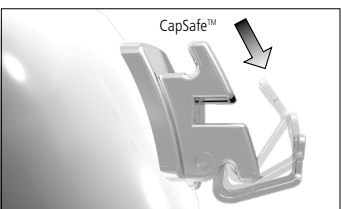
Fig. 6

Fig. 7

Fig. 8

Fig. 9

Fig. 10

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