Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

---

Federbolzenschrauben  
Piston spring screws  
Vis poussoir à piston  
Tornillos con resorte  
Viti a bullone

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## **Dear customer**

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at [www.dentaureum.com](http://www.dentaureum.com), even though you frequently use the same product.

### **1. Manufacturer**

Dentaureum GmbH & Co. KG

Turnstr. 31 | 75228 Ispringen | Germany

### **2. General description**

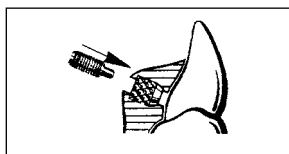
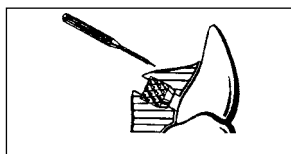
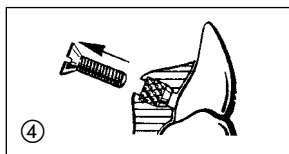
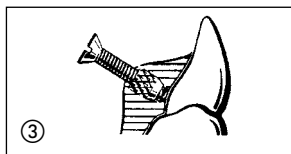
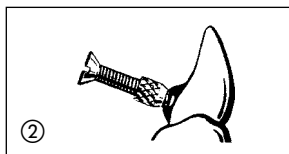
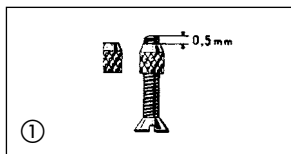
Piston spring screws are used to move single teeth during an orthodontic treatment using removable appliances.

### 3. Application of Orthocryl® in the salt-and-pepper technique

The mounting screw is inserted into the serrated housing, with 0.5 mm projecting beyond the conical end. The mounting screw must be positioned correctly against the tooth and held with sticky wax (keep the model dry to ensure adhesion of wax).

Alternatively, a 1.5 mm bur can be used to drill a 2.0 mm deep hole in the tooth on the model and insert the mounting screw into this. The housing should be approx. 0.5 mm away from the surface of the tooth.

Soak the model for 10–15 min in lukewarm water to release all entrapped air, then apply separating medium and use the salt-and-pepper technique to apply Orthocryl®.

## 4. Application of Orthocryl® LC

The mounting screw is fixed onto the model in the same way as with the salt-and-pepper technique. Put some separating medium on the dry, not soaked model and apply Orthocryl® LC according to the instructions for use and polymerize.

## 5. Finishing and use

After polymerization, unscrew the positioning screw from the model, finish and polish the plate and recut the thread using the reamer (REF 609-300-00).

The desired piston spring screw is then inserted. The procedure starts with the 4.0 mm screw which is later exchanged by a 6.0 mm screw and finally by an 8.0 mm screw, if the specific tooth has to be moved accordingly.

**Note: The screw-driver (REF 609-400-00) is not a medical device and therefore not suitable for the use in the mouth.**

## 6. Information about the composition

Information about the composition is included in the materials list, see catalog or [www.dentaurum.com](http://www.dentaurum.com).

## 7. Safety information

The product should not be used if there is a known allergy to one or more of the ingredients.

## 8. Additional tips

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

## 9. Availability

For the complete range please see our catalog or [www.dentaurum.com](http://www.dentaurum.com).

## 10. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

## 11. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of the symbols used on the label REF989-313-00).

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