Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

Fascination[®] / Fascination[®] 2

CE 0483

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Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaureum.com.

1. Manufacturer

Dentaureum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General description

Fascination®/Fascination®2 ceramic brackets are the aesthetic alternative to metal brackets. Their use corresponds in most respects with the use of metal brackets. Fascination®/Fascination®2 ceramic brackets are made of polycrystalline aluminium oxide. Thanks to their translucency, Fascination®/Fascination®2 ceramic brackets adopt the color of the natural tooth and are nearly invisible. They feature high fracture strength, low friction values and excellent bond strength.

The base surface of Fascination®/Fascination®2 ceramic brackets is coated with a silane layer. Furthermore, the base of Fascination®2 brackets is provided with a nub-structure. This ensures the optimal adhesive thickness for proper bonding and problem-free debonding. For the Roth system, Fascination®/Fascination®2 ceramic brackets are offered with an identification system (positioning guides), which facilitates the correct placement of the brackets.

Information on the use of the identification system can be found at the end of the instructions for use.

Use Fascination®/Fascination®2 only according to the instructions!

3. Working steps

3.1. Consulting patients

Inform the patient about the possible consequences and risks involved in treatment using brackets, in particular during debonding. This applies especially in the case of teeth with enamel that already exhibits signs of initial damage (cracks, crazing, splitting off, etc.).

3.2. Conditioning teeth

Adhere to the bracket adhesive manufacturer's instructions for conditioning the teeth.

3.3. Selecting bracket adhesive

Fascination®/Fascination®2 ceramic brackets can be bonded using all commercially available bracket adhesives. Adhere to the manufacturer's instructions for the bracket adhesive as well as the table „Application for adhesives“ in our Orthodontics Catalog.

Warning: Using so called No-Mix-Adhesives with Fascination®2 ceramic brackets can lead to insufficient mixing of the adhesive. This can result in a reduced bond of the bracket!

3.4. Removing and handling brackets

Grasp Fascination®/Fascination®2 ceramic brackets mesio-distally or at the bar of the positioning guide using clamping tweezers (e.g. REF 025-277-00) to remove them from the packaging and handle them according to the following steps.

The bonding surface of Fascination®/Fascination®2 ceramic brackets is coated with a silane layer to assure adhesion. This layer should not be touched or contaminated in any way, as this could otherwise detrimentally affect the bond.

3.5. Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, but enough to cover the entire bracket base. This guarantees optimal bond.

3.6. Placing and aligning brackets

After applying the adhesive, place, align and press the bracket, if necessary use positioning guides, in the usual way onto the tooth until the adhesive exudes from the sides. Completely remove the exuded excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

Warning: It is essential to avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary, use a palatal bite raise, e.g. Mini Mold (REF 024-000-00) or Bite-Bumper™ (REF 705-000-00).

3.7. Ligating arch wires

For initial treatment, we recommend Tensic®, rematitan® „LITE“ or translucent pearl arch wires. **Do not ligate wires with excessive force. Do not torque wires after ligating.** When adhesive has hardened (observe adhesive hardening times in the manufacturer's instructions for use) and the positioning guide has been removed, ligate as usual with wire ligatures (e.g. REF 768-036-/037-00, with white coating) or Dentalastics® plastic ligatures (e.g. 774-002-01).

Attention: Careful ligating avoids bracket damage!

3.8. Debonding brackets

For the debonding of Fascination®/Fascination®2 ceramic brackets, we recommend using Weingart pliers (REF 003-120-00). Using any other instrument can lead to bracket damage.

Remove arch wire from the bracket. To avoid bracket breakage while debonding, place a blue separating ring (e.g. REF 774-200-01) around the bracket base for protection. Grasp the bracket with Weingart tweezers (REF 003-120-00) on the mesial-distal side of the bracket base, directly at the tooth surface, but without applying pressure. Remove the bracket from the tooth with a short turn to the right or left side without pulling.

Note: When handled correctly, Fascination®/Fascination®2 ceramic brackets are not prone to breakage. Using other than recommended instruments can damage the ceramic brackets.

Should a breakage of the bracket occur due to incorrect handling, we recommend to scrape the bracket fragments and the adhesive carefully off the tooth enamel with bracket removing pliers (e.g. REF 003-349-00) or a scaler (e.g. REF 027-349-00).

Warning: Do not damage tooth enamel, danger of enamel fracture. Use proper evacuation procedures; protect everyone's eyes within the treatment area!

4. Information about the composition

Information about the composition is included in the materials list, see www.dentaurum.com.

5. Safety information

The product should not be used, if there is a known allergy to one or more of the ingredients.

6. Storage and shelf-life

Store Fascination®/Fascination®2 ceramic brackets in closed packaging. Do not expose to sunlight. Protect against heat and humidity. The shelf-life of the silane layer on the Fascination®/Fascination®2 ceramic brackets is limited; please refer to the product label for further information.

7. Information for single-use products

Fascination®/Fascination®2 ceramic brackets are intended for single use only. Reconditioning of Fascination®/Fascination®2 ceramic brackets (recycling) and reuse on patients are not permitted. Excessive stress during treatment and debonding can produce micro cracks in the ceramic, which can lead to breakage when reused!

8. Information about the identification system (positioning guides)

- 8.1.** The color of the positioning guide and the tooth symbol identify the tooth/bracket combination.
- 8.2.** Upper: right = red
 left = green
Lower: right = yellow
 left = blue
- 8.3.** The occlusal-gingival distance can be determined by means of the occlusal indicator. The distance between the occlusal end and the center of the slot is 6 mm.
- 8.4.** The tooth symbols always point from occlusal to distal.
- 8.5. Removing positioning guides**

When adhesive is hardened (according to manufacturer's instructions) remove the positioning guide at the mid section with a pair of tweezers, or with Weingart pliers (REF 003-120-00).

9. Additional tips

- In case of adhesion problems due to improper use, please use a new bracket, as the silane layer was damaged during application. This layer can only be applied by the manufacturer.
- For technical information, such as torque, angulation, size, etc., please refer to the product label or our Orthodontics Catalog.
- There are no particular precautions to be taken when disposing of Fascination®/Fascination®2 ceramic brackets after treatment.

10. Delivery program

For the complete range of our Fascination®/Fascination®2 products, please see our Orthodontics Catalog.

11. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

12. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols, REF 989-313-00).

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