DE

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EN

Dear customer,

Thank you for choosing a quality product from Dentaurum. Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

Herbst I is a classic retention hinge for welding onto bands. Depending on the aim of the treatment and the experience of the orthodontist, the Herbst appliance is suitable for the banding and the casting technique. This means maximum flexibility and enables individual implementation.

3. Intended purpose

Correction of distocclusions.

4. Intended users

All products are intended for dental professionals only.

5. Indications

Distocclusion/class II (unilateral/bilateral), molar distalization, gap closure (aplasia), temporomandibular dysfunction, sleep apnea therapy, dentoalveolar compensation of occlusion (elastics substitution).

6. Contraindications

The Herbst appliance made of stainless steel must not be used for patients with a nickel allergy. Warnings are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients for whom a fixed appliance can be placed as part of orthodontic treatment.

8. Fabrication of the Herbst I appliance

Cast the impressions, which include the waxed out bands fitted on the first premolars and molars, with dental stone (e. g. Rapidur®). Mount the models in the correct position in an articulator with plaster using the construction bite recorded by the dentist.

Bend and laser weld the half-round remaloy® wire (ø 0.75 x 1.5 mm) or round remaloy® wire (ø 1.0 mm) to the bands to form a lingual bow. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization.

Next laser weld the bases of the hinge joints to the bands. Alternatively, splints can be cast instead of using bands. The bases of the retention hinges should not be cast on, but welded afterwards, for accuracy and biocompatibility reasons.

Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the articulations between the upper and lower jaw operate without friction to ensure smooth functioning of the telescope mechanism. Friction can damage the appliance – especially the bands.

Fitting the telescope tubes and rods:

First, shorten the telescope tubes according to the construction bite and then the telescope rods, which are approx. 2 mm longer. The correct length should be checked again intra-orally and adjusted if necessary.

Note: “Double bands” should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the band already fitted (e. g. using the spot welding unit Assistent 3000, (REF 078-000-00)).

Generally the second band fits if it is 2 to 3 sizes larger than the first. Ready-welded “double bands” can be ordered from Dentaurum. The order form is available on request.

Fitting intra-orally:

The bands are fitted intra-orally using standard band cement. We recommend covering the bases of the articulations with wax to ensure that the cement does not penetrate into the thread. After the cement has set, the guidance tubes and sliding rods (already shortened) are attached to the bases.

If necessary, the Herbst I appliance can be activated using various spacer rings (please refer to our product range).

9. Composition

The Herbst I appliance is made of the following materials: DIN 1.4305 and DIN 1.4542. The material composition is included in the orthodontics materials list; please refer to the catalog or www.dentaurum.com. The SSCP is available at https://ec.europa.eu/tools/eudamed and www.dentaurum.com.

10. Safety information

The fit and tightness of the screws should be checked at regular intervals! If necessary, tighten the screws and secure them with a special screw securing adhesive, e. g. NOBIL GRIP. The product should not be used if there is a known allergic reaction to one or more of the material components.

If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

11. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

12. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly. For reasons of biocompatibility, welding (e.g. with a laser welding unit) should be chosen over soldering when preparing the appliance.

13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

14. Product range

For the complete range of Herbst products, please see the orthodontic catalog or visit www.dentaurum.com.

15. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

16. Explanation of symbols used on the label

 Please refer to the label. These Instructions for use and additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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