CE 0483

- Informationen zu Produkten finden Sie unter [www.dentaurum.com](http://www.dentaurum.com)
- For more information on our products, please visit [www.dentaurum.com](http://www.dentaurum.com)
- Vous trouverez toutes les informations sur nos produits sur [www.dentaurum.com](http://www.dentaurum.com)
- Descubra nuestros productos en [www.dentaurum.com](http://www.dentaurum.com)
- Informazioni su prodotti sono disponibili nel sito [www.dentaurum.com](http://www.dentaurum.com)


Stand der Information | Date of information | Mise à jour  
 Fecha de la información | Data dell'informazione: 2023-07

Änderungen vorbehalten | Subject to modifications |  
 Sous réserve de modifications | Reservado el derecho de modificación |  
 Con riserva di apportare modifiche

989-534-00 Printed by Dentaurum Germany 2023-11/BR1-8


Gebrauchsanweisung | Instructions for use |  
 Mode d'emploi | Modo de empleo | Modalità d'uso

**tomas®-pin SD**  
**tomas®-pin EP**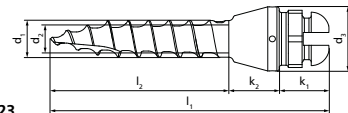

ISO 19023

tomas®-pin SD / SD N	
$d_1$	1,6 mm
$d_2$	1,2 mm
$d_3$	2,8 mm
$k_1$	2,2 mm
$k_2$	2,0 mm
$l_1$	10,2 / 12,2 / 14,2 mm
$l_2$	6,0 / 8,0 / 10,0 mm

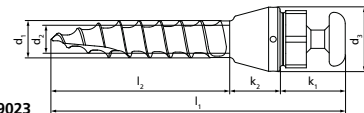ISO 19023

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### Dear customer,

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. Should you have any questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at [www.dentaurum.com](http://www.dentaurum.com), even though you may frequently use the same product.

### 1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

### 2. General description

A temporary skeletal anchorage option for orthodontic treatment has been created with the aid of an endosteally anchored mini screw (tomas<sup>®</sup>-pin). The tomas<sup>®</sup>-pin SD and tomas<sup>®</sup>-pin EP have a self-drilling (SD = self-drilling) thread (ø 1.6 mm) in different lengths (see point 10). The gingival collar has a height of 2.0 mm and a maximum ø of 2.8 mm. The head of the tomas<sup>®</sup>-pin SD (Fig. 1) has a cross slot (slot width: 0.56 mm / 22; slot depth: max. 1.15 mm). The tomas<sup>®</sup>-pin EP (Fig. 2) has a mushroom-shaped head. Different orthodontic appliances can be connected to the head of the tomas<sup>®</sup>-pin, according to the indication, to achieve or support the required tooth movement.

tomas<sup>®</sup> is a system of coordinated components for the insertion and coupling of the mini implants.

The tomas<sup>®</sup>-pin is manufactured from a titanium alloy in accordance with ASTM F 136 and ISO 5832-3. The fracture torque for the tomas<sup>®</sup>-pin is 30 Ncm.

### 3. Indications and contra-indications

The tomas<sup>®</sup>-pin is used to provide temporary orthodontic anchorage for the following treatments:

- Diastema closure with Class I occlusion.
- Distalization and mesialization of teeth.
- Uprighting of molars.
- Intrusion of teeth.
- Sliding mechanics in Class II.

- Preventing protrusion of incisor teeth.
- With oligodontia.
- Altering the tooth position as part of pre-prosthetic treatment.

### 3.1 General contraindications

The tomas<sup>®</sup>-pin should not be used in the case of immunodeficiency, steroid therapy, blood coagulation disorders, uncontrolled endocrine disorders, rheumatic ailments, bone system diseases, cirrhosis of the liver or any other acute illnesses.

### 3.2 Specific contraindications

The tomas<sup>®</sup>-pin should not be used in the case of osteomyelitis, cranial radiotherapy, recurring oral mucosa diseases or inadequate oral hygiene. Use is also contraindicated if there is no or insufficient bone availability in the planned insertion site.

## 4. Precautions

The tomas<sup>®</sup>-pin should only be inserted by orthodontists, dentists, oral surgeons or maxillofacial surgeons. Before use, operators should ensure that they have carefully read and followed the instructions for use supplied.

We recommend operators complete a relevant tomas<sup>®</sup> training course prior to working with the pin, as the instructions for use can only cover some of the many areas of application.

Only original tomas<sup>®</sup> components may be used according to the instructions for use.

All patients must be examined and given information on the tomas<sup>®</sup>-pin before use.

## 5. Form of presentation

The tomas<sup>®</sup>-pin SD and tomas<sup>®</sup>-pin EP are delivered both in sterile and non-sterile form, see the product overview listed under point 10. All other tomas<sup>®</sup> components are supplied non-sterile and must be sterilized before initial use. There is one exception: the tomas<sup>®</sup>-punch (gingiva remover) which is sterile on delivery.

### 5.1 Sterile pins – tomas<sup>®</sup>-pin SD and tomas<sup>®</sup>-pin EP

These tomas<sup>®</sup>-pin versions are delivered sterile, in a glass tube wrapped in blister foil packaging. The packaging should not be removed until shortly before insertion.

Dentaurum guarantees the sterility of the tomas<sup>®</sup>-pin until the expiration date on the packaging, provided that the original packaging is undamaged. Once

the time limit has been exceeded, the sterility can no longer be guaranteed and the tomas<sup>®</sup>-pin may no longer be used on a patient. If the sterile packaging is damaged, do not re-sterilize. Packaging that has been opened may not be re-sterilized. A tomas<sup>®</sup>-pin that has been removed from the packaging but has not been inserted, may also no longer be used or re-sterilized.

### 5.2 Non-sterile pins – tomas<sup>®</sup>-pin SD N and tomas<sup>®</sup>-pin EP N

The non-sterile tomas<sup>®</sup>-pin versions should be cleaned, disinfected and sterilized before clinical application. The steps that are described below have been validated (Medical Devices Services – Dr. Rossberger GmbH, Germany).

Throughout the entire procedure, from removing the pin from the packaging to insertion, the tomas<sup>®</sup>-pin thread should not be touched or contaminated in any way. Wear sterile gloves and only grip the head of the tomas<sup>®</sup>-pin with sterile tweezers. If several tomas<sup>®</sup>-pin are to be prepared, they should not come into contact with each other during cleaning, disinfection and sterilization.

#### 5.2.1 Manual cleaning of tomas<sup>®</sup>-pin

The tomas<sup>®</sup>-pin should be cleaned in an ultrasonic cleaner. We recommend using a 5 % n-propanol solution (consisting of 70 % n-propanol and de-ionized water) as the cleaning agent. Degas the solution for 5 min (time setting on the ultrasonic cleaner) before beginning the actual cleaning procedure.

Remove the tomas<sup>®</sup>-pin from the packaging and place it in the basket of the ultrasonic cleaner. Clean it in the ultrasonic unit for 8 min at 25 °C / 77 ° F. Then remove the cleaning basket. Allow the cleaning liquid to drain from the tomas<sup>®</sup>-pin and the basket. Then rinse thoroughly with sterile water or water with a low germ and endotoxin content.

The cleaning agent should be renewed regularly according to the manufacturer's instructions.

#### 5.2.2 Manual disinfection of the tomas<sup>®</sup>-pin

The pin is disinfected with 70% n-propanol. Immerse the cleaning basket for 20 min in the disinfectant (room temperature). Then remove the tomas<sup>®</sup>-pin, allow to drip dry and rinse thoroughly at least three times using sterile water or water with a low germ and endotoxin content. Allow the tomas<sup>®</sup>-pin to dry for 15 min at room temperature and then pack for sterilization.

The disinfectant should be renewed regularly according to the manufacturer's instructions.

#### 5.2.3 Mechanical cleaning and disinfection of the tomas<sup>®</sup>-pin

The tomas<sup>®</sup>-pin can be cleaned and disinfected in a thermodisinfector, which is DGHM (German Society for Hygiene and Microbiology) or FDA approved

or has a CE mark (DIN EN ISO 15883). The cleaning agent and disinfectant used should have a proven efficacy and be compatible with the material of the tomas®-pin. Adhere to the concentrations prescribed by the manufacturer.

Place the tomas®-pin in a suitable tray in the thermodisinfectant and start the program (DES-VAR-MEDIZYM). The thermal disinfection program must consist of the following sub-steps:

- Pre-rinse (rinse temperature:  $10 \pm 2 \text{ }^\circ\text{C} / 50 \pm 36 \text{ }^\circ\text{F}$ ; rinse time: 1 min)
- Cleaning (cleaning temperature:  $45 \pm 2 \text{ }^\circ\text{C} / 113 \pm 36 \text{ }^\circ\text{F}$ ; cleaning time: 5 min, cleaning agent: neodisher MediZym, concentration 0.2 % [2 ml / l])
- Rinse (sterile water or water with a low germ and endotoxin content; rinse temperature:  $10 \pm 2 \text{ }^\circ\text{C} / 50 \pm 36 \text{ }^\circ\text{F}$ ; rinse time: 2 min)
- Thermal disinfection (disinfection temperature:  $93 \pm 2 \text{ }^\circ\text{C} / 199 \pm 36 \text{ }^\circ\text{F}$ ; disinfection time: 10 min)
- Drying

When the program has finished, remove the tomas®-pin from the disinfectant, check and pack immediately for sterilization.

#### 5.2.4 Sterilization of the tomas®-pin

The dry tomas®-pin can be sterilized separately or together with other tomas® components in the tomas®-tool tray. Place the tomas®-pin in one of the empty inserts in the tray for sterilization.

Shrink-wrap each tomas®-pin in a suitably-sized foil bag for separate sterilization. Sterilize the pins in an autoclave. Adhere strictly to the autoclave manufacturer's instructions for use. Steam Complete steam sterilization at a temperature of  $134 \text{ }^\circ\text{C} / 273 \text{ }^\circ\text{F}$  with a minimum hold time of 25 min.

### 6. Insertion of the tomas®-pin SD and tomas®-pin EP

#### 6.1 General information

Adhere to all essential hygienic measures for invasive surgery such as sterile surgical area, sterile gloves, facemask etc. during insertion of the tomas®-pin. To ensure that the tomas®-pin functions safely it is essential that it has stable anchorage in the bone (primary stability) and that the head is placed in the region of the attached gingiva (gingiva alveolaris). When using the tomas®-pin as an anchorage unit, ensure that the head and surrounding soft tissue are not subjected to detrimental mechanical influences (e. g. movement of the mucosa, interference from bands and/or the tongue or manipulation).

In the maxilla, the tomas®-pin can be placed buccally, palatally or on the alveolar ridge. The tomas®-pin EP is primarily intended for insertion in the anterior palatal region. In the mandible the pin shall always be inserted

buccally and on the centre of the alveolar ridge. In accordance with the latest findings, insertion of the tomas®-pin in the region of the mobile mucosa, on the lingual side of the mandible or close to extraction wounds, tooth follicles or deciduous teeth is not advised.

#### 6.2 Procedure

- Select the insertion site. Subsequent loading should, as far as possible, be at an angle of  $90^\circ$  or at right angles to the long axis of the tomas®-pin.
- Local anaesthesia
- We recommend measuring the thickness of the mucosa in the path of insertion and punching out the mucosa using the tomas®-punch (REF 302-001-00).
- If required, pilot drilling for perforation of the cortical bone.
- Pick up tomas®-pin with insertion instrument.
- Insertion of the tomas®-pin in the bone. Please follow section 6.2.3.
- The tomas®-pin can be loaded immediately.
- Link the tomas®-pin with the orthodontic appliance (direct anchorage) or with individual teeth or tooth groups (indirect anchorage). The maximum load should not exceed 2 N (approx. 200 g).

Generally, pilot drilling is not required before insertion of the tomas®-pin when used in the maxilla. Pilot drilling is required for insertion in the mandible if the compact bone is thicker than 2 mm. The pilot hole should be drilled to a maximum depth of 4 mm, using the sterilizable, reusable tomas®-drill SD 1.1 short pilot drill (REF 302-103-00). The optimal motor speed is  $800 \text{ min}^{-1}$  (1500  $\text{min}^{-1}$  max.); ensure that adequate cooling is provided using a sterile, physiological saline solution.

#### 6.2.1 Picking up the tomas®-pin (sterile version)

- Remove the glass tube from the sterile packaging.
- When the glass tube is opened, the tomas®-pin sits in the metal holder of the cap and is secured in position by a silicone plug against accidental falling out.
- Hold the silicone plug upwards and carefully remove it.
- Insert instrument into the metal holder and press until the head of the tomas®-pin perceptibly engages in the retention of the instrument.  
**Caution:** The head of the tomas®-pin must engage securely in the holder of the insertion instrument.
- The tomas®-pin can now be removed directly from the metal holder using the insertion instruments and can be inserted.

### 6.2.2 Picking up the tomas®-pin (non-sterile version)

Place the insertion instrument along the longitudinal axis on the head of the tomas®-pin and press until the instrument perceptibly engages in the retention. **Caution:** The head of the tomas®-pin must engage securely in the holder of the insertion instrument. The thread of the tomas®-pin should not be touched during the procedure.

### 6.2.3 Insertion in the bone

The tomas®-pin can be inserted in the bone manually or using a handpiece. The tomas®-screwdriver (REF 302-004-10) or tomas®-applicator (REF 302-004-20 or 302-004-70) with or without the tomas®-wheel (REF 302-004-30) can be used as the manual insertion instrument. The tomas®-torque ratchet (REF 302-004-40) is available as an additional aid for the manual procedure. The torque of the ratchet should be set to a max. of 20 Ncm; please refer to the relevant instructions for use. The pin should be inserted using a rotary movement that is as uniform as possible.

The tomas®-driver (REF 302-004-50) or tomas®-guide driver (REF 302-080-00) is available for insertion using a handpiece. A motor and contra-angle with adjustable torque should be used. The torque should be set to a max. of 20 Ncm and the motor speed to a max. of 25 min<sup>-1</sup>. Alternatively the manually turned unit mod. acc. to Pauls (REF 302-004-81) – without motor – can be used. Instructions for use are available for this handpiece.

The insertion depth depends on the thickness of the gingiva. If the gingiva is thicker than 2 mm, wind the tomas®-pin into the bone until the insertion instrument touches the gingiva. If the gingiva is thinner than 2 mm, insert the tomas®-pin into the bone only until the junction point between the thread and gingival collar. The gingival collar (height 2 mm) is used as a means of control. If the tomas®-pin is inserted deeper, the thread may lose its grip in the bone. In this case there is no adequate primary stability. The maximum insertion torque is highly dependent on the bone structure. The insertion torque can be controlled with the help of the tomas®-torque ratchet (REF 302-004-40), the manually turned unit mod. acc. to Pauls (REF 302-004-81) or the motor with adjustable torque. This should not exceed the fracture torque for the tomas®-pin of 30 Ncm.

If you need to remove the insertion instrument to check the situation, reinstall it the way it was, exactly along the longitudinal axis of the tomas®-pin. Otherwise the head can be damaged!

## 7. Removal of the tomas®-pin

- Local anesthetic (optional)
- Remove the coupling elements
- Unscrew the tomas®-pin manually with the tomas®-applicator (REF 302-004-20 or 302-004-70)/tomas®-wheel (REF 302-004-30), tomas®-screwdriver (REF 302-004-10) or mechanically with the tomas®-driver (REF 302-004-50) or tomas®-guide driver (REF 302-080-00).

## 8. Composition

The composition is included in the materials list; please refer to the catalog or [www.dentaurum.com](http://www.dentaurum.com).

## 9. Storage and shelf life

The tomas®-pin should always be stored in a dry, dark place at room temperature in the original packaging. The tomas®-pin should not be used after the expiry date (see label).

## 10. Information for single use products

The tomas®-pin is designed for single use only. Reconditioning of a tomas®-pin that has already been used (recycling) and its reuse on a patient are not permitted.

## 11. Further information

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident is informed accordingly.

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### 13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

### 14. Quality

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

### 15. Explanation of symbols

Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

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