
Gebrauchsanweisung | Instructions for use | Mode d'emploi |
Modo de empleo | Modalità d'uso

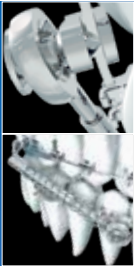SUS2
Sabbagh Universal Spring

CE 0483

D
DENTAURUM

Fig. 1 Bestandteile der SUS² – Sabbagh Universal Spring.
Components of the SUS² – Sabbagh Universal Spring.
Composants du SUS² – Sabbagh Universal Spring.
Componentes del SUS² – Sabbagh Universal Spring.
Componenti del SUS² – Sabbagh Universal Spring.

Position für Turbofeder oder Distanzringe
Position for turbo spring or spacer rings
Position pour ressort turbo ou anneaux d'écartement
Posición para el turbo-resorte o anillos distanciadores
Posizione per molla turbo o anelli distanziatori

Fixierschraube
Fixing screw
Vis de fixation
Tornillo de fijación
Vite di fissaggio

Teleskopstange mit Ring
Telescope rod with ring
Tige télescopique avec bague
Varilla telescópica con anillo
Perno telescopico con anello

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.....

Dear customer,

Thank you for choosing a quality product from Dentaaurum.

Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

The SUS² - Sabbagh Universal Spring - produces dentoalveolar adjustments when used as a spring with maximum spring activation (Jasper-Jumper™ effect/elastics substitute). To correct distoclusion or attain a reposition effect, set the spring activation to minimum (Herbst appliance/activator effect).

3. Intended purpose

For universal intermaxillary treatment of class II.

4. Intended users

All products are intended for dental professionals only.

5. Indications

Distocclusions/class II (uni and bilateral), molar distalization, gap closure (aplasia), temporomandibular joint disorders and sleep apnea therapy.

Fig. 2 SUS² in situ.

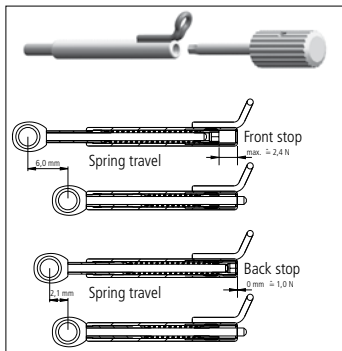
Fig. 3 Setting the required length or force of the SUS², above as a spring, below as a hinge (Herbst effect).

6. Contraindications

The SUS² appliance made of stainless steel must not be used for patients with a nickel allergy. Warnings are found on the respective product labels. Also see 11. Composition.

7. Patient target group

All patients for whom a fixed appliance can be placed as part of orthodontic treatment.

8. Processing steps

The SUS² as a spring: the inner spring of the SUS² is set at maximum activation on delivery (spring travel 6.0 mm, force approx. 2.4 N when fully compressed). This reflects the most common indication for the appliance: its use, in the same way as the Jasper-Jumper™, with fitted bands/brackets without laboratory-fabricated retention. Its effect, mainly dentoalveolar, is comparable to wearing headgear with class II elastics for 24 hours (Fig. 3).

The SUS² as a hinge: the inner spring can be deactivated using the hex key (REF 607-144-00) by turning the hexagon socket screw of the telescope element one full turn anti-clockwise (Fig. 3). This converts the flexible SUS² spring into a rigid SUS² hinge. (Spring travel 2.1 mm, reserve force when fully compressed approx. 1.0 N..

1. Insert the SUS² ball retainer clasp distally through the loop of the SUS² telescope element (Fig. 4).
2. Insert the SUS² ball retainer clasp together with the SUS² telescope element into the headgear tube from distal and bend in mesial direction. It is essential to maintain a gap of approx. 4 mm between the head of the ball retainer clasp and the distal end of the headgear tube (Fig. 5). This ensures that there is enough clearance for the SUS².
3. Bend the SUS² ball retainer clasp between the band and hook of the buccal tube, or shorten it and secure in position using composite/acrylic (**important!**) (Fig. 6).

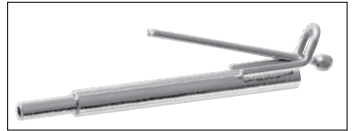
Fig. 4 Inserting the SUS² ball retainer clasp into the loop of the SUS² telescope element.

Fig. 5 SUS² ball retainer clasp in position.

Fig. 6 SUS² ball retainer clasp bent between the band and hook of buccal tube.

Fig. 7 SUS² ball retainer clasp bent buccally by 20 to 30 degrees.

Fig. 8 Inserting the SUS² arch adapter on the stainless steel archwire.

Fig. 9 L bend.

4. Bend the SUS² ball retainer clasp at the distal end of the headgear tube by approx. 20 degrees in the buccal direction and finely adjust (Fig. 7).
5. Insert the SUS² telescope rod into the SUS² telescope element (guidance tube and middle telescope tube) which is fixed in position in the maxilla.
6. Attach the preassembled SUS² arch adapter (closed side facing upwards) into the stainless steel archwire from above (minimum 0.41 mm x 0.56 mm/16 x 22) between the lower canine and the first lower premolar (Fig. 8). The SUS² arch adapter can be also fixed in position between the lower lateral incisor and lower canine if required, e.g. in the case of a missing premolar. L bend (Fig. 9).

Fully tighten the hexagon socket screw using the hexagon socket key (REF 607-144-00). The arch adapter should not come into contact with the bracket when tightening the screw.

9. Setting and activating the SUS²

SUS² can be adjusted to the individual needs as follows, either individually or in combination with other devices:

SUS² as a spring: The SUS² turbo spring, with an additional force of approx. 3.0 N, is available if greater activation is required. The turbo spring is placed on the SUS² telescope element so that the three closed coils are pointing towards the SUS² telescope rod (Fig. 10).

SUS² as a hinge: Standard activation is completed using the open 1 mm and 2 mm spacer rings supplied (REF 607-103-11 and 607-104-11). Depending on requirements, one or several spacer rings can be clamped on the SUS² telescope rod using special crimping pliers (REF 003-710-00) or Weingart universal pliers (REF 003-120-00) (Fig. 11).

Caution: The SUS² can only be used as a hinge on cast or soldered appliances that are retained using bands – the appliance can only be activated using slotted spacer rings (no SUS² turbo spring).

Fig. 10 Additional activation when using as a spring.

Fig. 11 Additional activation when using as a hinge.

5. Tips and tricks

Problem	Possible cause	Remedy
Patient bites on the metal or telescope element catches on the premolar brackets	SUS ² ball retainer clasp not long enough distally, not bent or not adjusted	<ul style="list-style-type: none"> - SUS² ball retainer clasp should be approx. 4 mm long distally from the headgear tube and should be bent (see Figs. 5-7) - Adjust the SUS² ball retainer clasp buccally or, if required, caudally
Patient cannot bite together properly/SUS ² is too long	SUS ² too long due to extraction or very narrow premolars	<ul style="list-style-type: none"> - Place the SUS² telescope rod/arch adapter between lower lateral incisor and lower canine instead of between the lower canine and the first premolar - L bend (Fig. 9)
Middle telescope tube jams in the guidance tube (no spring function)	Calculus in the telescope thread, patient with a heavy build-up of calculus	Compensate by fitting SUS ² turbo spring, if necessary replace the SUS ² telescope element (Vaseline in the guidance tube helps reduce deposit build-up)
The SUS ² telescope rod slides out of the SUS ² telescope element when the mouth is fully open	Patient's mouth opens wider than average, condylar hypermobility	<ul style="list-style-type: none"> - Place the SUS² arch adapter further distally if possible - Explain to the patient how the parts can be reinserted - Physiotherapy/rotation exercises
Lower anterior teeth are severely protruded	Too little anchorage and/or overactivation	<ul style="list-style-type: none"> - Insert a strong rectangular steel archwire, min. 0.41 x 0.56 mm / 16 x 22, with lingual crown torque and bend it distally - Elasto-Force plastic chain from 6-6 - Temporary anchorage with tomas®-pin - Reduce the force setting
SUS ² telescope rod catches / rubs as mouth is closed	SUS ² telescope rod has bent after uncontrolled mouth closure (e.g. after yawning)	Replace the SUS ² telescope rod (can be ordered separately)
Brackets loosen frequently	<ul style="list-style-type: none"> - Excessive force (incorrect anchorage/indication) - Patient bites on the SUS² telescope element (see above) 	<ul style="list-style-type: none"> - Rigid hinge and slotted spacer rings (Herbst version/skeletal) only on cemented bands similar to Herbst retention - Only use the SUS² with activated middle telescope tube on multi-bracket anchorage (dental)
Fixing screw on the SUS ² arch adapter comes loose	Fixing screw too loose or not tightened enough	Tighten the fixing screw and fix in position using a screw securing adhesive (e.g. Ceka® Bond)

11. Composition

SUS² is made of the following materials: DIN 1.4305, DIN 1.4310, DIN 1.4301 and NiTi alloy. The material composition is included in the orthodontics materials list; please refer to the catalog or www.dentaurum.com. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

12. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

13. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

14. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

15. Product range

For the complete product range of SUS², please see our orthodontics catalog or www.dentaurum.com.

16. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

17. Explanation of symbols used on the label

Please refer to the label. These Instructions for use and additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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