Estimado cliente

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**Dear customer**

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General description

The tomas®-aligner hook is a tool for horizontal and vertical tooth movements during orthodontic treatment. It can aid movements such as the mesialization, distalization, angulation, intrusion of single teeth or groups of teeth. The tomas®-aligner hook consists of a prefabricated hook that can be individually shortened and a mesh base. It can be glued to a tooth or an aligner.

3. Precautions

Before use, operators should ensure that they have carefully read and followed the instructions for use supplied.

Use elastic elements (tension spring, elastic ring or chain) to connect the tomas®-aligner hook to a tooth or an aligner to the orthodontic appliance. A correctly bonded connection is secure. Due to safety reasons, the tomas®-aligner hook has a slot between the hook and the bonding base. The slot can be used for a safety thread (REF 775-210-01) or wire (REF 500-025-00) that can be fixed to the neighboring teeth or the aligner.

4. Use

With the tomas®-aligner hooks you can vary the force of an elastic element (tension spring, elastic ring or chain) in relation to the center of resistance. The hooks can be shortened individually. The tomas®-aligner hook can be used as connection of the following element combinations:

- Aligner – tomas®-pin EP
- Aligner – Tooth
- Tooth – tomas®-pin EP
- Tooth – Tooth

The tomas®-aligner hook can be bonded with all common bracket adhesives on a tooth. Use a special adhesive (e.g. Bond Aligner™, Reliance) for bonding on the aligner plastic. For a secure bond, observe the instructions for use of the adhesive.

5. Information about the composition

Information about the composition is included in the materials list, see catalog or www.dentaurum.com.

6. Safety information

The product should not be used if there is a known allergy to one or more of the ingredients.

7. Information for single use products

The tomas®-aligner hook is designed for single use only. Reconditioning of tomas®-aligner hooks that have been inserted previously (recycling) or reuse on patients is not permitted.

8. Product overview

tomas®-aligner hook, Upper left/Lower right **REF 302-021-00**
tomas®-aligner hook, Upper right/Lower left **REF 302-021-10**

9. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

10. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

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- Contention – tomas®-pin EP
- Contention – dent
- Dent – tomas®-pin EP
- Dent – dent

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