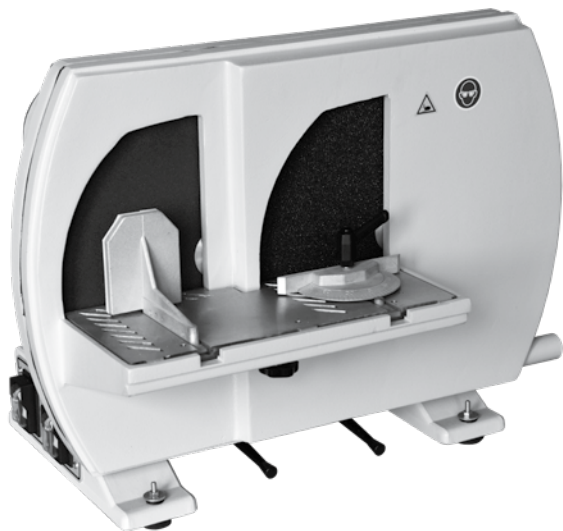
Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo

Doppelscheiben-Modelltrimmer
Dual wheel model trimmer
Taille-plâtre double-meule
Recortadora de modelos de doble disco

REF 091-100-00

Bedienungselemente

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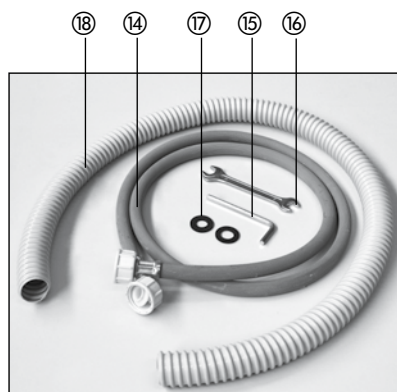
Druckdatum: 03.11.15

Operating elements

- ① witch for left corundum disc
- ② Switch for right corundum disc
- ③ Water control lever
- ④ Connection for drain hose
- ⑤ Angulation device
- ⑥ Model positioning tool
- ⑦ Coarse corundum disc, right side
- ⑧ Fine corundum disc, left side
- ⑨ Bolt for workplate

- ⑩ Connection point for water supply (solenoid valve)
- ⑪ Power plug
- ⑫ Domed nuts
- ⑬ Fuse F15AL 6,3x32 mm
- ⑭ Water supply hose
- ⑮ Allen wrench size 5
- ⑯ Spanner wrench size 10/12
- ⑰ Seals for water supply hose n
- ⑱ Drain hose

Dear customer

Thank you for choosing a quality product from Dentaaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaaurum.com, even though you frequently use the same product.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. Safety information

Read through the instructions for installing and operating the unit carefully before switching it on.

Caution!

- **Keep the working area free of all unneeded objects.**
- **Do not allow water to enter the ventilation slots. Do not block them.**

2.1 Danger symbols and markings

In the instructions for use, the following symbols are used to indicate sources of danger to the user:

This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

This symbol gives important information on the correct use of the unit. Disregarding this information may cause the unit to malfunction.

3. General notes and disposal

There are overall pictures of the unit on pages 10 and 11 where the operating elements are marked and numbered. Reference is also made to these numbers in the text.

When a Dentaaurum product is finally no longer in use the disposal regulations of that particular country apply. Dentaaurum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

**Note:**

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaurum or the dental trade.

4. Unpacking the unit

To open the wooden crate, remove the screws at the side and lift off the top of the crate. Remove the four screws that attach the Dual wheel model trimmer to the base of the pallet and carefully lift it out of the crate. We recommend that you retain the crate and hardware in case it is ever necessary for you to return the unit to us.

5. Installation

- Connect the water supply to the water inlet point ⑩ located above the right leg of the unit, using the water supply hose ⑭.
- Connect the drain hose ⑮ to the drain hose connection ④ on the unit. Place the end of the hose so that the water from the unit flows to the sink drain. The hose should be bent evenly with no kinks to impede water flow. The water drain hose must not be placed in an upward pointing position, in order to prevent backwater.
- The unit should be positioned so that the water can flow out of the unit completely.
- We recommend to install a plaster drap to protect your plumbing system!
- Plug the power cord ⑪ into the proper electricity supply..

Be certain that the power supply is the correct voltage for your unit.

REF 091-100-00 230 V

Warning! To avoid the risk of electric shock, the Dual wheel model trimmer should never be allowed to stand in water!

We recommend to connect the unit to a RCD.

6. Before beginning work

Caution!

- Pieces of plaster or corundum may fly off the model or disc.
- Always wear protective goggles when using the Dual wheel model trimmer.
- The discs rotate at high speed.
- Never touch the disc when it is rotating!

Turn on the water supply. The Dual wheel model trimmer is fitted with a solenoid valve ⑩ that automatically turns the water supply on and off. When the unit is switched on, water flows to the grinding discs.

- To switch the unit on, turn the respective switch to position "ON".
- The motors ① and ② can be switched on and off independent of each other.
- The switch remains in the "ON" position either until there is a power cut or the unit is switched off. Safety circuit: to prevent motors from starting automatically following a power cut.
- The water supply can be controlled for each disc by using the water control lever ③.

7. After completing work

- To flush out the drain hose, allow the unit to run for 30 seconds before turning off.
- Turn off the water supply and move the motor switches ① and ② to "0" (lower position).

Note: Allow the disc to stop. Do not stop them with an object.

8. Equipment

The Dual wheel model trimmer is fitted with two special devices.

They are:

The Model Positioning Tool f. This is used to apply pressure to the model during trimming. It protects the model from damage and avoids injury to the operator's hands. The Angulation Device e. This device allows you to trim models to a predetermined angle.

9. Maintenance

Caution! Always unplug the unit before cleaning or carrying out any maintenance work. Never clean the unit by holding it under running water!

The guide system for the model positioning tool and the angulation device must be cleaned regularly. To clean the workplate, remove the bolt ⑨ under the plate. The workplate can now be removed and cleaned.

The Dual wheel model trimmer should be cleaned externally from plaster residues on a weekly basis. Every 3 weeks remove the front casing in order to clean the plaster deposits from the water pipe. To do this, remove the acorn nuts ⑫ using the enclosed key ⑬ and then remove the front casing.

Check the hoses and couplings at regular intervals to ensure that they do not leak.

10. Changing the corundum disc

The unit is fitted with a fine ⑧ and a coarse ⑦ corundum disc. Both sides of the disc can be used:

Caution! Unplug the unit from the electricity supply (Danger of accidents)

- Unscrew the domed nuts ⑫ using the spanner wrench ⑬ and remove the front panel.
- Using the spanner wrench ⑬ remove the coupling for the water supply.
- The corundum disc can now be removed with the Allen wrench ⑮ and either be turned around or replaced.

Before reusing a disc, check it to be sure it is undamaged!

- Mount the corundum disc in the correct position against the disc holder.
- Reconnect the water supply and direct it towards the center of the disc.
- Clean the rubber seal at the front and replace the front panel.

11. Technical data

Voltage	230 V/50/60 Hz	REF 091-100-00
Motor power	2 x 450 W	
Motor speed	1400 min ⁻¹	
Dimensions	Width 53 cm	
	Height 40 cm	
	Depth 38.5 cm	
Weight	39 kg	
Color	Front grey	
	Rear black	
Discs	Corundum, coarse	REF 091-101-00
	Corundum, fine	REF 091-102-00
	Diamond disc, coarse	REF 091-105-00

12. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

13.

EC-Declaration of Conformity

according Machinery Directive 2006/42/EC, Annex IIA

DENTAURUM GmbH & Co. KG

Turnstr. 31

75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of unit:

Dual wheel model trimmer

REF:

091-100-00

Start with No.:

14.1514.1

EC guidelines

Machinery directive (2006/42/EC)

Directive relating to electromagnetic compatibility (2004/108/EC)

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Notes

[illegible]

This image shows a full page of white paper with horizontal dotted lines, typical of primary school handwriting practice paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- ➔ Informationen zu Produkten und Serviceleistungen finden Sie unter www.dentaurum.com
- ➔ For more information on our products and services, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits et services sur www.dentaurum.com
- ➔ Descubra nuestros productos y servicios en www.dentaurum.com
- ➔ Informazioni su prodotti e servizi sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 2023-10

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications
Reservado el derecho de modificación | Con riserva di apportare modifiche

D
DENTAURUM