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Dentaurum GmbH & Co. KG  
Turnstr. 31 · 75228 Ispringen · Germany

## 2. Product description

Orthocryl® / Orthocryl® EQ are cold polymers that are created by mixing a liquid (monomer, methyl methacrylate “MMA”) with a powder (polymer, polymethyl methacrylate “PMMA”). Liquids and powders are available in different colorations.

Accessories such as Orthocryl® Disco glitter and decals are intended to be used for removable orthodontic appliances only.

**Orthocryl®** attains optimal stability when used with the salt-and-pepper technique. It is also well suited for the doughing technique. When used with the doughing technique, **Orthocryl® EQ** attains optimal flow behavior. It is also well suited for the salt-and-pepper technique.

## 3. Intended purpose

Orthodontic acrylics are used for the fabrication of orthodontic appliances.

## 4. Intended users

All products are intended for dental professionals only.

## 5. Indications

Misalignment of teeth and jaws that are to be treated with orthodontic appliances.

## 6. Contraindications

For users/patients who are allergic to the ingredients in cold-curing acrylics. See 13. Composition.

## 7. Patient target group

Appliances fabricated by the specialist user are used for the treatment of children, adolescents and adults.

## 8. Processing steps

To avoid direct skin contact when processing the Orthocryl® components, it is recommended to wear face protection/ safety glasses (REF 162-920-00) as well as special protective gloves, e.g. Dermatril® disposable protective gloves made of special nitrile (REF 162-907-00, 162-908-00 and 162-909-00), see the respective safety data sheets for more information. To protect against the vapors of Orthocryl® liquid (monomer) during processing, we recommend working under an extraction hood or with an extraction device (e.g. REF 072-001-00).

### 8.1. Stone model/ plastic model

Stone models must be **“vented”** during the first step in processing. Otherwise, air bubbles from the stone model could lift the acrylic or penetrate into the acrylic. Models should therefore be placed in lukewarm water for approx. 10 minutes – 25 minutes, depending on the quality of the stone, until no more air bubbles rise from the stone.

This step is not necessary in the case of plastic models.

When working with stone models, we recommend using a special separating medium (REF 162-800-00) in addition after soaking in order to attain a smooth, silky basal surface on the appliance and to avoid discolorations and stone inclusions.

Plastic models **must** be insulated before Orthocryl® is applied in order to prevent the plastics from bonding. We recommend the 3D separating medium (REF 162-801-00), not available in the USA.

### **8.1.1 Application of the separating medium (REF 162-800-00) on stone models**

- Blow the soaked model carefully with compressed air.
- Apply the separating medium from the dosing bottle (REF 162-100-00) directly on the soaked, still slightly damp model and spread evenly using a brush.
- Stand the model on its back (tuber plane) so that excess liquid separating medium can drain off.
- After approx. 30 seconds, carefully blow away separating medium residues or droplets with compressed air and, if necessary, remove residues from the screw and wire retentions with a dry brush. Processing of the acrylic can then commence immediately using the salt-and-pepper technique (8.2) or the doughing technique (8.3).

### **8.1.2 Application of the 3D separating medium (REF 162-801-00) on plastic models**

For the fabrication of braces, we recommend high-quality full plastic models produced using the DLP process (Digital Light Processing), e.g. prime4me® base model (REF 430-110-10) or the FFF process (Fused Filament Fabrication) with FILAPRO® (REF 440-000-00) and the filaments FILAPRO® model gray (REF 440-200-00), model white (REF 440-201-00) and model cream (REF 440-202-00). Besides being highly accurate and therefore ensuring very precise fitting, these models allow for the usual process in the laboratory.

Due to the large variety of materials for model printing, it is essential to test the suitability of the 3D separating medium before use.

- Apply the 3D separating medium evenly with a separating brush over the entire model before the expansion screw(s) and wire elements are finally fixed in place and allow the separating medium to dry thoroughly. To this end, apply the separating medium from the dosing bottle (REF 162-100-00) directly on the model and apply evenly using a brush.

- Stand the model on its back (tuber plane) so that excess separating medium can drain off.
- Allow the separating layer to dry thoroughly. The separating medium is completely dry when it no longer feels sticky.
- Repeat the process to obtain a sufficient, full-surface separating layer.
- In order to accelerate drying of the separating medium or to shorten the drying phases, careful blowing with compressed air or a household hair dryer can be helpful. For faster drying of the separating medium, the model can also be placed in a light curing unit for a few minutes\*.

\*The time required in the light curing unit may vary depending on the size of the workpiece and the amount of separating medium. The drying time also depends on the type of device – with the Solidilite LED unit from Shofu, up to 3 minutes should be sufficient.

- If necessary, block out any tooth and jaw undercuts and fixate the expansion screw(s) and wire elements in the usual manner on the dry, insulated plastic model.
- Processing of the acrylic can then commence immediately using the salt-and-pepper technique (8.2) or the doughing technique (8.3).

## **8.2. Salt-and-pepper technique**

Starting with the powder (polymer), apply powder and liquid (monomer) alternately with a slight pendulum motion directly onto the prepared working cast.

Only apply as much liquid as the powder can absorb. The material should not flow away!

To avoid the formation of air pockets under screws, apply a slightly larger amount of liquid to begin with so that the acrylic spreads under the screws.

As a final step, always apply enough powder to leave a dry powder layer. This ensures an optimal fit and only minimal shrinkage.

### 8.3. Doughing technique

Mix Orthocryl®/Orthocryl® EQ in a silicone mixing bowl at a ratio of 2.5 : 1 (by volume powder : fluid).

Allow to swell – Orthocryl®, 5 minutes – 7 minutes, Orthocryl® EQ 3 minutes – 5 minutes (possibly shorter if room temperature is high), then apply the acrylic dough directly on the model.

### 8.4. Processing Orthocryl® color concentrates

Orthocryl® liquids can be shaded using the Orthocryl® color concentrates. Precise dosing of the color concentrates can be achieved with the dispensing pipette (REF 162-101-00). The recommended mixing ratios refer to the volume of color concentrate to Orthocryl® Clear liquid.

Our recommendations for standard colors:

red 1 : 10                      blue 1 : 50

For more intense coloring, we recommend a maximum concentration of:

red 1 : 5                      blue 1 : 5

### 8.5. Processing Orthocryl® Disco glitter

Add Orthocryl® Disco glitter to the Orthocryl® powder. We recommend only adding a small amount of glitter, max. mixing ratio:

Glitter (iris, red, blue, silver, gold):

1 g Glitter : 100 g Orthocryl® powder

1 ml Glitter : 100 ml Orthocryl® powder

Glitter (mother-of-pearl):

0.5 g Glitter : 100 g Orthocryl® powder

1 ml Glitter : 100 ml Orthocryl® powder

## 8.6. Processing decals

The decals – for visually enhancing appliances – work particularly well with highly transparent acrylics. The decals and the acrylic do not bond together chemically so it is important to embed them completely in the acrylic! Avoid abrading the decals during processing.

## 8.7. Processing of Orthocryl® black & white

When processing Orthocryl® black & white, **bear in mind that the colors are very opaque.** Orthocryl® Disco glitter and decals lose their impact in opaque acrylic. In particular during the finishing phase, avoid grinding the screws and wire elements. The opacity of Orthocryl® black & white can be reduced by adding clear Orthocryl®.

Mixing 50% clear polymer with 50% black or white polymer will reduce the opacity so that wire elements, screws etc. become visible in the Orthocryl®, yet the acrylic will still appear black or white. The polymers can of course also be mixed in other ratios to achieve greater or lesser opacity.

## 9. Time required for hardening/ polymerization

After processing the acrylic using the salt-and-pepper technique or doughing technique, cure the acrylic in a pressure vessel. We recommend the Polyclav as a pressure vessel for polymerization (REF 070-010-00) with the heating plate (REF 070-900-00 and accessories REF 070-015-00).

### **Polymerization conditions:**

Pressure vessel: 2.2 bar

Temperature: 40 °C – 46 °C

Time: **not less than 15 minutes (Orthocryl® EQ)!**  
**not less than 20 minutes (Orthocryl®)!**

Do not interrupt polymerization in the pressure vessel during the first 8 minutes (Orthocryl® EQ) or 10 minutes (Orthocryl®) to prevent the formation of micro bubbles caused by boiling. Only remove the finished Orthocryl® acrylic from the water in the pressure vessel wearing gloves.

**Observe the Instructions for use and safety information for the pressure vessel!**

## 10. Finishing and polishing

Orthocryl® acrylics can be finished and polished with the usual mills and polishers. Wear the prescribed protective equipment (face protection / eye protection) and work with air extraction.

## 11. Reduction of the residual monomer content and biocompatibility of Orthocryl® acrylics

When Orthocryl® acrylics are processed correctly and professionally, the residual monomer content is less than 5% by mass in accordance with DIN EN ISO 20795-2. When not wearing the appliance, the patient should place it in water to reduce the residual monomer content.

Orthocryl® acrylics have undergone scientific testing.

## 12. Cleaning and disinfection in the laboratory

Orthocryl® acrylics can be cleaned with a tooth brush and tooth paste. Please refer also to: "Oral hygiene during orthodontic treatment", (REF 989-328-10).

Orthocryl® acrylics can be disinfected with standard disinfecting agents for impression materials. (for example MD 520/ Dürre Dental, Dentavon®/Schülke+Mayr or Mucalgin®/Merz Dental). Follow the respective manufacturer's instructions.

## 13. Composition

Monomer (liquid) = methyl methacrylate "MMA," ethylene glycol dimethacrylate, 2-(2H-benzotriazol-2-yl)-p-cresol, various dyes.

Polymer (powder) = polymethyl methacrylate "PMMA" (powder), dibenzoyl peroxide, and various pigments.

Refer also to the safety data sheets which are delivered automatically to customers with every first order and which are available at any time on request!

## 14. Information on dangers and safety

Highly flammable liquid and vapor.

Monomer contains methyl methacrylate and ethylene dimethacrylate. Polymer contains dibenzoyl peroxide; benzoyl peroxide.

Causes irritation to the skin.

Can cause allergic reactions.

Can cause irritation to the respiratory tracts.

Keep away from heat/sparks/open flames/hot surfaces as well as other ignition sources. Do not smoke.

Avoid breathing dust/fume/gas/mist/vapors/spray. Wash hands thoroughly after use.

Seek medical attention in case of skin irritation or rash. If in contact with skin: Wash with plenty of water and soap.

Wear protective gloves/protective clothing/eye protection/face protection. When handling openly, use devices with local suction.

If you feel unwell, call the POISON INFORMATION CENTER or a doctor.

The product should not be used if there is a known allergic reaction to one or more of the material components.

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, inform the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident. For further information, please refer to the safety data sheet. The Orthocryl® range of products must not be used directly on patients without first being processed! The products may only be combined within the Dentaurum portfolio.

## 15. Storage and shelf life

Powder (polymer):	10 years in closed container.
Liquid (monomer):	48 months from date of manufacture in originally sealed bottle. Protect from contamination and sunlight as they may cause premature polymerization. Store in a well-ventilated place. Store cool.

## 16. Instructions for cleaning and caring for appliances

Appliances should be cleaned regularly and stored in fresh water when not in use. This procedure prevents gap corrosion or brown discoloration of the wire elements. For further tips on care, see also "Oral hygiene during orthodontic treatment." (REF 989-328-10), which is available for download at [www.dentaurum.com](http://www.dentaurum.com).

## 17. Disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and – if available for the products – the details outlined in the safety data sheets. Seal the polymer particles in the container or in tightly sealed outer packaging and dispose of in the residual waste. Avoid disposing of loose polymer particles/chips/dust in the waste.

## 18. Product range

Please refer to the Orthodontics or Dental Prosthetics Catalog for information on the Orthocryl® product range.

## 19. General information

The SSCP is available at <https://ec.europa.eu/tools/eudamed> and [www.dentaurum.com](http://www.dentaurum.com).

For more information on our products, please visit the internet at [www.dentaurum.com](http://www.dentaurum.com). You will find these Instructions for use and information on the current status at [www.dentaurum.com/ifu](http://www.dentaurum.com/ifu).

## 20. Explanation of symbols used on the label

Observe the label. Additional information can be found on the internet at [www.dentaurum.com/ifu](http://www.dentaurum.com/ifu) (Explanation of symbols used on the label REF 989-313-00).

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DENTAURUM GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany