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EN

Dear Customer

Thank you for choosing this high-quality product from Dentaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaurum.com.

Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

With reference to our quality standards

Dentaurum ensures a faultless quality of the products that we manufacture. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Three-dimensional Calipers

acc. to Prof. Dr. Dr. Korkhaus, cannot be sterilized

Modern orthodontic treatment techniques require careful control of the direction and extent of tooth movement. These calipers assist in this with a single series of measurements directly on the model.

The following data may be obtained:

- the distance between the premolars or molars from the small transverse scale
- the median separation of the central incisors from the premolar or molar connecting line with the middle scale
- the palatal height in the premolar or molar region with the small vertical pin scale.

The middle scale will give the length of the perpendicular to the connecting line of the caliper points when the calipers are opened. The scale is divided into mm and has a small cursor and a vertical plate.

Using the calipers is extremely simple. The locking screw is loosened until the caliper points are far enough apart to slip into the fissure measuring points of the first molars or first premolars, then the cursor on the middle scale is slid back until the vertical plate rests against the labial surfaces of the central incisors. If there is interest in the changes in the palate, the vertical measuring peg is pushed downwards until it touches the palate. After these adjustments are made, which only take a few seconds, the calipers are removed from the model and the various dimensions are read off directly without the need for other measuring devices. The calipers may be used and be equally effective for the mandible (on the model) as well as for determining transverse and anteroposterior tooth movement.

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Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Allemagne
Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Alemania