Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

CE 0483

equilibrium[®] mini
equilibrium[®] 2
equilibrium[®] ti

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Dear customer,

Thank you for choosing a quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these Instructions for use.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaaurum GmbH & Co. KG

Turnstr. 31 | 75228 Ispringen | Germany

2. General description

equilibrium® brackets are part of Dentaureum's standard bracket range. They are formed, milled and processed out of rod material and undergo many tried-and-tested working stages. Brackets in the equilibrium® family are genuine one-piece brackets. They are produced in one piece - body and base - making them specially biocompatible as no solders are used. equilibrium® mini and equilibrium® 2 brackets are made of stainless steel whilst equilibrium® ti is made of titanium and is therefore completely nickel-free.

3. Processing

3.1 Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding.

3.2 Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid unexpected damage to the tooth during debonding. Follow the instructions from the bracket adhesive manufacturer for conditioning the teeth.

3.3 Selecting bracket adhesive

equilibrium® brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC Ic (REF 163-111-00), ConTec Go! (REF 163-321-00) and SmartBond® (REF 163-506-00). Please follow the instructions of the bracket adhesive manufacturer.

3.4 Handling brackets

Grip the equilibrium® brackets mesio-distally using crimping tweezers (e.g. REF 025-280-00) to remove them from the packaging and handle according to the following steps. The patented laser-structured base of the equilibrium® brackets should not be touched with your fingers or contaminated in any way, as this could otherwise detrimentally affect the bond.

3.5 Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

3.6 Placing brackets - direct bonding

After applying the adhesive, place the equilibrium® bracket in the usual way onto the tooth. Place the equilibrium® bracket in the best possible position and press it onto the tooth so that excess adhesive is pushed out to the sides.

Avoid contact with antagonist teeth due to the risk of abrasion and/or premature bracket loss. If necessary, a palatal bite raise can be used.

Completely remove any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

3.7 Placing brackets - indirect bonding

After applying the adhesive, place the transfer tray with the equilibrium® brackets as usual in the mandible and/or maxilla.

Completely remove any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

3.8 Ligating archwires

Once the adhesive has cured (observe adhesive hardening times in the manufacturer's instructions for use), ligate as usual with wire or plastic ligatures.

We recommend using rematitan® LITE or Tensic® archwires at the beginning of treatment.

3.9 Debonding brackets

The excellent bonding properties of equilibrium® brackets require special attention during debonding:

For the debonding of equilibrium® brackets, we recommend using Weingart pliers (e.g. REF 003-120-00). Grasp the bracket mesio-distally by the wings and carefully remove the equilibrium® bracket from the tooth by pressing the pliers together and bending the bracket base away from the tooth. If necessary, turn the bracket briefly to the right or the left, without pulling, to remove it from the tooth.

4. Composition

Information about the composition is included in the material list, see catalog or www.dentaurum.com.

5. Safety information

There are no known risks or side effects (allergic reaction) for equilibrium® brackets. The product should not be used if there is a known allergic reaction to one or more of the material components.

6. Storage and shelf life

Store equilibrium® brackets in closed packaging.

7. Information for single use products

equilibrium® brackets are intended for single use only.

Due to the mechanical stress during treatment and debonding, neither the dimensions nor the bonding strength can be guaranteed should the brackets be recycled.

Reconditioning of equilibrium® brackets that have already been used (recycling) and their reuse on a patient is not permitted.

8. Additional information

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog.
- There are no particular precautions to be taken when disposing of equilibrium® brackets after treatment.

9. Availability

Please refer to our Catalog for information on the products available in the equilibrium® range.

10. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon Dentaurum's own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

11. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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