Gebrauchsanweisung | Instructions for use  
Mode d'emploi | Modo de empleo  
Modalità d'uso

- Informationen zu Produkten finden Sie unter [www.dentaurum.com](http://www.dentaurum.com)
- For more information on our products, please visit [www.dentaurum.com](http://www.dentaurum.com)
- Vous trouverez toutes les informations sur nos produits sur [www.dentaurum.com](http://www.dentaurum.com)
- Para más información sobre nuestros productos, visite [www.dentaurum.com](http://www.dentaurum.com)
- Informazioni sui prodotti sono disponibili nel sito [www.dentaurum.com](http://www.dentaurum.com)

Stand der Information | Date of information | Mise à jour |  
Fecha de la información | Data dell'informazione: 2023-12

Änderungen vorbehalten | Subject to modifications |  
Sous réserve de modifications | Reservado el derecho de modificación |  
Con riserva di apportare modifiche

CE 0483

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31/803-0 | Fax +49 72 31/803-295  
[www.dentaurum.com](http://www.dentaurum.com) | [info@dentaurum.com](mailto:info@dentaurum.com)

989-610-00 Printed by Dentaurum Germany 2023-12/BR1-4

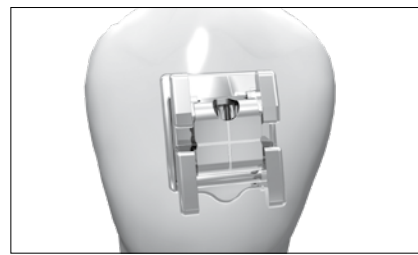
Fig. 1

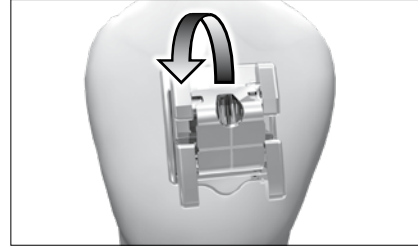
Fig. 2

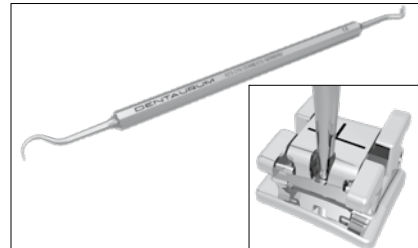
Fig. 3

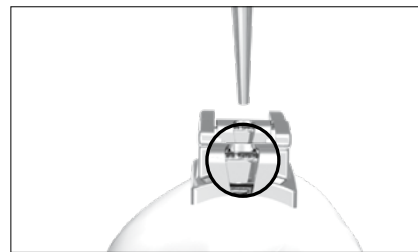
Fig. 4 a

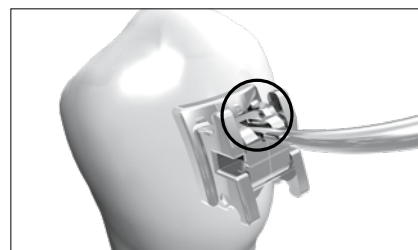
Fig. 4 b

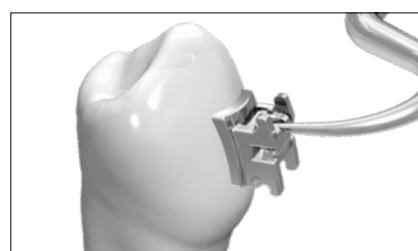
Fig. 5 a

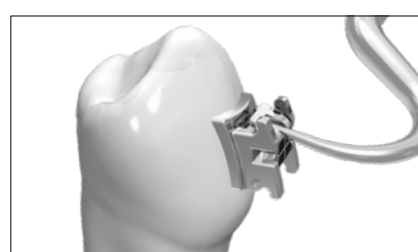
Fig. 5 b

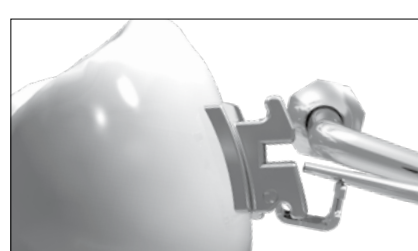
Fig. 6 a

Fig. 6 b

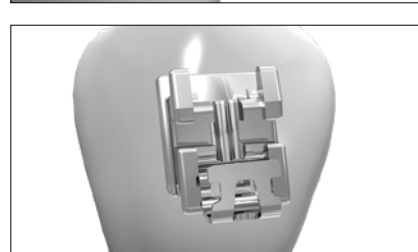
Fig. 7

Fig. 8

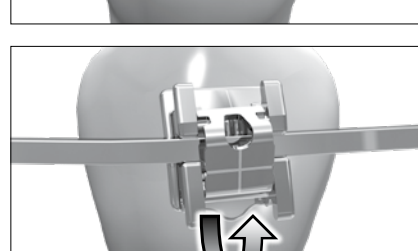
Fig. 9

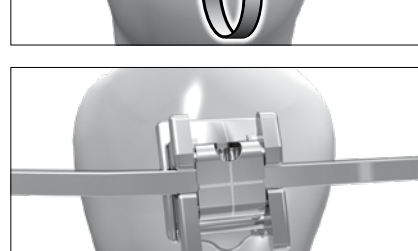
Fig. 10

DE

## Selbstligierende Brackets – discovery® sl2.0

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EN

## Self-ligating brackets – discovery® sl2.0

### Dear customer,

Thank you for choosing a quality product from Dentaurum. Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

#### 1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

#### 2. General device description

discovery® sl2.0 is a self-ligating bracket with a body manufactured by MIM and an integrated closure lid for the slot.

#### 3. Intended purpose

For the treatment of tooth misalignments as part of orthodontic treatment.

#### 4. Intended users

All products are intended for dental professionals only.

#### 5. Indications

For tooth and jaw misalignments.

#### 6. Contraindications

Stainless steel brackets must not be used in patients with a nickel allergy. Warnings are found on the respective product labels. Also see 10. Composition.

#### 7. Patient target group

All patients for whom a fixed appliance with orthodontic braces can be placed as part of orthodontic treatment.

#### 8. Processing

**8.1. Informing the patient**  
Inform the patient about the possible consequences and risks of bracket treatment.

##### 8.2. Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding.

##### 8.3. Selection of bracket adhesive

Our brackets can be bonded using all commercially available bracket adhesives.

##### 8.4. Removal and handling

Grip the discovery® sl2.0 brackets mesially-distally using crimping tweezers to remove them from the packaging. The laser-structured base of the brackets should not be touched with your fingers or contaminated in any way, as this could otherwise have a detrimental effect on the bond.

##### 8.5. Using the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

##### 8.6. Placing and aligning brackets

After applying the adhesive, place the discovery® sl2.0 bracket in the usual way onto the tooth. Laser-markings on the lid of the bracket can help orientation when placing the bracket (Fig. 1). The vertical line marks the middle of the bracket. The horizontal line is identical to the middle of the slot. The slot is not in the middle of the bracket! This should be taken into account when aligning the bracket (Fig. 4 a to Fig. 7). Remove completely any excess adhesive before it has cured.

##### 8.7. Using the closure mechanism / ligating the archwires

All lids, both in the maxilla and in the mandible, are opened from occlusal to gingival (Fig. 2). **Always use the sl-instrument (REF 023-277-00) to open the brackets (Fig. 3).**

To open the lid, the pointed end of the sl-instrument is inserted into the opening on the lid between the occlusal bracket tie wings (Fig. 4 a and Fig. 4 b). Open the lid using the sl-instrument, levering it slightly towards vestibular/gingival (Fig. 5). The lid opens. discovery® sl brackets have a mechanism (CapSafe™) which holds the lid open while the archwire is inserted. To activate the CapSafe™, press the lid, after it has opened spontaneously, using either a finger or the sl-instrument towards the gingiva until a resistance is felt (Fig. 6 a and Fig. 6 b). The lid is now fixed securely in open position (Fig. 7). Insert the archwire into the slot (Fig. 8) using the fork-shaped end of the sl-instrument if necessary. Close the lid from gingival to occlusal using either a finger or the sl-instrument (Fig. 9). The lid must be felt to click into place. This ensures the slot is closed securely (Fig. 10).

##### 8.8. Bracket debonding

Weingart pliers can be used to debond discovery® sl2.0 brackets. Grasp the bracket from mesial-distal on the tie wings. Turn the bracket briefly to the left or right, without pulling, to remove it from the tooth. The angled bracket removing pliers are particularly suitable for the posterior region as they shear the bracket off between the bracket base and the adhesive.

#### 9. Caution

Please note that the teeth should not be cleaned while the archwire is being changed as long as the bracket lids are open. This would cause distortion to the delicate lid. This would in turn destroy the function of the closure mechanism and it would no longer be possible to close the lid.

#### 10. Composition

discovery® sl2.0 brackets are made of the following materials: 1.4301, 1.4305 and 1.4310. The composition is included in the orthodontic materials list; please refer to the catalog or [www.dentaurum.com](http://www.dentaurum.com). The SSCP is available at <https://ec.europa.eu/tools/eudamed> and [www.dentaurum.com](http://www.dentaurum.com).

#### 11. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

#### 12. Storage and shelf life

Store discovery® sl2.0 brackets in the closed packaging. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

#### 13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

#### 14. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

#### 15. Product range

For the complete product range of discovery® sl2.0 brackets, please see our orthodontics catalog or [www.dentaurum.com](http://www.dentaurum.com).

#### 16. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

#### 17. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information and the Instructions for use can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

Fig. 1

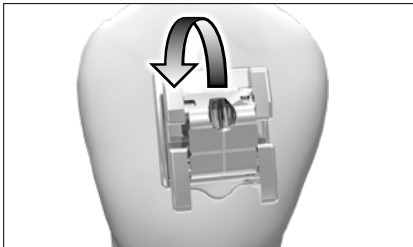
Fig. 2

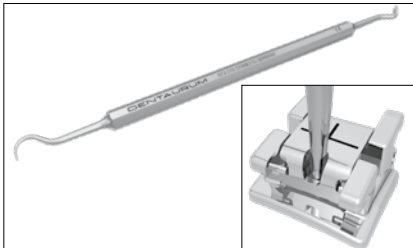
Fig. 3

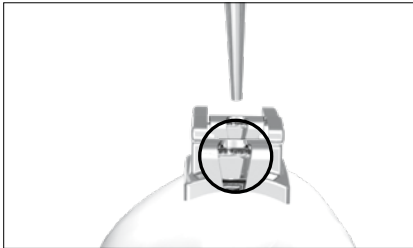
Fig. 4 a

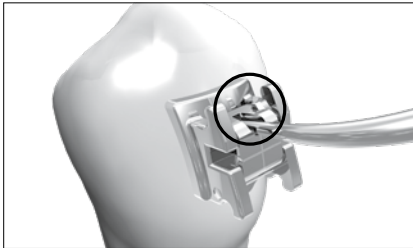
Fig. 4 b

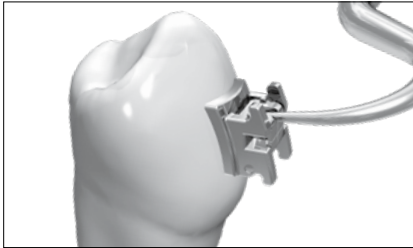
Fig. 5 a

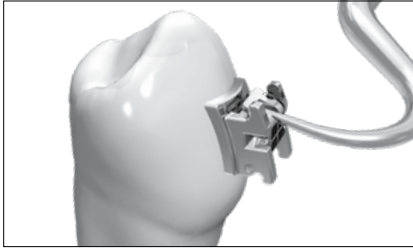
Fig. 5 b

Fig. 6 a

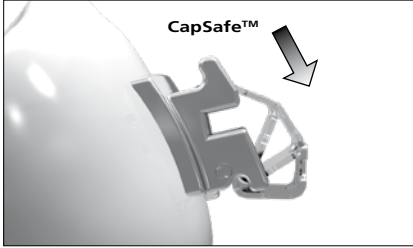
Fig. 6 b

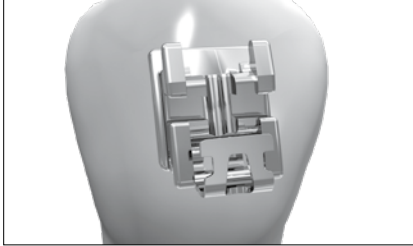
Fig. 7

Fig. 8

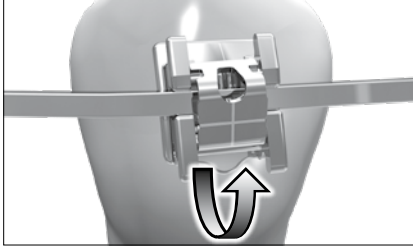
Fig. 9

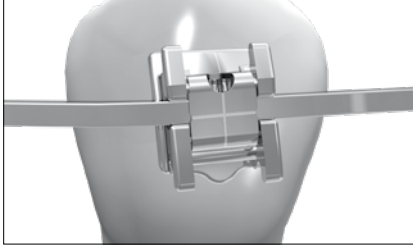
Fig. 10

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