Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

## tomas®-screwdriver

- Informationen zu Produkten finden Sie unter [www.dentaurum.com](http://www.dentaurum.com)
- For more information on our products, please visit [www.dentaurum.com](http://www.dentaurum.com)
- Vous trouverez toutes les informations sur nos produits sur [www.dentaurum.com](http://www.dentaurum.com)
- Descubra nuestros productos en [www.dentaurum.com](http://www.dentaurum.com)
- Informazioni sui prodotti sono disponibili nel sito [www.dentaurum.com](http://www.dentaurum.com)

Stand der Information | Date of information | Mise à jour |  
Fecha de la información | Data dell'informazione: 2024-02

Änderungen vorbehalten | Subject to modifications | Sous réserve  
de modifications | Reservado el derecho de modificación |  
Con riserva di apportare modifiche

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31 / 803 - 550 | Fax +49 72 31 / 803 - 295  
[www.dentaurum.com](http://www.dentaurum.com) | [info@dentaurum.com](mailto:info@dentaurum.com)

989-634-00 Printed by Dentaaurum Germany 2024-02/BR/1-13

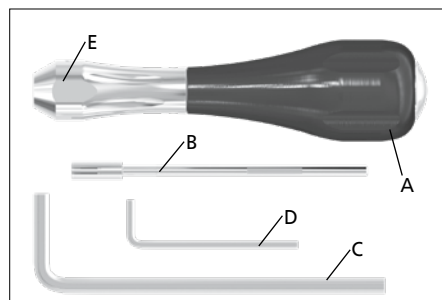
Fig. 1

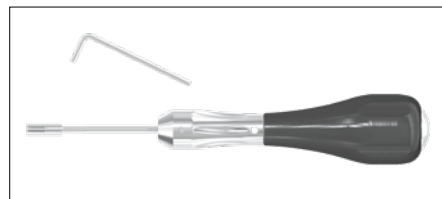
Fig. 2

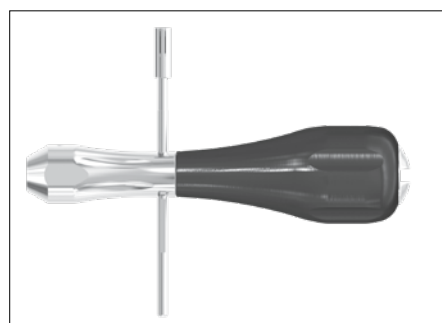
Fig. 3

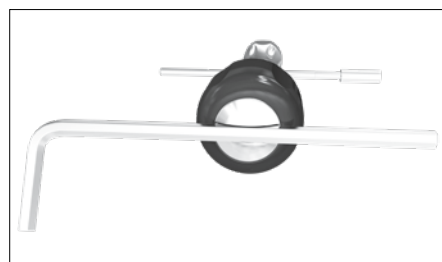
Fig. 4

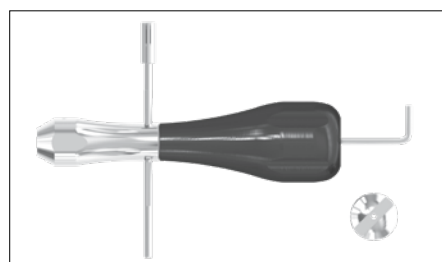
Fig. 5

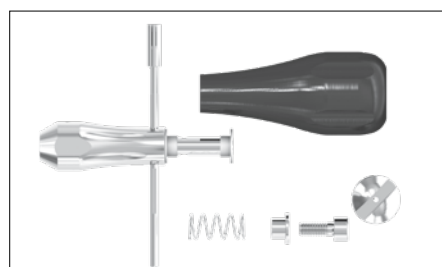
Fig. 6

## DE

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## EN

### Dear customer,

Thank you for choosing a quality product from Dentaaurum.

Please read these instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

#### 1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

#### 2. General device description

The tomas®-screwdriver (REF 302-004-10) is composed of the following parts (Fig. 1):

- A** screwdriver handle (REF 302-004-11)
- B** screwdriver tip (REF 302-004-18)
- C** screwdriver key 3.0 (REF 302-004-14)
- D** screwdriver key 1.3 (REF 302-004-16)
- E** screwdriver Allen set screw (REF 302-004-17)

#### 3. Intended purpose

The tomas®-screwdriver is intended for the manual insertion of the tomas®-pin.

#### 4. Intended users

All products are intended for dental professionals only.

#### 5. Indications

Following preparation (cleaning, disinfection and sterilization), the tomas®-screwdriver is used for the manual insertion of the tomas®-pin. The space available should be taken into consideration.

#### 6. Contraindications

The use of the tomas®-screwdriver is not indicated if there is insufficient space in the insertion area to insert the tomas®-pin in the direction required. The product may only be used if it has been correctly prepared.

#### 7. Patient target group

All patients (children and adults) for whom the tomas®-pin can be used for orthodontic treatment.

#### 8. Processing steps

The tomas®-screwdriver must be disinfected, disassembled, cleaned, dried, assembled and then sterilized before each clinical application, see 8.1 and 12.

The tomas®-screwdriver tip (Fig. 1 B) is used to pick up the tomas®-pin and insert it into the bone. Due to the design of the tomas®-screwdriver a very high torque can be attained when it is used to insert the tomas®-pin. Relevant care should therefore be taken during insertion to avoid fracture of the tomas®-pin.

#### 8.1. Disassembly and assembly

Loosen the tomas®-screwdriver Allen set screw with the tomas®-screwdriver key 1.3 and remove the tomas®-screwdriver tip (Fig. 2).

Put the tomas®-screwdriver tip through the drillhole on the tomas®-screwdriver handle (Fig. 3).

Open the cap on the tomas®-screwdriver handle (Fig. 4).

Unscrew the cylindrical screw concealed beneath it with the tomas®-screwdriver key 3.0 screwdriver (Fig. 5) and dismantle the tomas®-screwdriver (Fig. 6).

After cleaning, the tomas®-screwdriver is completely reassembled in reverse order and prepared for sterilization. See also section 11.

#### 9. Safety information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

#### 10. Storage and shelf life

If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

#### 11. Information for multiple use products

The tomas®-screwdriver is intended for multiple use. The tomas®-screwdriver must be disinfected, disassembled, cleaned, dried, assembled and then sterilized before being used again. For details, see Instructions for the preparation of pliers and instruments (REF 989-590-20).

When disposing of the product, the laws of the country in which the user and / or patient reside should be observed.

#### 12. Product range

For the product range, please see our Orthodontics Catalog.

#### 13. Quality information

Dentaaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaaurum has no influence on the processing on site.

#### 14. Explanation of symbols

⚠ Please observe the label and the Instructions for use. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

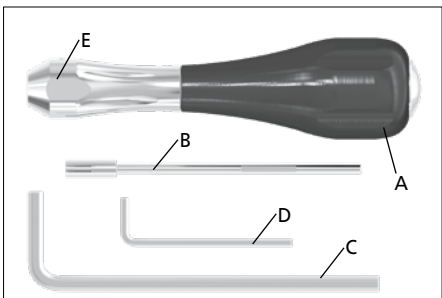
Fig. 1

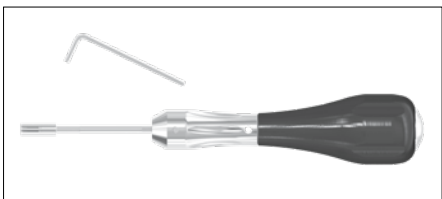
Fig. 2

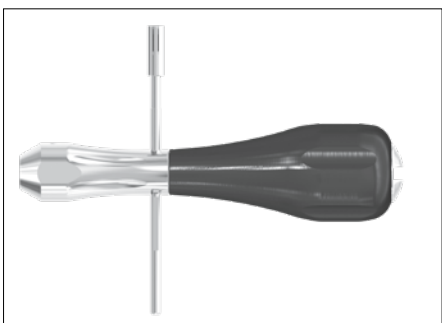
Fig. 3

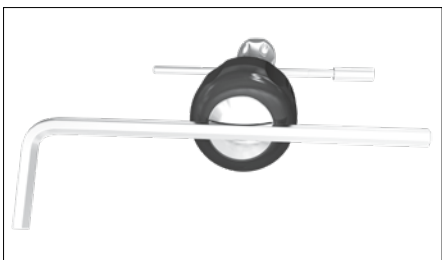
Fig. 4

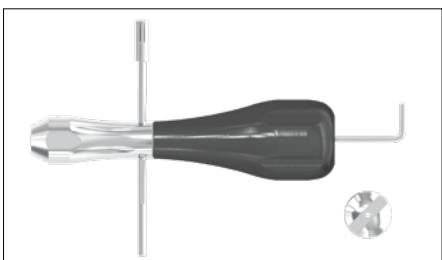
Fig. 5

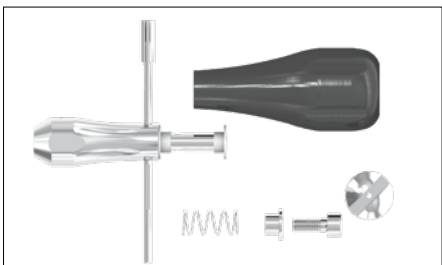
Fig. 6

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