
D
DENTAURUM
1886

tomas

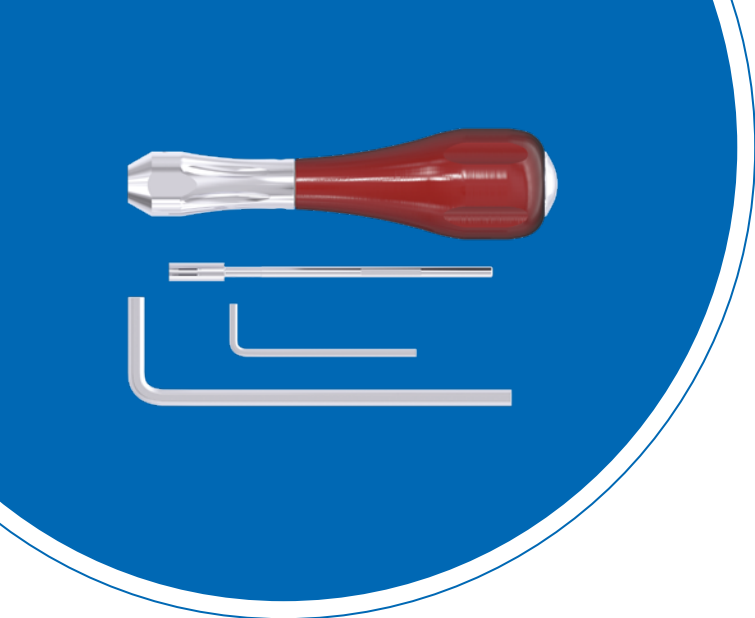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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

The tomas®-screwdriver (REF 302-004-10) is composed of the following parts (Fig. 1):

- A** screwdriver handle (REF 302-004-11)
- B** screwdriver tip (REF 302-004-18)
- C** screwdriver key 3.0 (REF 302-004-14)
- D** screwdriver key 1.3 (REF 302-004-16)
- E** screwdriver Allen set screw (REF 302-004-17)

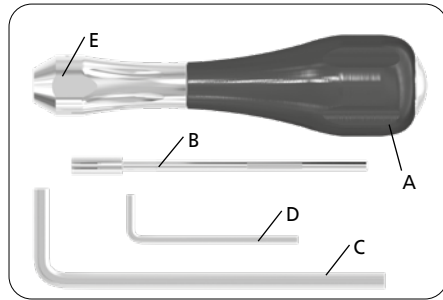
Fig. 1

3. Intended purpose

The tomas®-screwdriver is intended for the manual insertion and explantation of the tomas®-pin.

4. Intended users

All products are intended for dental professionals only.

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6. Contraindications

The contraindications of the tomas®-pin apply.

7. Patient target group

All patients (children and adults) for whom the tomas®-pin can be used for orthodontic treatment.

8. Processing steps

The tomas®-screwdriver must be disinfected, disassembled, cleaned, dried, assembled and then sterilized before each clinical application, see 8.1 and 12.

The tomas®-screwdriver tip (Fig. 1 B) is used to pick up the tomas®-pin and insert it into the bone. Due to the design of the tomas®-screwdriver a very high torque can be attained when it is used to insert the tomas®-pin. Relevant care should therefore be taken during insertion to avoid fracture of the tomas®-pin.

8.1. Disassembly and assembly

Loosen the tomas®-screwdriver Allen set screw with the tomas®-screwdriver key 1.3 and remove the tomas®-screwdriver tip (Fig. 2).

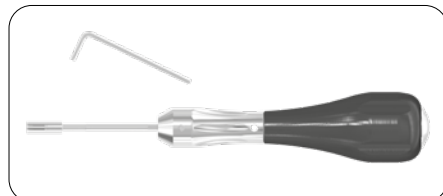
Fig. 2

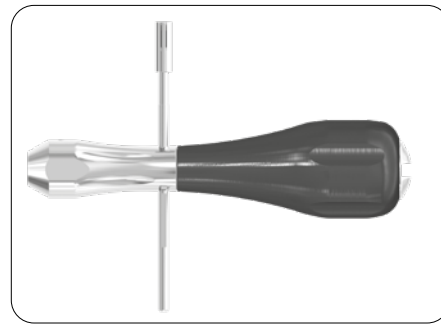
Fig. 3

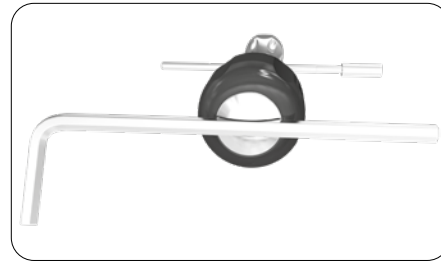
Fig. 4

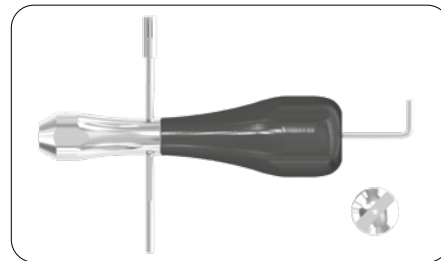
Fig. 5

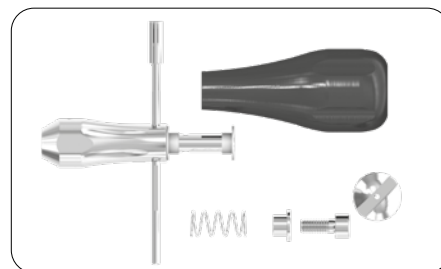
Fig. 6

Put the tomas®-screwdriver tip through the drillhole on the tomas®-screwdriver handle (Fig. 3).

Open the cap on the tomas®-screwdriver handle (Fig. 4).

Unscrew the cylindrical screw concealed beneath it with the tomas®-screwdriver key 3.0 screwdriver (Fig. 5) and dismantle the tomas®-screwdriver (Fig. 6).

After cleaning, the tomas®-screwdriver is completely reassembled in reverse order and prepared for sterilization. See also section 11.

9. Safety information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

10. Storage and shelf life

If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

11. Information for multiple use products

The tomas®-screwdriver is intended for multiple use. The tomas®-screwdriver must be disinfected, disassembled, cleaned, dried, assembled and then sterilized before being used again. For details, see Instructions for the preparation of pliers and instruments (REF 989-590-20).

When disposing of the product, the laws of the country in which the user and / or patient reside should be observed.

12. Product range

For the product range, please see our Orthodontics Catalog.

13. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

14. Explanation of symbols

Please observe the label and the Instructions for use. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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