Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

tomas®-torque ratchet

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Descubra nuestros productos en www.dentaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 05/21

Änderungen vorbehalten | Subject to modifications | Sous réserve
de modifications | Reservado el derecho de modificación |
Con riserva di apportare modifiche

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31 / 803-550 | Fax +49 72 31 / 803-295
www.dentaurum.com | info@dentaurum.com

DE

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EN

Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General Product Description

The tomas®-torque ratchet (REF 302-004-40) consists of the ratchet head, the ratchet handle and an adjusting screw.

The tomas®-torque ratchet is used together with the tomas®-applicator (REF 302-004-20, REF 302-004-70) to screw the tomas®-pin into the bone.

It is possible to lock the ratchet head in order to block the torque limitation. If the ratchet head is not locked, the torque may be adjusted from 10 to 30 Ncm variably using the adjusting screw.

3. Application

If the tomas®-pin is inserted with the tomas®-torque ratchet a maximum torque of 20 Ncm must not be exceeded – also see the instructions for use tomas®-pin.

- Set the ratchet head so that it may be bent over if pressure is applied from the side. If the ratchet head is locked, set the adjusting screw to 10 Ncm by turning a few times. The ratchet head is then removed from the ratchet handle, turned by 90° and clicked into place on the ratchet head.
- Screw the adjusting screw tighter until 20 Ncm torque is visible and corresponds with the markings on the ratchet handle.
- Click the tomas®-applicator into place on the ratchet head and subsequently place onto the hexagonal matrix in the tomas®-pin.
- The tomas®-pin is screwed into the bone until the ratchet head bends. This means the set torque has been reached. Do not screw in any further once the ratchet head is bent! There is a danger that the tomas®-pin will break.

Important!

After each use, screw the adjusting screw back to the lowest marking in order to relieve the spring!

Before the first use and after each use on a patient the tomas®-torque ratchet must be disinfected, dismantled, cleaned, dried, reassembled and sterilized, according to the „Instructions for the preparation of pliers and instruments“ (REF 989-590-20).

The tomas®-torque ratchet must be calibrated every two years by the manufacturer.

4. Dismantling

1. Remove the inserted parts from the ratchet head.
2. Completely dismantle the adjusting screw from the ratchet handle.
3. Remove the ratchet head from the ratchet handle and remove the torque spring.
4. Disinfect and clean the individual parts. Before sterilizing, re-assemble the ratchet.

5. Assembly

1. Slide the torque spring into the ratchet handle.
2. Slide the ratchet head with the threaded bar into the ratchet handle.
3. Screw the adjusting screw onto the ratchet handle up to the lowest marking.

6. Preparing the instruments

The tomas®-torque ratchet is intended for repeated use. Before reuse the tomas®-torque ratchet must be disinfected, dismantled cleaned, reassembled and sterilized. For details please see the „Instructions for the preparation of pliers and instruments“ (REF 989-590-20).

7. Additional tips

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

8. With reference to our quality standards

Dentaurum ensures a faultless quality of the products that we manufacture. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

9. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

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