tomas®-hook

CE 0483

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Instructions for use

Mode d'emploi

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D
DENTAURUM

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Instructions for use tomas®-hook

Dear Customer

Thank you for having chosen a quality product from Dentaaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaurum.com.

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2. With reference to our quality standards

Dentaaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. Instructions for use

The large opening of the tomas®-hook (Fig.1) is placed over the top of the tomas®-pin. The recommended fixation point is the lower notch between the hexagon and the neck (Fig.2). Apply pressure to the tomas®-hook until the smaller section is secured within the notch (Fig.3). It is recommended to slightly compress the large section in order to ensure a safe fixation of the tomas®-hook. Now the elastic elements (rubber chains, rubber rings, tension springs) can be attached (Fig.4-6). If necessary the tomas®-hook can be bent so that the elastic element is securely fixed.

The tomas®-hook enables a combination of treatments simultaneously from elastic elements at the head to square wires in the slot of the tomas®-pin.

4. Availability

tomas®-hook, 10 pieces, REF 302-009-10

5. Explanations of symbols used on the label

⚠ Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00)

If you have any questions, please contact the Dentaaurum Orthodontic Hotline: +49 72 31/803-550.

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Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Deutschland