tomas®-hook

**D**  
DENTAURUM

Dentaurum

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EN

### Dear customer,

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at [www.dentaurum.com](http://www.dentaurum.com), even though you may frequently use the same product.

### 1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

### 2. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

### 3. Application

The large opening of the tomas®-hook (Fig. 1) is placed over the top of the tomas®-pin. The recommended fixation point is the lower notch between the hexagon and the neck (Fig. 2). Pull the tomas®-hook until the smaller section is secured within the notch (Fig. 3). It is recommended to slightly compress the large section in order to ensure a safe fixation of the tomas®-hook. The elastic elements (rubber chains, elastics, tension springs) can then be attached (Fig. 4 - 6). If necessary the tomas®-hook can be bent so that the elastic element is securely fixed.

The tomas®-hook enables a combination of treatments simultaneously from elastic elements at the head to square wires in the slot of the tomas®-pin.

### 4. Information for single use products

The tomas®-hook is designed for single use only. Reconditioning of a tomas®-hook that has already been used (recycling) and its reuse on a patient are not permitted.

### 5. Additional information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

### 6. Delivery program

tomas®-hook, 10 pieces, REF 302-009-10

### 7. Explanation of symbols used on the label

⚠ Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

Fig. 1

Fig. 2

Fig. 3

Fig. 4

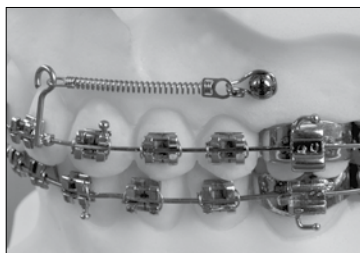
Fig. 5

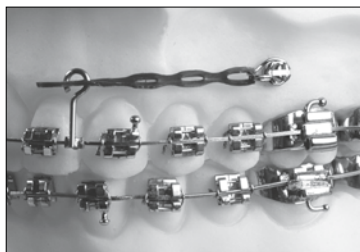
Fig. 6

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Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germania