---

**DE** Gebrauchsanweisung · Osteotome

---

**EN** Instructions for use · Osteotomes

---

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---

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---

**IT** Istruzioni d'uso · Osteotomi

---

## 1. Hersteller

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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

## 2. General device description

The osteotomes are used to manually form the available bone by condensing it in order to achieve sufficient primary stability of the tioLogic® ST / tioLogic® TWINFIT implants. They can be used to work the bone by pushing, turning or knocking. They are available in angled form with a concave tip and can be used in the anterior and posterior region of the jaw. The osteotomes are available in various diameters. A sequence of lines on the tip of the instrument indicates different heights.

The osteotomes are intended for multiple use and reprocessing.

The osteotome tray is only available in non-equipped state and is intended for the safe storage and sterilization of the osteotomes.

## 3. Intended purpose

Osteotome to displace and compress intra-oral osseous structures and to detach the sinus floor.

The osteotome tray is used for secure storage and sterilization of the osteotomes.

## 4. Intended users

The tioLogic® ST / tioLogic® TWINFIT product range is supplied exclusively to qualified personnel such as surgeons, dentists and dental technicians. It should only be used by qualified personnel who are familiar with dental implantology, including diagnosis, preoperative planning, surgical technique and prosthetic treatment.

## 5. Medical indications / contraindications

### 5.1 Indications

#### 5.1.1 Osteotomes

- Used for condensing and displacing local bone in the soft maxilla and
- mandible (bone quality 3 and 4) in order to achieve sufficient primary stability of the
- tioLogic® ST / tioLogic® TWINFIT implants
- Used for direct and indirect sinus lift

#### 5.1.2 Osteotome tray, not equipped

- Container for the secure storage of osteotomes
- Container for the sterilization of osteotomes

### 5.2 Contraindications

#### 5.2.1 Osteotomes

##### General contraindications:

Insufficient bone or tissue available and/or inadequate bone quality, local root remnants, bone disease and wound healing disorders, local infection of implant site, serious therapy-resistant functional disorders, uncontrolled diabetes mellitus, long-term immunosuppressive therapy, connective tissue disease, blood disorders (e.g. leukemia, hemophilia), intra-oral infection or malignancies, uncontrolled parafunctional habits, non-treatable occlusal or articulation conditions, serious psychological illnesses, xerostomia and titanium allergies.

##### Local / personal contraindications:

Incomplete growth of jaw, unfavorable anatomical bone situations, previously irradiated bone, temporomandibular joint disorders and treatable pathologic diseases to the jaw, severe systemic diseases, reduced immune defense and leukocyte disfunctions leading to increased risk of infection, endocrine disorders, medical anticoagulation / hemorrhagic diathesis, arteriosclerosis and stroke, hypertension, heart attack, illnesses with periodic use of steroids, hepatitis, diabetes mellitus and pregnancy, metabolic bone disorders, all illnesses affecting bone regeneration or blood microcirculation, rheumatic diseases and drug/alcohol/tobacco addiction, lack of patient compliance, inadequate oral hygiene, periodontitis, bruxism, parafunctional habits and oral mucosal abnormalities.

#### 5.2.2 Osteotome tray, not equipped

- Used as cleaning container
- Sterilization and storage without approved sterilization film

## 6. Patient target group

All skeletally mature patients undergoing implant treatment. Please consider the indications / contraindications.

## 7. Information on use, processing, fitting and insertion

Prior to use, dental professional should ensure that they have carefully read and understood the tioLogic® ST / tioLogic® TWINFIT Instructions for use.

We strongly recommend that, prior to using Dentaurum products, dental professionals attend a Dentaurum training course to familiarize themselves with the Dentaurum products.

Carry out thorough radiological, psychological and physical examinations before implantation and inform patients of the process in detail. Dentaurum recommends full clinical, radiological, photographic and statistical documentation.

### Precautions:

Before surgical implantation, it is important to carry out a careful patient anamnesis and, if required, a third-party anamnesis by the patient's doctor (GP/PCP) in order to clarify if 1) implantation may be difficult due to an anatomical situation, 2) there is the risk of a serious surgical problem or a general risk, 3) wound healing/osseointegration may be difficult, or 4) good hygiene and care of the implant, abutment and restoration may worsen. Do not fit dental implants if any of the above cases or a combination hereof is severe or uncontrollable.

Prosthetic restoration of the implant must not be performed until the soft tissue has healed without irritation. Furthermore, a radiographic check-up should be performed before the prosthetic restoration.

### Important:

**The implantologist should ensure the products cannot be aspirated during intra-oral use.**

### 7.1 Osteotomes

The osteotomes are available in angled form with a concave tip and can be used in the anterior and posterior region of the jaw.

Osteotomes are used to condense or displace local bone in the soft maxilla and mandible (bone quality 3 and 4) in order to achieve sufficient primary stability of tioLogic® ST / tioLogic® TWINFIT implants.

They can be used to work the bone by pushing, turning or knocking. They are used to manually form the available bone by condensing it laterally and apically.

Depending on the bone situation, osteotomes are used in ascending order after the marking drill and the  $\varnothing$  2.0 mm depth drill. The osteotomes are available in various diameters

and are matched to the implant diameters. For clear identification, the osteotomes are color-coded and also laser-marked. A sequence of lines on the tip of the instrument indicates different heights (3.0/5.0/7.0/9.0/11.0/13.0/15.0 mm).

REF	Article	Color coding	Implant diameter
350-170-00	Osteotome $\varnothing$ 2.0 – 2.5 mm	White	-
350-172-00	Osteotome $\varnothing$ 2.6 – 3.1 mm	Braun	3.3 mm
350-174-00	Osteotome $\varnothing$ 3.0 – 3.5 mm	Yellow	3.7 mm
350-176-00	Osteotome $\varnothing$ 3.5 – 4.0 mm	Red	4.2 mm
350-178-00	Osteotome $\varnothing$ 4.0 – 4.6 mm	Blue	4.8 mm
350-180-00	Osteotome $\varnothing$ 4.8 – 5.3 mm	Green	5.5 mm

### IMPORTANT:

When using the osteotomes, proper safety clearance to the remaining teeth and vital structures must be maintained. Drilling or working the bone with osteotomes beyond the planned depth due to an incorrect calculation of the drill depth in relation to the radiograph can lead to permanent nerve damage or to the damage of other vital structures.

A 1.5 mm safety clearance to the nervus mandibularis or nervus alveolaris inferior must be maintained. Smoothing the bone edge will reduce the available bone height.

The osteotomes can also be used to elevate the sinus floor.

When performing a sinus lift with osteotomes, it is important to ensure that a minimum residual bone height of 5.0 mm is available.

The number of osteotomes used to prepare the implant site depends on the following factors:

- Width of the alveolar ridge
- Implant diameter
- Thickness and structure of the cortical bone

## 7.2 Osteotome tray, not equipped

The osteotome tray is used for the safe storage of the instruments and can be sterilized together with these items.

## 8. Technical specification

### Composition

Product	Composition
Osteotomes, angled, concave	1.4021
Osteotome tray	Radel® (PPSU) / silicone

Information about the composition is included in the material list for our implantology products (REF 989-801-05), or see [www.dentaurum.com/ifu](http://www.dentaurum.com/ifu). The SSCP is available at <https://ec.europa.eu/tools/eudamed> and [www.dentaurum.com](http://www.dentaurum.com).

## 9. Adverse reactions and residual risks

Bone compaction may cause unintentional perforation of the sinus cavity and injury to nerves.

When performing a sinus lift (internal/indirect) using the osteotome technique, the following complications and adverse reactions may occur:

- Perforation of the Schneiderian membrane, postoperative paroxysmal positional vertigo, nose bleeds, postoperative infections, nasal congestion, extensive hematoma.
- An augmentation height of more than 4.0 mm may result in a higher complication rate.

## 10. Safety information and precautions

- Do not use the product if there is a known allergic reaction to one or more of the material components.
- Due to the small size, the article could be swallowed or aspirated. Aspiration could lead to difficulty in breathing or death due to asphyxiation. Therefore, secure all articles used intra-orally against swallowing and/or aspiration.
- Different types of alloy in the oral cavity can lead to galvanic reactions.
- If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.
- Drills and instruments are intended for defined implant systems and implant diameters. Using these tools with other implant systems and/or implant diameters can cause damage to the tissue or lead to unsatisfactory aesthetics results.
- Use of third-party components can negatively impact the function and safety. In such cases, Dentaurum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- Observe the indications for the individual products. Consider all indications that are not listed explicitly to be contraindications.
- Disinfection, cleaning and sterilizing should not be done in the original packaging.
- Products must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.
- Technical advice (oral and written) is based on the scientific and technical knowledge available at the time the product is put on the market. It does not release the dental professional from the responsibility of personally checking the suitability of the products for the intended indications and applications. Advice is only given as a non-binding recommendation and cannot be assumed to provide any form of assurance, liability or guarantee.

### Severe incidents:

Report all severe incidents arising from the use of the product to the manufacturer and the competent authority in the country in which the dental professional and/or the patient are resident.

Severe incidents are identified as malfunctions and a drop in performance or deterioration of the properties of a medical device which resulted or may have resulted in a direct or indirect temporary or permanent deterioration in a patient's state of health, in the death of a patient, a dental professional or other persons or as posing a serious danger to public health.

## 11. Storage and shelf life

The osteotomes should always be stored in a dry, dark place at room temperature in the original packaging. If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

## 12. Disposal

Adhere strictly to the rules that apply for your region and the details outlined in the safety data sheets, if available. Products to be disposed of should be treated as surgical waste and be decontaminated in compliance with the relevant regulations for medical waste disposal.

## 13. Information for multiple use products

The osteotomes and the osteotome tray are supplied non-sterile and are intended for multiple use. The product must be disinfected, cleaned and sterilized before being used initially and before being used again. Osteotomes must be replaced if they cannot be clearly identified or if their function is impaired due, for example, to poor readability of the markings or labels.

## 14. Information on disinfection / cleaning / sterilizing

The osteotomes and the osteotome tray are delivered non-sterile. They are intended for multiple use. They must be cleaned, disinfected and sterilized before and after being used on a patient (e.g. transfer to dental laboratory). Only sterilized components may be used if bleeding occurs. To guarantee adequate cleaning to conform with Robert Koch Institute regulations, the osteotomes must be cleaned separately and not in the osteotome tray. The osteotome tray serves only to hold the osteotome during sterilization and then to store the osteotome. For sterilization, the loaded osteotome tray is packed in disposable sterilization packaging dedicated to the sterilization process in accordance with ISO 11607-1/2.

## 15. Product overview and scope of delivery

Please refer to the tiologic® ST product catalog for the tiologic® ST product range.

Please refer to the tiologic® TWINFIT product catalog for the tiologic® TWINFIT product range.

## 16. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

## 17. Explanation of symbols

Please refer to the label. See chapter "Additional information".

## 18. Additional information

Additional information can be found at [www.dentaurum.com/ifu](http://www.dentaurum.com/ifu)

Instructions for use	REF
Preparation instructions for the implant system components	989-801-07
Explanation of symbols used on the label	989-313-00

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