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Dear customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the Internet at www.dentaureum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General

2.1. Handling the vessel

These Instructions for use will help you to work safely and efficiently with the pressure vessel. The Instructions are an integral part of the Polyclav® pressure vessel; store them close-by, accessible to personnel at all times. Operators of the vessel are obliged to read through the Instructions for use and understand them before working with the vessel. To work safely with the vessel, it is imperative to adhere to the safety instructions and handling instructions included in the Instructions for use. Local accident prevention regulations and general safety regulations for the working area of the vessel are also valid.

2.2. Limitation of liability

The manufacturer accepts no liability in the following cases:

- Instructions for use are not followed
- Use contrary to intended use
- Use by untrained staff
- Unauthorized changes to construction
- Technical changes
- Use of non-authorized spare parts

We reserve the right to make technical alterations pertaining to improvements or developments of the product.

2.3. Copyright

These Instructions for use are copyright protected. Written approval from the manufacturer is required if you wish to hand these Instructions for use to a third party in whatever form – including in parts – or if you wish to use or pass on content thereof, unless this is for internal use only.

2.4. Spare parts

Using the wrong or defective spare parts may impair the safety of the vessel or lead to damage, malfunction or total failure.

3. Safety instructions

This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

This symbol indicates important safety instructions. Disregarding this warning may cause damage to the unit and/or other property.

3.1. Operator's responsibility

The Polyclav® pressure vessel is for commercial use only. The operator of the Polyclav® pressure vessel is therefore subject to the statutory obligations for occupational health and safety. The operator must ensure that all employees who use the Polyclav® pressure vessel have read and understood the Instructions for use. The Polyclav® pressure vessel may only be used by trained personnel.

3.2. Intended use

The Polyclav® pressure vessel is designed and constructed exclusively for polymerization of cold-curing polymers.

The Polyclav® pressure vessel is made of stainless steel and is suitable for polymerizing cold-curing polymers with homogeneous and non-porous results. It is possible to maintain the required temperature in the Polyclav® pressure vessel by using our heating plate (REF 070-900-00).

The Polyclav® pressure vessel fulfils the EU Pressure Equipment Directive 97/23/EC and the German Pressure Device Regulations (Druckgeräteverordnung, 14. GSGV), category I (Annex no. 1).

3.3. Safety instructions

Please take careful note of the safety instructions and warnings listed in the following sections of these Instructions for use in order to reduce health hazards and to avoid dangerous situations.

Never use the Polyclav® pressure vessel to boil dentures or similar appliances.

Never heat up to more than 50 °C / 122 °F.

No liability will be accepted in the case of damage caused by an excessive workload.

Do not leave the Polyclav® pressure vessel unattended during operation.

It is particularly important to ensure that the lid is correctly positioned and closed in accordance with the Instructions for use.

The Polyclav® pressure vessel is correctly closed when both handles (vessel handle and lid handle) are aligned exactly. Only then may the vessel be pressurized.

Do NOT use gas or unregulated heating units to heat up the Polyclav® pressure vessel – use only adjustable heating units, such as heating plate (REF 070-900-00).

Make sure that the outlet valve ❸ is open before opening the Polyclav® pressure vessel (lever is upright) and that the pressure has reduced completely (manometer ❷ needle is at "0").

The operating pressure should not exceed 2.2 bar. The safety valve ❹ opens automatically at 2.5 bar pressure.

The safety valve ❹ must be checked regularly (at least once a month). Increase the pressure in the Polyclav® pressure vessel. The safety valve must open at 2.5 bar. If this is not the case, replace the safety valve. Limescale can impair the function of the safety valve.

For safety reasons, we recommend that your Polyclav® pressure vessel is checked over in our works every two years (depending on extension of use).

Please note: If repairs at our works are necessary, please always send us the complete Polyclav® pressure vessel and never only the lid or the vessel.

4. Description and usage

4.1. Principle

The advantage of this method is that the cold-curing polymer is guaranteed to harden bubble-free and without pores since the Polyclav® pressure vessel works at a pressure of 1.8 – 2.2 bar, distributing the reaction heat during polymerization and increasing the boiling point of the monomer. The warm water in the vessel transfers pressure to the cold-curing polymer and hardens the acrylic.

4.2. Operation

4.2.1. Positioning the lid

Place the lid onto the Polyclav® pressure vessel (see image) and use slight pressure to turn the lid to the left until the handle of the lid ⑤ is on level with the handle of the vessel ⑥. The Polyclav® pressure vessel will then lock automatically.

4.2.2. Pressure increase in the Polyclav® pressure vessel

Remove the closure cap of the inlet valve before using the Polyclav® pressure vessel. Pressurize the closed and locked Polyclav® pressure vessel to 1.8 – 2.2 bar (within green marking, see image) using the pressure hose (accessory, REF 070-015-00):

- Should the needle of the pressure gauge go beyond the area marked green, open the outlet valve ③ briefly to release excess pressure.
- Should the pressure exceed 2.5 bar, the safety valve will automatically open, reducing the pressure to approx. 2.2 bar.

4.2.3. Relieving pressure and opening the Polyclav® pressure vessel

Lift the lever of the outlet valve ③ to an upright position to release pressure. Pressure is released. The manometer ② must indicate that the pressure level is zero (0 bar).

Then push the sliding switch ④ backwards in the direction of the arrow as far as it will go and, at the same time, turn the lid handle ⑤ anti-clockwise. Remove the lid from the pressure vessel.

1 Inlet valve (REF 070-011-00)

The closure cap is fitted with a feature to check the position of the valve. The inlet valve corresponds to the same norm as for car tires.

2 Manometer

The manometer displays an area that is marked green corresponding to 1.8 – 2.2 bar operating pressure.

3 Safety valve

This comes into operation if the vessel has been filled to a pressure exceeding the red mark at 2.2 bar. It releases pressure at max. 2.5 bar.

4 Sliding switch

The switch is used to lock the lid. It is an additional safety feature to prevent the vessel from opening whilst under pressure. When the mechanical lid lock is released, a release valve opens simultaneously, allowing pressure compensation.

5 Lid handle

6 Vessel handle

7 Grab handle

8 Outlet valve

9 Open / Close symbol (O/C)

10 Lid sealing gasket
(no illustration)

4.3. Notes on hardening cold-curing polymers in the Polyclav® pressure vessel

- Before loading the Polyclav® pressure vessel, make sure the plaster models have been saturated with water. If the plaster model is dry, the formation of bubbles in the acrylic will be unavoidable.
- Fill the Polyclav® pressure vessel with water up to approx. 3 – 5 cm below the brim (temperature approx. 40 – 46 °C / 104 – 114 °F).
- Use a compressor or compressed air (max. 8.0 bar) to achieve an operating pressure of 1.8 – 2.2 bar. The manometer monitors the pressure level.
- The Polyclav® pressure vessel remains under pressure until the acrylic is completely hardened.
- Observe the Instructions for use for the acrylic used.
- The Polyclav® pressure vessel can be attached via the pressure hose (accessory, REF 070-015-00) to any compressor in a dental office or laboratory, provided the compressor operates with more than 3 bar.

5. Maintenance

- We recommend you clean the inside of the lid once a week to remove acrylic and limescale. Any standard descaling agent can be used to remove limescale.
- Pointed or corrosive cleaning materials or sharp, pointed instruments should not be used in order to avoid damaging the surface.
- Please note that sharp-edged objects such as fixators, articulators etc. may damage the surface.
- Heating the Polyclav® pressure vessel with no water inside may damage mechanical components and gaskets.
- Please maintain the lid gasket clean and free of grease. If necessary, sprinkle with talcum powder.

6. Technical data

| | |
|---|---------------------------------------|
| Inner height | 110 mm, with valve 200 mm |
| Inner ø max. | 220 mm |
| Weight | approx. 2.2 kg |
| Material | Stainless steel, induction-compatible |
| Contents | 4.5 l |
| Operating pressure | 1.8 – 2.2 bar |
| Maximum allowable working pressure | 2.5 bar |
| Maximum pressure in compressed air network | 8.0 bar |
| Maximum operating temperature | 50 °C / 122 °F |
| Pressure vessel | Category I acc. to Directive 97/23 EG |

7. Packaging

The Polyclav® pressure vessel is protected against damage during transport. Should you notice any damage which may have occurred in transit, notify immediately your dealer or freight carrier.

Please keep the packaging material in case the vessel needs to be returned to the manufacturer.

8. Components supplied

Polyclav® pressure vessel (REF 070-010-00), without pressure hose

9. Accessory

| Description | REF | Quantity |
|----------------------|------------|-----------------|
| Pressure hose, 1.5 m | 070-015-00 | 1 piece |

10. Disposal

Disposal regulations of the country of operation apply when a Dentaaurum product is taken out of operation once and for all. Dentaaurum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

Note:

This symbol indicates that the products should not be disposed of together with household garbage. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available from Dentaaurum or the dental trade.

11. Information in accordance with Art. 33(1) of EU regulation for chemicals

Please be informed that parts of the Polyclav® pressure vessel (REF 070-010-00) contain SVHC with a mass concentration of > 0.1 %. The information contained herein is based on our knowledge at the date of publication.

| | |
|--------------------------|--------------------------|
| Material | Pb |
| EINECS | 231-100-4 |
| LIST | SVHC Candidate List ECHA |
| Date of inclusion | 27.06.2018 |

12. Quality

Dentaaurum ensures faultless quality of its products. These recommendations are based upon Dentaaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaaurum has no influence on the processing on site.

Declaration of Conformity

DENTAURUM GmbH & Co. KG
Turnstr. 31
75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

| | |
|----------------------|---------------------------------------|
| Description of unit: | Polyclav (REF 070-010-00) |
| Type of unit: | Pressure equipment for polymerisation |
| Safety features: | Safety valve, manometer, outlet valve |
| EC guidelines: | 97/23/EC Pressure Equipment Directive |

The identification plate carries the following information:

| | |
|---|-------------------------|
| Manufacturer: | Dentaurum GmbH & Co. KG |
| Capacity: | 4,5 liters |
| Maximum pressure P_{max} : | 2,5 bar |
| Maximum operating temperature T_{max} : | 50°C / 122°F |

Date and manufacturers signature: 12.11.2019
Signatory:

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