Step-by-Step.

Befestigung von Attachments.

1

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1

» Prepare teeth.

Clean the teeth with oil-free pumice powder or fluoride-free paste. Rinse with water and manage moisture using air syringe and cotton wool rolls. Apply phosphoric acid etching gel according to the manufacturer's instructions on the tooth surface and etch. If the dental enamel has been fluoridated, the etching process may take a little longer.

Rinse thoroughly with water, ensure the etching gel is removed completely and isolate again with cotton wool rolls. Dry thoroughly.

2

» Prepare composite.

Prepare the light-curing composite according to the manufacturer's instructions.

3

» Fill attachment guide.

Remove the transfer tray from its packaging and fill with light-curing composite.

Important: Completely fill the attachments with composite; avoid excess.

4

» Fit attachment guide.

Place the transfer tray carefully into position and press firmly. Avoid excess.

5

» Light-cure attachments and remove attachment guide.

Light-cure each attachment individually according to the manufacturer's instructions. Then remove the transfer tray and any excess carefully.

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