tiologic®ST

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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

Prosthetic abutments and components replace missing teeth, restore chewing function and thus improve the quality of life. The implants of the tioLogic® implant family are used to fixate corresponding abutment components, depending on the indication. The abutment components serve to fabricate prosthetic meso-constructions, which are then restored with a prosthetic superstructure.

Abutment components					
Series of abutments*	S		M		L
Implant diameters in mm	3.3	3.7	4.2	4.8	5.5

*all of the abutment components listed in these Instructions for use are available for the S, M and L series of abutments.

The tioLogic® ST bridge abutments are used to fabricate bridges and bar restorations that can be occlusally screw-retained. The bridge abutments are marked according to the S, M or L series of abutments.

The bridge abutments are manufactured in one piece and provided with an apical thread. The bridge abutments should not be ground or shortened.

Bridge abutment			
Series of abutments	S	M	L
GH in mm	1.0 / 2.5 / 4.0		

The tioLogic® ST bridge CAD/CAM scan cap titanium is designed for digital capture of the prosthetic platform of the tioLogic® ST bridge abutments intra-orally or of the tioLogic® ST bridge laboratory implants in the model. The tioLogic® ST bridge CAD/CAM scan caps titanium are marked with a 2 for bridge and are supplied together with an AniTite screw L 6.0 mm. The tioLogic® ST bridge CAD/CAM scan caps titanium are reusable. The AniTite screw (L 6.0 mm) is intended for single use and is marked on the blister packaging with .

tioLogic® ST bridge impression components with a connector geometry for bridge abutment shoulder are available for the open and closed impression-taking methods. The tioLogic® ST bridge impression post, open, is supplied together with a screw for open bridge impression posts. The tioLogic® ST bridge impression post, closed, includes a tioLogic® ST bridge impression cap and an AniTite screw (L 6.0 mm).

tioLogic® ST bridge laboratory implants for printed and cast models are used to create working casts. They accurately reflect the position of the tioLogic® ST bridge abutments. The bridge laboratory implants are marked with a 2 for bridge and are provided together with a counter screw for model fixation. Processing and use take place exclusively in the (practice) laboratory or in the manufacturing center and not on the patient. This is not a medical device.

tioLogic® ST bridge cap acrylic is designed for fabricating bridge and bar frameworks that are waxed up with modeling wax or acrylic, or are ground with a milling cutter for acrylic to achieve the final shape. The wax-up is then transferred to a framework using the casting technique. Processing and use take place exclusively in the (practice) laboratory or in the manufacturing center and not on the patient. The bridge cap acrylic is supplied with an AniTite screw (L 6.0 mm). This is not a medical device.

tioLogic® ST bridge cap titanium is used for fabricating tension-free bonding in prosthetic restorations. In difficult spatial conditions, the bridge caps titanium can be ground and shortened to the height of the upper edge of the screw for bonding. The bridge cap titanium is supplied with an AniTite screw (L 6.0 mm).

tioLogic® ST bridge closure screw is intended for closing the bridge abutment. The closure screws are screwed onto the bridge abutment to ensure that no liquids penetrate the bridge abutment during healing and gingiva forming and that the connector geometry to the bridge caps (tertiary structure) is not damaged.

The tioLogic® ST implant system contains specially coordinated abutments and accessories, including instruments, for the insertion and prosthetic restoration of the implants.

3. Intended purpose

Abutments are intended to connect the implants with the prosthetic restoration.

tioLogic® ST bridge CAD/CAM scan cap titanium is intended for taking digital impressions of the intra-oral situation and bridge abutment position in the mouth or on a working cast and then transferring it to a working cast or to the CAD software.

tioLogic® ST bridge impression components are designed to take an impression of the intra-oral bridge abutment position three-dimensionally and transfer it to a working cast

tioLogic® ST bridge laboratory implants for printed and cast models are designed to represent the position of the bridge abutment (axial inclination and height) in a working cast.

tioLogic® ST bridge cap acrylic is intended to connect the implants with the prosthetic restoration.

Closure screws are intended to close the inserted implant or abutment during the healing period to prevent penetration of saliva.

4. Intended users

The tioLogic® ST product range is supplied exclusively to qualified personnel such as surgeons, dentists and dental technicians. It should only be used by qualified personnel who are familiar with dental implantology, including diagnosis, preoperative planning, surgical technique and prosthetic treatment.

5. Medical indications / contraindications

5.1 Indications

5.1.1 tioLogic® ST bridge abutments

- For fabricating splinted bridges and bar restorations as connecting pieces between implants and prosthetic restorations (cemented, individually screw-retained)
- combination with other anchorage elements
- at least 6 implants in the maxilla and 4 implants in the mandible

5.1.2 tioLogic® ST bridge CAD/CAM scan cap, titanium

- Localization of the exact implant position in the intra-oral scan
- Localization of the exact implant position in the model scan

5.1.3 tioLogic® ST bridge impression components

- Intra-oral impression-taking on bridge abutments in partially or fully edentulous jaws

5.1.4 tioLogic® ST bridge laboratory implants for printed and cast models

- precise reproduction of positions when printing models
- precise reproduction of positions for cast models (dental stone)

5.1.5 tioLogic® ST bridge cap acrylic

- For use on bridge abutments
- Wax-up for the fabrication of bridge or bar frameworks

5.1.6 tioLogic® ST bridge cap titanium

- For use on bridge abutments
- Bonding in provisional restorations or in bar frameworks

5.1.7 tioLogic® ST bridge closure screw

- Abutment closure for healing and gingiva forming
- Protection of the connector geometry to the tertiary structure during healing and gingiva forming

5.2 Contraindications

- In patients with a known intolerance to individual components.
- The contact surface between the implant and abutment must not be blasted or otherwise mechanically processed
- Insufficient oral hygiene
- Bruxism

5.2.1 tioLogic® ST bridge abutments

- Use as single tooth restoration
- Restorations with mixed retention (tooth/bridge abutment)
- Grinding or shortening of bridge abutments
- For biomechanical reasons, we recommend that the following angulations of the implants are not exceeded:

Article	Angulation compensation
Bridge abutment	-20° - 20°

5.2.2 tioLogic® ST bridge CAD/CAM scan cap titanium

- Use as restoration

5.2.3 tioLogic® ST bridge impression components

- Use as restoration
- Use of the supplied screws on other prosthetic components

5.2.4 tioLogic® ST bridge laboratory implants for printed and cast models

- Use of laboratory implants with defective or damaged connector geometry
- Use on patients

5.2.5 tioLogic® ST bridge cap acrylic

- Use as single tooth restoration
- Use on patients
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bridge abutments
- Use as direct restoration (without cast)

5.2.6 tioLogic® ST bridge cap titanium

- Use as single tooth restoration
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bridge abutments

5.2.7 tioLogic® ST bridge closure screw

- Use of other abutments than on bridge abutments
- Use as restoration

6. Patient target group

All skeletally mature patients undergoing implant treatment. Please consider the indications / contraindications.

7. Information on use, processing, fitting and insertion

Prior to use, the dental professional ensures that they have carefully read and understood the tioLogic® ST Instructions for use.

We strongly recommend that, prior to using Dentaureum products, dental professionals attend a Dentaureum training course to familiarize themselves with the Dentaureum products.

Carry out thorough radiological, psychological and physical examinations before implantation and inform patients of the process in detail. Dentaureum recommends full clinical, radiological, photographic and statistical documentation.

The tioLogic® ST implant system bridge abutments and components can be documented, e.g. in the patient file or PatientPass, using the additional labels.

Precautions:

Before surgical implantation, it is important to carry out a careful patient anamnesis and, if required, a third-party anamnesis by the patient's doctor (GP/PCP) in order to clarify if 1) implantation may be difficult due to an anatomical situation, 2) there is the risk of a serious surgical problem or a general risk, 3) wound healing/osseointegration may be difficult, or 4) good hygiene and care of the implant, abutment and restoration may worsen. Do not fit dental implants if any of the above cases or a combination hereof is severe or uncontrollable.

Prosthetic restoration of the implant must not be performed until the soft tissue has healed without irritation. Furthermore, a radiographic check-up should be performed before the prosthetic restoration.

Important:

The implantologist should ensure the products cannot be aspirated during intra-oral use.

The CAD/CAM data sets for 3shape™, dental wings and exocad can be downloaded free of charge from www.dentaureum.de/cadcam. The data sets include all the information needed for all types of restorations as a complete package, i.e. for printed implant models, hybrid abutments, one-piece abutments, as well as bridges and bars.

7.1 tioLogic® ST bridge abutments

A prerequisite for successful use of the bridge system is the most accurate possible angular position of the implant. The more accurate these angles can be maintained, the easier the prosthetic treatment. The insertion direction and the alignment of the abutments should be parallel to one another.

In fully edentulous cases, a minimum of four tioLogic® ST implants \varnothing 3.3 mm must be inserted in the mandible and a minimum of six in the maxilla, each with a splinted bar restoration without extension.

Restorations with bridge abutments on tioLogic® ST implants require a minimum length of 11.0 mm.

When handling the bridge abutments, ensure that the occlusal screw is not damaged by tweezers etc.

To ensure that the implants are reliably placed at this angle, we recommend the use of navigated implant placement with the pOosition for tioLogic® system.

Bridge abutments are available for the S, M and L series of abutments in a straight shape. They are available in the gingival heights 1.0 mm, 2.5 mm and 4.0 mm.


The bridge caps and posts fit on all three series of abutments (S-M-L), as the seating surface of all bridge abutments (\varnothing 4.1 mm), the angle of convergence (20° per side) and the cone height (2.1 mm) are identical. They are secured in position on the bridge abutments using the AniTite screw L 6.0 mm supplied

For biomechanical reasons we recommend that the following angulations of the implants are not exceeded:

Article	Angulation compensation
Bridge abutment	-20° - 20°

Using the tioLogic® ST insertion key bar/bridge/AngleFix – ratchet, the bridge abutments are screwed into the tioLogic® ST implant.

Tightening torque	
Bridge abutments on the model / intra-orally	35 Ncm
Prosthetic screw for bridge caps on the model	25 Ncm
AnoTite screw for bridge caps intra-orally	25 Ncm

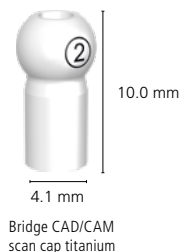7.2 tioLogic® ST bridge CAD/CAM scan cap titanium

The tioLogic® ST bridge CAD/CAM scan caps titanium are available specifically for use on bridge abutments. Place the scan cap intra-orally on the bridge abutment or extra-orally on the bridge laboratory implant in the master model. Then fixate with the enclosed AniTite screw (L 6.0 mm) and tighten with the hex key AF 1.3. The surface of the bridge scan cap titanium is optimized for digital capture, both intra-orally as well as on the model without scanning spray. The scanning and matching process of the scan data within the respective CAD software is performed according to the specifications of the individual software manufacturer. The bridge scan caps titanium are intended for multiple use while maintaining the integrity of the scan body and the connection to the (laboratory) implant. The AniTite screw (L 6.0 mm) is intended for single use and is marked on the blister packaging with

Tightening torque	
AnoTite screw for bridge caps on the model / intra-orally	25 Ncm

7.3 tioLogic® ST bridge impression components

The impression can be taken using either the open or closed technique. The relevant components with connector geometry on the bridge abutments are available for both impression-taking methods.


IMPORTANT:

When positioning the impression post on the bridge abutment, ensure it is seated without a gap. Please also note that the impression component should NOT come into contact with the individual impression tray.

Silicone or polyether impression materials are recommended for impression-taking due to their high precision and elastic recovery.

7.3.1 Open impression technique over bridge abutments

Impression post bridge

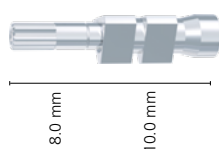To achieve a stable gingival attachment, it is possible to take an impression on the abutment that is fitted in the mouth.

To do this, fixate the bridge abutment in the implant and attach the corresponding open impression post. If the impression post is correctly seated on the bridge abutment, the visible marking on the screw is

level with the upper edge of the impression post (screw not fixated, only inserted). If the impression post does not fit flush, the marking on the screw will not be visible. Reposition the impression post, check for correct seating and tighten with the screw either by hand or with the hex key 1.3 width across flats.

Groove on sure-grip screw

When preparing and customizing the individual impression tray, make sure that neither the impression posts nor the screws are in contact with the tray perforation. The impression is to be taken with a material based on silicone or polyether. The impression posts are secured in the impression material by the retention. Care must be exercised to ensure that the screws are still accessible after the impression has been taken.

Once the impression material has cured, loosen the sure-grip screws, pull out and remove the impression tray with the impression posts. Send the impression tray together with the screws to the dental technician. Use the practice record card (REF 989-966-22) to provide the dental technician with all the relevant information. The abutments are then covered with the corresponding closure screws.

In the laboratory, the bridge laboratory implant is fixed on the impression post with the screw. The upper section of this laboratory implant is identical to that of the respective abutment.

Note:

- Impression posts should fit on the inserted implant without any gaps.
- The impression components should NOT come into contact with the individual impression tray.

Tightening torque

Screw for open impression post on the model/intra-orally	15 Ncm
--	--------

7.3.1 Closed impression technique over bridge abutments

Impression post bridge, closed, incl. impression cap.

Impression posts including screws and impression caps are available for taking the closed impression. For this purpose, the bridge abutment must be fixated in the implant and the corresponding

impression post for the closed impression screwed onto the abutment using the hex key AF 1.3. The corresponding impression cap is fitted according to the vertical retention grooves until it perceptibly and audibly clicks into place. The design of the retention grooves ensures that they can be positioned without coming into contact with the adjacent teeth.

When preparing and customizing the individual impression tray, make sure that neither the impression posts nor the screws are in contact with the tray perforation. The impression is to be taken with a material based on silicone or polyether. The impression posts are secured in the impression material by the retention.

Remove the impression tray after curing. Send the impression posts with AnoTite screws together with the impression to the laboratory. Use the practice record card (REF 989-966-22) to provide the dental technician with all the relevant information. The respective gingiva formers are refitted after impression-taking.

IMPORTANT:

The impression caps are items for single use only and are NOT suitable for sterilization or multiple use. Multiple use results in transfer inaccuracies.

Tightening torque

AnoTite screw on the model / intra-orally	15 Ncm
---	--------

7.4 tioLogic® ST bridge laboratory implants for printed and cast models

The bridge laboratory implant can be used in both analog and digital workflows. The cavities of the bridge laboratory analogs are stored in the CAD/CAM libraries of the various software manufacturers to achieve an optimum fit and the correct positioning of the bridge laboratory implants with the counter screw in 3D model printing. The parameters of the respective software must be checked and adjusted if necessary.

To ensure that the supplied counter screw can fixate the bridge laboratory implant at the apical end in the printed model, the model base should be printed with a sufficient height. If a detachable mucosa is created, it should not be lower than 1.0 mm below the interface to ensure the optimal locking and retaining function of the bridge laboratory implant in the neck area of the printed model. The 3 retention tabs should still be visible in the printed model for monitoring purposes.


When the model is printed with the planned cavities, the bridge laboratory implant for printed and cast models can be inserted. To this end, the laboratory implant is placed on the manual insertion key bar/Bridge/AngleFix. The bridge laboratory implant is then aligned in the cavity and pressed in. The counter screw supplied with the bridge laboratory implant can be used to fixate the bridge laboratory implant in the end position from apical.

Bridge laboratory implants for printed and cast models are intended for multiple uses in the laboratory, provided that the outer contour and the connection to the impression and abutment components remain intact.

Tightening torque

Counter screw in the model	15 Ncm
----------------------------	--------




7.5 tioLogic® ST bridge cap acrylic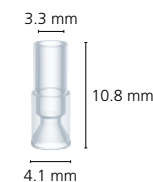

Plastic cap bridge

The bridge cap acrylic which burns out free of any residue, can be ground with a milling cutter for acrylics and shaped into the desired form of the mesostructure or the crown framework with standard burn-out modeling waxes or acrylics and individually adapted to the available space. The wax-up is fabricated taking the lingual and palatal overcasts into consideration. This procedure guarantees that there is still sufficient space for subsequent working stages and the pre-prosthetic planning can be maintained. The wax-up is then transferred to a framework using

the casting technique. The processing instructions given by the investment material manufacturer must be adhered to for investing and preheating. After casting, the casting ring must be cooled to room temperature for devesting to avoid tension in the cast object.

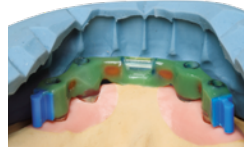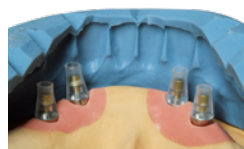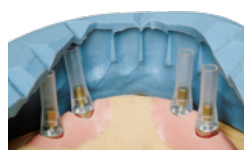The cast object should never be blasted with abrasive agents since these can damage the inner configuration. Devesting is carried out either by steam cleaning or in an ultrasonic bath. Finishing and polishing must be carried out according to the manufacturer's instructions. A countersink or cutter are available for the post-processing of the screw aperture or the screw seating. The seating surface for all bridge caps on the bridge abutments is identical (\varnothing 4.1 mm) for the S, M and L series of abutments. Fixate the bridge caps acrylic with the AnoTite screw L 6.0 mm to the bridge abutments and tighten with the hex key AF 1.3.

Passive fit ("Sheffield Test")


After laser-welding or casting, every bar restoration must be checked for passive fit on the model and prior to placing it intra-orally. This involves placing the construction on the bar abutments and fixation to the

bar abutment with only one prosthetic screw. If the construction is raised and creates a gap between the bar and bar abutment, then stresses are present and must be eliminated.

Tightening torque	
AnoTite screw for bridge caps intra-orally	25 Ncm
Prosthetic screw for bridge caps on model	25 Ncm

7.6 tioLogic® ST bridge cap titanium

Titanium cap bridge

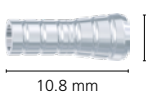The bridge caps, titanium, are available for bonding the cast bar construction intra-orally. They fit all three bridge abutments (S, M and L). The seating surface of all bridge abutments is identical (∅ 4.1 mm). Bonding the bridge caps, titanium, can compensate for imprecise casting in the tertiary structure and possible differences during impression-taking or model casting.

For a restoration with four implants on three bridge abutments, fixate the bridge caps, titanium, with the AnoTite screw L 6.0 mm and tighten with the hex key AF 1.3. These three act as adhesive bases. Fixate a bridge cap, acrylic, on the fourth bridge abutment. The bridge caps, titanium, for bonding can be shortened with a milling cutter or a cutting disc and individually adapted to the available spatial conditions. The bridge caps, titanium, must not become overheated during preparation as this can lead to differences in surface hardening (alpha-case layer). This could complicate or handicap further processing. Customization of the bridge caps titanium is performed taking the lingual and palatal overcasts into consideration. This procedure guarantees that there is still sufficient space for subsequent working stages and the pre-prosthetic planning can be maintained. The bridge caps titanium and the bridge cap, acrylic, are joined together with acrylic. The screw in the plastic cap must be loosened before raising the pattern. This keeps the cast bar in place during bonding.

The wax-up is then transferred to a framework using the casting technique. The processing instructions given by the investment material manufacturer must be adhered to for investing and preheating. After casting, the casting ring must be cooled to room temperature for devesting to avoid tension in the cast object. The cast object should never be blasted with abrasive agents since these can damage the inner configuration. Devesting is carried out either by steam cleaning or in an ultrasonic bath. Finishing and polishing must be carried out according to the manufacturer's instructions. A countersink or cutter are available for the post-processing of the screw aperture or the screw seating. Prior to bonding, the contact surfaces of the bridge caps titanium and the inner aspects of the bar must be provided with additional retention and conditioned according to the adhesive manufacturer's instructions. The inner hex of the AnoTite screw must be sealed with wax.

Once the adhesive has been mixed, it is applied to both contact surfaces of the three connectors, the cast bar construction is inserted intra-orally and fixed with the guide screw of the cast bridge cap, acrylic. To ensure that the prosthetic screws can be loosened easily, any excess adhesive above them must be removed before it hardens. Once the adhesive has hardened, loosen all prosthetic screws, remove the construction and remove any further adhesive residues. Reintegrate the cast construction. Further processing of the restoration can then continue.

Tightening torque	
AnoTite screw for bridge caps intra-orally	25 Ncm
Prosthetic screw for bridge caps on model	25 Ncm

7.7 tioLogic® ST bridge closure screw

The bridge closure screw is intended for closing the bridge abutment. The closure screws are placed on the bridge abutment to ensure that no liquids penetrate the bridge abutment during open healing and gingiva forming with bridge abutments and that the connector geometry to the bridge caps (tertiary structure) is not damaged and then tightened with hex key AF 1.3. They fit all three bridge abutments (S, M and L). The seating surface of all bridge abutments is identical (∅ 4.1 mm).

Tightening torque	
Bridge closure screw intra-oral	15 Ncm

7.8 Additional components

The prosthetic screw (not included in scope of delivery) serves only to help the dental technician fix the abutment on the working cast temporarily.

A new AnoTite screw should always be used for the final fitting and fixation of the abutment in the mouth at the prescribed torque.

Overview of additional components:

Torque ratchet
Hex key 1.3
Adapter, ISO shank, hexagon/ratchet
Insertion key bar/bridge/AngleFix ratchet
Adapter, ISO shank, hexagon/ratchet
Prosthetic screw on the model
Prosthetic screw for bridge caps on model

Caution:

Do not confuse the prosthetic screw with the AnoTite screw. The prosthetic screw is colored silver and the AnoTite screw is anodized gold.

8. Technical specification

Composition

Product	Composition
tioLogic® ST bridge abutment	3.7165
tioLogic® ST bridge scan cap, titanium	3.7065 / 3.7165
tioLogic® ST bridge impression post, open	3.7165 / VMQ
tioLogic® ST bridge impression post, closed	3.7165
tioLogic® ST bridge impression cap	POM
tioLogic® ST bridge laboratory implant	3.7165
tioLogic® ST bridge cap, acrylic	PC
tioLogic® ST bridge cap, titanium	3.7165
tioLogic® ST bridge closure screw	3.7165
tioLogic® ST bar/bridge/AngleFix insertion key, ratchet	1.4197
AnoTite screw	3.7165
Screw for impression post, open	3.7165 / VMQ
Counter screw	3.7165

Information about the composition is included in the material list for our implantology products (REF 989-801-06), or see www.dentaurum.com/ifu. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

9. Adverse reactions and residual risks

Possible complications and risks during prosthetic treatment (temporary adverse reactions) are:

Pain, swellings, difficulties speaking, inflammation of the gums.

Possible complications and risks during prosthetic treatment (permanent adverse reactions) are:

fractured dentures, aesthetic problems.

10. Safety information and precautions

- Do not use the product if there is a known allergic reaction to one or more of the material components.
- Due to the small size, the article could be swallowed or aspirated. Aspiration could lead to difficulty in breathing or death due to asphyxiation. Therefore, secure all articles used intra-orally against swallowing and/or aspiration.
- Different types of alloy in the oral cavity can lead to galvanic reactions.
- If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.
- Only use all tioLogic® ST prosthetic components in combination with tioLogic® ST implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaurum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- Observe the indications for the individual products. Consider all indications that are not listed explicitly to be contraindications.
- Disinfection, cleaning and sterilizing should not be done in the original packaging.
- Products must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.

- Technical advice (oral and written) is based on the scientific and technical knowledge available at the time the product is put on the market. It does not release the dental professional from the responsibility of personally checking the suitability of the products for the intended indications and applications. Advice is only given as a non-binding recommendation and cannot be assumed to provide any form of assurance, liability or guarantee.

Severe incidents:

Report all severe incidents arising from the use of the product to the manufacturer and the competent authority in the country in which the dental professional and/or the patient are resident.

Severe incidents are identified as malfunctions and a drop in performance or deterioration of the properties of a medical device which resulted or may have resulted in a direct or indirect temporary or permanent deterioration in a patient's state of health, in the death of a patient, a dental professional or other persons or as posing a serious danger to public health.

11. Storage and shelf life

Store the tioLogic® ST implants, abutments and components always in a dry, dark place at room temperature in the original packaging. If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

12. Disposal

Adhere strictly to the rules that apply for your region and the details outlined in the safety data sheets, if available. Products to be disposed of should be treated as surgical waste and be decontaminated in compliance with the relevant regulations for medical waste disposal.

13. Information for single use products

Prosthetic abutments and components are supplied non-sterile and are intended for single use on a single patient only. The product must be disinfected, cleaned and sterilized before first use.

Reconditioning of abutments and components (recycling) as well as reuse on patients are not permitted as safe reconditioning and/or functional safety cannot be guaranteed and may lead to a loss in mechanical, chemical and/or biological properties. Reuse may cause infection. Do not use implants after the expiry date stated on the label.

14. Information for multiple use products

The bridge CAD/CAM scan caps, titanium, are supplied non-sterile and are intended for multiple use. The product must be disinfected, cleaned and sterilized before being used initially and before being used again. Bridge scan caps, titanium, must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings and/or labels.

15. Information on disinfection / cleaning / sterilizing

The tioLogic® ST prosthetic abutments and components are supplied non-sterile. They are for single use on one patient only, unless explicitly intended for multiple use. They must be cleaned and disinfected before and after being used on a patient (e.g. transfer to dental laboratory). Dentaurem generally recommends that the product be sterilized in addition, as non-sterile components may cause infections. Only sterilized prosthetic components may be used if bleeding occurs.

16. Product overview and scope of delivery

Please refer to the tioLogic® ST product catalog for the tioLogic® ST product range.

17. Quality information

Dentaurem ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurem has no influence on the processing on site.

18. Explanation of symbols

Please refer to the label. See chapter "Additional information".

19. Information on MRT safety

The products within these Instructions for use are MRT safe.

20. Additional information

Additional information can be found at www.dentaurem.com/ifu

Instructions for use	REF
Screws	989-804-98
tioLogic® ST Additional components	989-805-87
Torque ratchet	989-719-00
pOPosition for tioLogic® rotating instruments and components for guided surgery	989-805-01
Preparation instructions for the implant system components	989-801-07
Explanation of symbols used on the label	989-313-00
MRI safety of Dentaurem products	989-804-43

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