tiologic®ST

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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

Prosthetic abutments and components replace missing teeth, restore chewing function and thus improve the quality of life. The implants of the tioLogic® implant family are used to fixate corresponding abutment components, depending on the indication. The abutment components serve to fabricate prosthetic meso-constructions, which are then restored with a prosthetic superstructure.

Abutment components					
Series of abutments*	S		M		L
Implant diameters in mm	3.3	3.7	4.2	4.8	5.5

*all of the abutment components listed in these Instructions for use are available for the S, M and L series of abutments.

The tioLogic® ST bar abutments are used to fabricate bridge and bar restorations that can be occlusally screw-retained. The bar abutments are marked according to the S, M or L series of abutments.

The bar abutments are manufactured in one piece and provided with an apical thread. The bar abutments should not be ground or shortened.

Bar abutment			
Series of abutments	S	M	L
GH in mm	1.0 / 2.5 / 4.0 / 5.5		

The tioLogic® ST bar CAD/CAM scan cap titanium is designed for digital capture of the prosthetic platform of the tioLogic® ST bar abutments intra-orally or the tioLogic® ST bar laboratory implants in the model. The tioLogic® ST bar CAD/CAM scan caps titanium are marked with a 1 for bar and are supplied together with an AniTite screw L 6.0 mm. The tioLogic® ST bar CAD/CAM scan caps titanium are reusable. The AniTite screw (L 6.0 mm) is intended for single use and is marked on the blister packaging with .

tioLogic® ST bar impression components with a connector geometry for bar abutment shoulders are available for the open and closed impression-taking methods. The tioLogic® ST bar impression post, open, is supplied together with a screw for open bar impression posts. The tioLogic® ST bar impression post, closed, includes a tioLogic® ST bar impression cap and an AniTite screw (L 6.0 mm).

tioLogic® ST bar laboratory implants for printed and cast models are used to create working casts. They accurately reflect the position of the tioLogic® ST bar abutments. The bar laboratory implants are marked with a 1 for bar and are provided together with a counter screw for model fixation. Processing and use take place exclusively in the (practice) laboratory or in the manufacturing center and not on the patient. This is not a medical device.

tioLogic® ST bar cap acrylic is designed for fabricating bridge and bar frameworks that are waxed up with modeling wax or acrylic, or are ground with a milling cutter for acrylic to achieve the final shape. The wax-up is then transferred to a framework using the casting technique. Processing and use take place exclusively in the (practice) laboratory or in the manufacturing center and not on the patient. The bar cap acrylic is supplied with an AniTite screw (L 6.0 mm). This is not a medical device.

tioLogic® ST bar cap titanium for lasering is intended for fabricating lasered bar restorations. The bar cap titanium for lasering is supplied with an AniTite screw (L 6.0 mm).

tioLogic® ST bar cap titanium is used for fabricating tension-free bonding in prosthetic restorations. The bar cap titanium is supplied with an AniTite screw (L 6.0 mm).

tioLogic® ST bar cap titanium is used for fabricating tension-free bonding in prosthetic restorations. In difficult spatial conditions, the bar caps titanium can be ground and shortened to the height of the upper edge of the screw for bonding. The bar cap titanium is supplied with an AniTite screw (L 6.0 mm).

tioLogic® ST bar closure screw is intended for closing the bar abutment. The closure screws are screwed onto the bar abutment to ensure that no liquids penetrate the bar abutment during healing and gingiva forming and that the connector geometry to the bar caps (tertiary structure) is not damaged.

The tioLogic® ST implant system contains Implant system with specially coordinated abutment and components incl. instruments.

3. Intended purpose

Abutments are intended to connect the implants with the prosthetic restoration.

tioLogic® ST bar CAD/CAM scan cap titanium is intended for taking digital impressions of the intra-oral situation and bar abutment position in the mouth or on a working cast and then transferring it to a working cast or to the CAD software.

tioLogic® ST bar impression components are designed to take an impression of the intra-oral bar abutment position three-dimensionally and transfer it to a working cast.

tioLogic® ST bar laboratory implants for printed and cast models are designed to represent the position of the bar abutment (axial inclination and height) in a working cast.

tioLogic® ST bar cap acrylic is intended to connect the implants with the prosthetic restoration.

Closure screws are intended to close the inserted implant or abutment during the healing period to prevent penetration of saliva.

4. Intended users

The tioLogic® ST product range is supplied exclusively to qualified personnel such as surgeons, dentists and dental technicians. It should only be used by qualified personnel who are familiar with dental implantology, including diagnosis, preoperative planning, surgical technique and prosthetic treatment.

5. Medical indications / contraindications

5.1 Indications

5.1.1 tioLogic® ST bar abutments

- For fabricating splinted bridges and bar restorations as connecting pieces between implants and prosthetic restorations (cemented, individually screw-retained)
- combination with other anchorage elements
- at least 6 implants in the maxilla and 4 implants in the mandible

5.1.2 tioLogic® ST bar CAD/CAM scan cap, titanium

- Localization of the exact implant position in the intra-oral scan
- Localization of the exact implant position in the model scan

5.1.3 tioLogic® ST bar impression components

- Intra-oral impression-taking on bar abutments in partially or fully edentulous jaws

5.1.4 tioLogic® ST bar laboratory implants for printed and cast models

- precise reproduction of positions when printing models
- precise reproduction of positions for cast models (dental stone)

5.1.5 tioLogic® ST bar cap acrylic

- For use on bar abutments
- Wax-up for the fabrication of bridge or bar frameworks

5.1.6 tioLogic® ST bar cap titanium, for lasering

- For use on bar abutments
- Direct laser welding with prefabricated titanium bar segments

5.1.7 tioLogic® ST bar cap titanium, for bonding

- For use on bar abutments
- Bonding in prosthetic restorations or in bar frameworks

5.1.8 tioLogic® ST bar cap titanium

- For use on bar abutments
- Bonding in provisional restorations or in bar frameworks

5.1.9 tioLogic® ST Bar closure screw

- Abutment closure for healing and gingiva forming
- Protection of the connector geometry to the tertiary structure during healing and gingiva forming

5.2 Contraindications

- In patients with a known intolerance to individual components.
- The contact surface between the implant and abutment must not be blasted or otherwise mechanically processed
- Insufficient oral hygiene
- Bruxism

5.2.1 tioLogic® ST bar abutments

- Use as single tooth restoration
- Mixed supported reconstructions (tooth / bar abutment)
- Grinding or shortening bar abutments
- For biomechanical reasons, we recommend that the following angulations of the implants are not exceeded:

Article	Angulation compensation
Bar abutment	-40° - 40°

5.2.2 tioLogic® ST bar CAD/CAM scan cap, titanium

- Use as restoration

5.2.3 tioLogic® ST bar impression components

- Use as restoration
- Use of the supplied screws on other prosthetic components

5.2.4 tioLogic® ST bar laboratory implants for printed and cast models

- Use of laboratory implants with defective or damaged connector geometry
- Use on patients

5.2.5 tioLogic® ST bar cap acrylic

- Use as single tooth restoration
- Use on patients
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bar abutments
- Use as direct restoration (without cast)

5.2.6 tioLogic® ST bar cap titanium, for lasering

- Use as single tooth restoration
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bar abutments

5.2.7 tioLogic® ST bar cap titanium, for bonding

- Use as single tooth restoration
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bar abutments

5.2.8 tioLogic® ST bar cap titanium

- Use as single tooth restoration
- Restorations with a length ratio to the length of the implant which exceeds 1:1.25
- Use of other abutments than on bar abutments

5.2.9 tioLogic® ST bar closure screw

- Use of other abutments than on bar abutments
- Use as restoration

6. Patient target group

All skeletally mature patients undergoing implant treatment. Please consider the indications / contraindications.

7. Information on use, processing, fitting and insertion

Prior to use, the dental professional ensures that they have carefully read and understood the tioLogic® ST Instructions for use.

We strongly recommend that, prior to using Dentaaurum products, dental professionals attend a Dentaaurum training course to familiarize themselves with the Dentaaurum products.

Carry out thorough radiological, psychological and physical examinations before implantation and inform patients of the process in detail. Dentaaurum recommends full clinical, radiological, photographic and statistical documentation.

The tioLogic® ST implant system's bar abutments and components can be documented, e.g. in the patient file or PatientPass, using the additional labels.

Precautions:

Before surgical implantation, it is important to carry out a careful patient anamnesis and, if required, a third-party anamnesis by the patient's doctor (GP/PCP) in order to clarify if 1) implantation may be difficult due to an anatomical situation, 2) there is the risk of a serious surgical problem or a general risk, 3) wound healing/osseointegration may be difficult, or 4) good hygiene and care of the implant, abutment and restoration may worsen. Do not fit dental implants if any of the above cases or a combination hereof is severe or uncontrollable.

Prosthetic restoration of the implant must not be performed until the soft tissue has healed without irritation. Furthermore, a radiographic check-up should be performed before the prosthetic restoration.

Important:

The implantologist should ensure the products cannot be aspirated during intra-oral use.

The CAD/CAM data sets for 3shape™, dental wings and exocad can be downloaded free of charge from www.dentaaurum.de/cadcam. The data sets include all the information needed for all types of restorations as a complete package, i.e. for printed implant models, hybrid abutments, one-piece abutments, as well as bridges and bars.


7.1 tioLogic® ST bar abutments

A prerequisite for successful use of the bar system is the most accurate possible angular position of the implant. The more accurate these angles can be maintained, the easier the prosthetic treatment. The insertion direction and the alignment of the abutments should be parallel to one another.

In fully edentulous cases, a minimum of four tioLogic® ST implants \varnothing 3.3 mm must be inserted in the mandible and a minimum of six in the maxilla, each with a splinted bar restoration without extension.

Restorations with bar abutments on tioLogic® ST implants require a minimum length of 11.0 mm.

When handling the bar abutments, ensure that the occlusal screw is not damaged by tweezers, etc.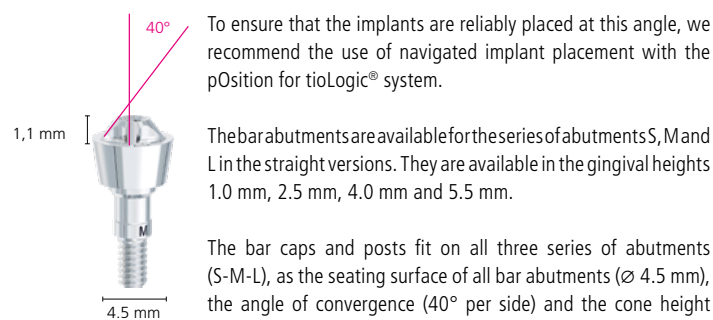

To ensure that the implants are reliably placed at this angle, we recommend the use of navigated implant placement with the pOsition for tioLogic® system.

The bar abutments are available for the series of abutments S, M and L in the straight versions. They are available in the gingival heights 1.0 mm, 2.5 mm, 4.0 mm and 5.5 mm.

The bar caps and posts fit on all three series of abutments (S-M-L), as the seating surface of all bar abutments (\varnothing 4.5 mm), the angle of convergence (40° per side) and the cone height (1.1 mm) are identical. They are secured in position on the bar abutments using the AniTite screw L 6.0 mm supplied.


For biomechanical reasons we recommend that the following angulations of the implants are not exceeded:

Article	Angulation compensation
Bar abutment	-40° - 40°

Using the tioLogic® ST insertion key bar/bar/AngleFix – ratchet, the bar abutments are screwed into the tioLogic® ST implant.

Tightening torque	
Bar abutment on the model / intra-orally	35 Ncm
Prosthetic screw for bar caps on the model	25 Ncm
AnoTite screw for bar caps intra-orally	25 Ncm

7.2 tioLogic® ST bar CAD/CAM scan cap, titanium

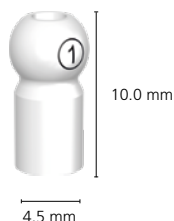The tioLogic® ST bar CAD/CAM scan caps, titanium, are available specifically for use on bar abutments. Place the scan cap intra-orally on the bar abutment or extra-orally on the bar laboratory implant in the master model. Then fixate with the enclosed AnoTite screw (L 6.0 mm) and tighten with the hex key 1.3 width across flats. The surface of the bar scan cap titanium is optimized for digital capture, both intra-orally as well as on the model without scanning spray. The scanning and matching process of the scan data within the respective

CAD software is performed according to the specifications of the individual software manufacturer. The bar scan caps titanium are intended for multiple use while maintaining the integrity of the scan body and the connection to the (laboratory) implant. The AnoTite screw (L 6.0 mm) is intended for single use and is marked on the blister packaging with

Tightening torque	
AnoTite screw for bar caps on the model/intra-orally	25 Ncm

7.3 tioLogic® ST bar impression components

The impression is taken according to the open impression technique. The relevant components with connector geometry on the bar abutments are available for this impression-taking method.


IMPORTANT:

When positioning the impression post on the bar abutment, ensure it is seated without a gap. The impression component should NOT come into contact with the individual impression tray.

Silicone or polyether impression materials are recommended for impression-taking due to their high precision and elastic recovery.

7.3.1 Open impression technique via bar abutments

Impression post bar

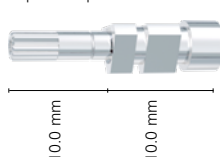To achieve a stable gingival attachment, it is possible to take an impression on the abutment that is fitted in the mouth.

To do this, fixate the bar abutment in the implant and attach the corresponding open impression post. If the impression post is correctly seated on the bar abutment, the visible marking on the screw is level

with the upper edge of the impression post (screw not fixated, only inserted). If the impression post does not fit flush, the marking on the screw will not be visible. Reposition the impression post, check for correct seating and tighten with the screw either by hand or with the hex key 1.3 width across flats.

Groove on sure-grip screw

When preparing and customizing the individual impression tray, make sure that neither the impression posts nor the screws are in contact with the tray perforation. The impression is to be taken with a material based on silicone or polyether. The impression posts are secured in the impression material by the retention.

Care must be exercised to ensure that the screws are still accessible after the impression has been taken. Once the impression material has cured, loosen the sure-grip screws, pull out and remove the impression tray with the impression post. Send the impression

tray together with the screws to the dental technician. Use the practice record card (REF 989-966-22) to provide the dental technician with all the relevant information. The abutments are then covered with the corresponding closure screws.

In the laboratory, the bar laboratory implant is fixed on the impression post with the screw. The upper section of this laboratory implant is identical to that of the respective abutment.

Note:

- Impression posts should fit on the inserted implant without any gaps.
- The impression components should NOT come into contact with the individual impression tray.

Tightening torque	
Screw for open impression post on the model/intra-orally	15 Ncm

7.4 tioLogic® ST bar laboratory implants for printed and cast models

The bar laboratory implant can be used in both analog and digital workflows. The cavities of the bar laboratory analogs are stored in the CAD/CAM libraries of the various software manufacturers to achieve an optimum fit and the correct positioning of the bar laboratory implants with the counter screw in 3D model printing. The parameters of the respective software must be checked and adjusted if necessary.

To ensure that the supplied counter screw can fixate the bar laboratory implant at the apical end in the printed model, the model base should be printed with a sufficient height. If a detachable mucosa is created, it should not be lower than 1.0 mm below the interface to ensure the optimal locking and retaining function of the bar laboratory implant in the neck area of the printed model. The 3 retention tabs should still be visible in the printed model for monitoring purposes.

When the model is printed with the planned cavities, the bar laboratory implant for printed and cast models can be inserted. To this end, the laboratory implant is placed on the manualAngleFix insertion key bar/bridge. The bar laboratory implant is then aligned in the cavity and pressed in. The counter screw supplied with the bar laboratory implant can be used to fixate the bar laboratory implant in the end position from apical.

Bar laboratory implants for printed and cast models are intended for multiple uses in the laboratory, provided that the outer contour and the connection to the impression and abutment components remain intact.



Tightening torque	
Counter screw in the model	15 Ncm

7.5 tioLogic® ST bar cap acrylic

The bar cap acrylic which burns out free of any residue, can be ground with a milling cutter for acrylics and shaped into the desired form of the mesostructure or the crown framework with standard burn-out modeling waxes or acrylics and individually adapted to the available space. The wax-up is fabricated taking the lingual and palatal overcasts into consideration. This procedure guarantees that there is still sufficient space for subsequent working stages and the pre-prosthetic planning can be maintained. The wax-up is then transferred to a framework using the casting technique.

The processing instructions given by the investment material manufacturer must be adhered to for investing and preheating. After casting, the casting ring must be cooled to room temperature for devesting to avoid tension in the cast object. The cast object should never be blasted with abrasive agents since these can damage the inner configuration. Devesting is carried out either by steam cleaning or in an ultrasonic bath. Finishing and polishing must be carried out according to the manufacturer's instructions. A countersink or cutter are available for the post-processing of the screw aperture or the screw seating. The seating surface for all bar caps on the bar abutments is identical (\varnothing 4.5 mm) for the S, M and L series of abutments. Fixate the bar caps acrylic with the AnoTite screw L 6.0 mm to the bar abutments and tighten with the hex key AF 1.3.

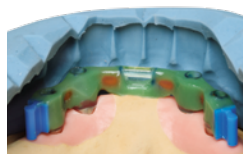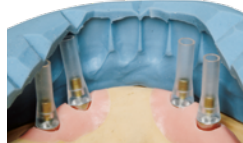Passive fit ("Sheffield Test")

After laser-welding or casting, every bar restoration must be checked for passive fit on the model and prior to placing it intra-orally. This involves placing the construction on the bar abutments and fixation to the bar abutment with only one prosthetic screw. If the construction is raised and creates a gap between the bar and bar abutment, then stresses are present and must be eliminated.


construction is raised and creates a gap between the bar and bar abutment, then stresses are present and must be eliminated.

Tightening torque	
AnoTite screw for bar caps intra-orally	25 Ncm
Prosthetic screw for bar caps on model	25 Ncm

7.6 tioLogic® ST bar cap titanium, for lasering


Depending on the impression technique, the prefabricated conical bar caps, titanium should be fixated on the bar abutments or the bar laboratory implants with the enclosed AnoTite L 6.0 mm screw and tightened with the hex key AF 1.3. They fit all three bar abutments (S, M and L), as the seating surface of all bar abutments is identical (∅ 4.5 mm).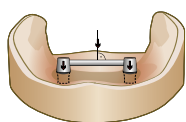

Horizontal bar positioning:

To ensure that the masticatory forces are directed correctly, the bar must be positioned horizontally to the ideal occlusal plane. Tilting the bar would load the implants incorrectly and cause excessive pressure to be exerted on the mucous membranes.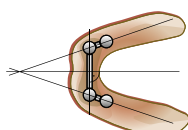

Vertical bar positioning:


With fixed/free dentures, the bar attachment acts as an axis of rotation. To ensure that the alveolar ridges are loaded uniformly, the bar must be aligned at an angle of 90° to the bisection angle of the dental arches.

The prefabricated titanium bars are shortened and fixated in the correct horizontal axis for laser-welding with a parallelometer and fixator for the laser technique. The bar components and titanium caps are then lasered together. Pure titanium wire (e.g. Dentaorium GmbH & Co. KG) is required as filler. Should further questions arise on laser-welding and processing titanium, please contact your local representative. Trimming and polishing of the titanium bar after laser-welding (e.g. with the rematitan® finishing kit, Dentaorium GmbH & Co. KG).

The titanium bar rider is then laser-welded to the titanium retention and polymerized into the denture.

Tightening torque	
AnoTite screw for bar caps intra-orally	25 Ncm
Prosthetic screw for bar caps on model	25 Ncm

7.7 tioLogic® ST Bar cap titanium, for bonding

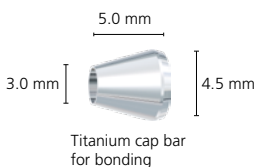bar caps, titanium, for bonding, are available for bonding the cast bar construction intra-orally. They fit all three bar abutments (S, M and L). The seating surface of all bar abutments is identical (∅ 4.5 mm). Bonding the bar caps, titanium, for bonding, can compensate for imprecise casting in the tertiary structure and possible differences during impression-taking or model casting.

For a restoration with four implants on three bar abutments, fixate the bar caps, titanium, for bonding, with the AnoTite screw L 6.0 mm and tighten with the hex key AF 1.3. These three act as adhesive bases. Fixate a bar cap, acrylic on the fourth bar abutment.

The bar caps, titanium, must not become overheated during processing as this can lead to differences in surface hardening (alpha-case layer). This could complicate or handicap further processing. Processing of the bar caps titanium, for bonding, is performed taking the lingual and palatal overcasts into consideration. This procedure guarantees that there is still sufficient space for subsequent working stages and

the pre-prosthetic planning can be maintained. The bar caps titanium, for bonding, and the bar cap acrylic are joined together with acrylic. Before lifting off the model, loosen the screw of the plastic cap. This keeps the cast bar in place during bonding.

The wax-up is then transferred to a framework using the casting technique. The processing instructions given by the investment material manufacturer must be adhered to for investing and preheating. After casting, the casting ring must be cooled to room temperature for devesting to avoid tension in the cast object. The cast object should never be blasted with abrasive agents since these can damage the inner configuration. Devesting is carried out either by steam cleaning or in an ultrasonic bath. Finishing and polishing must be carried out according to the manufacturer's instructions. A countersink or cutter are available for the post-processing of the screw aperture or the screw seating. Prior to bonding, the contact surfaces of the bar caps titanium and the inner aspects of the bar must be provided with additional retention and conditioned according to the adhesive manufacturer's instructions. Seal the inner hex of the AnoTite screw with wax.

Once the adhesive has been mixed, it is applied to both contact surfaces of the three connectors, the cast bar construction is inserted intra-orally and fixated with the guide screw of the cast bar cap acrylic. Remove excess adhesive above the prosthetic screws before the adhesive hardens to ensure that the screws can be loosened easily. Once the adhesive has hardened, loosen all prosthetic screws, remove the construction and remove any further adhesive residues. Reintegrate the cast construction. Further processing of the restoration can then continue.

Tightening torque	
AnoTite screw for bar caps intra-orally	25 Ncm
Prosthetic screw for bar caps on model	25 Ncm

7.8 tioLogic® ST bar closure screw

The bar closure screw is intended for closing the bar abutment. The closure screws are placed on the bar abutment to ensure that no liquids penetrate the bar abutment during open healing and gingiva forming with bar abutments and that the connector geometry to the bar caps (tertiary structure) is not damaged and then tightened with hex key AF1.3. They fit all three bar abutments (S, M and L). The seating surface of all bar abutments is identical (∅ 4.5 mm).

Tightening torque	
Bar closure screw intra-orally	15 Ncm

7.9 Additional components

The prosthetic screw (not included in scope of delivery) serves only to help the dental technician fix the abutment on the working cast temporarily. A new AnoTite screw should always be used for the final fitting and fixation of the abutment in the mouth at the prescribed torque.

Overview of additional components:

Torque ratchet
Hex key 1.3
Adapter, ISO shank, hexagon/ratchet
Insertion key bar/bridge/AngleFix ratchet
Adapter, ISO shank, hexagon/ratchet
Prosthetic screw on the model
Prosthetic screw for bar caps on model

Caution:

Do not confuse the prosthetic screw with the AnoTite screw. The prosthetic screw is colored silver and the AnoTite screw is anodized gold.

8. Technical specification

Composition

Product	Composition
tioLogic® ST bar abutment	3.7165
tioLogic® ST bar scan cap titanium	3.7065 / 3.7165
tioLogic® ST bar impression post, open	3.7165 / VMQ
tioLogic® ST bar impression post, closed	3.7165
tioLogic® ST bar impression cap	POM
tioLogic® ST bar laboratory implant	3.7165
tioLogic® ST bar cap acrylic	PC
tioLogic® ST bar cap titanium, for lasering	3.7165
tioLogic® ST bar cap titanium, for bonding	3.7165
tioLogic® ST bar cap titanium	3.7165
tioLogic® ST bar closure screw	3.7165
tioLogic® ST bar/bridge/AngleFix insertion key, ratchet	1.4197
AnoTite screw	3.7165
Screw for impression post, open	3.7165 / VMQ
Counter screw	3.7165

Information about the composition is included in the material list for our implantology products (REF 989-801-06), or see www.dentaurum.com/ifu. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

9. Adverse reactions and residual risks

Possible complications and risks during prosthetic treatment (temporary adverse reactions) are:

Pain, swellings, difficulties speaking, inflammation of the gums.

Possible complications and risks during prosthetic treatment (permanent adverse reactions) are:

fractured dentures, aesthetic problems.

10. Safety information and precautions

- Do not use the product if there is a known allergic reaction to one or more of the material components.
- Due to the small size, the article could be swallowed or aspirated. Aspiration could lead to difficulty in breathing or death due to asphyxiation. Therefore, secure all articles used intra-orally against swallowing and/or aspiration.
- Different types of alloy in the oral cavity can lead to galvanic reactions.
- If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.
- Only use all tioLogic® ST prosthetic components in combination with tioLogic® ST implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaurum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- Observe the indications for the individual products. Consider all indications that are not listed explicitly to be contraindications.
- Disinfection, cleaning and sterilizing should not be done in the original packaging.
- Products must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.
- Technical advice (oral and written) is based on the scientific and technical knowledge available at the time the product is put on the market. It does not release the dental professional from the responsibility of personally checking the suitability of the products for the intended indications and applications. Advice is only given as a non-binding recommendation and cannot be assumed to provide any form of assurance, liability or guarantee.

Severe incidents:

Report all severe incidents arising from the use of the product to the manufacturer and the competent authority in the country in which the dental professional and/or the patient are resident.

Severe incidents are identified as malfunctions and a drop in performance or deterioration of the properties of a medical device which resulted or may have resulted in a direct or indirect temporary or permanent deterioration in a patient's state of health, in the death of a patient, a dental professional or other persons or as posing a serious danger to public health.

11. Storage and shelf life

Store the tioLogic® ST implants, abutments and components always in a dry, dark place at room temperature in the original packaging. If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

12. Disposal

Adhere strictly to the rules that apply for your region and the details outlined in the safety data sheets, if available. Products to be disposed of should be treated as surgical waste and be decontaminated in compliance with the relevant regulations for medical waste disposal.

13. Information for single use products

Prosthetic abutments and components are supplied non-sterile and are intended for single use on a single patient only. The product must be disinfected, cleaned and sterilized before first use.

Reconditioning of abutments and components (recycling) as well as reuse on patients are not permitted as safe reconditioning and/or functional safety cannot be guaranteed and may lead to a loss in mechanical, chemical and/or biological properties. Reuse may cause infection. Do not use implants after the expiry date stated on the label.

14. Information for multiple use products

The bar CAD/CAM scan caps, titanium, are supplied non-sterile and are intended for multiple use. The product must be disinfected, cleaned and sterilized before being used initially and before being used again. Bar scan caps, titanium, must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.

15. Information on disinfection / cleaning / sterilizing

The tioLogic® ST prosthetic abutments and components are supplied non-sterile. They are for single use on one patient only, unless explicitly intended for multiple use. They must be cleaned and disinfected before and after being used on a patient (e.g. transfer to dental laboratory). Dentaurum generally recommends that the product be sterilized in addition, as non-sterile components may cause infections. Only sterilized prosthetic components may be used if bleeding occurs.

16. Product overview and scope of delivery

Please refer to the tioLogic® ST product catalog for the tioLogic® ST product range.

17. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

18. Explanation of symbols

Please refer to the label. See chapter "Additional information".

19. Information on MRT safety

The products within these Instructions for use are MRT safe.

20. Additional information

Additional information can be found at www.dentaurum.com/ifu

Instructions for use	REF
Screws	989-804-98
tioLogic® ST Additional components	989-805-87
Torque ratchet	989-719-00
pOosition for tioLogic® rotating instruments and components for guided surgery	989-805-01
Preparation instructions for the implant system components	989-801-07
Explanation of symbols used on the label	989-313-00
MRI safety of Dentaurum products	989-804-42

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