hyrax®
hyrax® neo

Schraube ohne Retentionsarme
Screw without retention legs

Vis sans bras de rétention

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D
DENTAURUM

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Más informaciones sobre los productos se hallan en www.dentaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 07/21

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications | Reservado el derecho de modificación |
Con riserva di apportare modifiche

D
DENTAURUM
Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 7231/803-0 | Fax +49 7231/803-295
www.dentaurum.com | info@dentaurum.com

ES

Estimado cliente

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EN

Dear customer,

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General product description

Hyrax® and Hyrax® neo screws without retention legs (hygienic rapid expansion screw, figs. 1–2) are special screws used for fast palatal expansion. The body of this screw is welded onto individual retention legs, connectors and/or so-called cap splints which have been either cast or produced by the SLM method in metal. The appliances are cemented or fixed to the Tomax®-pin as usual. Follow the instructions from the cement manufacturer.

3. Area of application

Used for rapid palatal expansion with a fixed appliance.

4. Fabrication of a Hyrax®/Hyrax® neo appliance with individualized retention legs and cap splints

The following steps should be followed to manufacture individualized retention legs and cap splints by casting or by the SLM method (Fig. 3–5). Individualized cap splints are fitted on the first premolars in the maxilla and the first molars, each left and right (Fig. 10–11). The retention legs are connected to the Hyrax®/Hyrax® neo screw in such a way that two weld seams form along the lateral or the upper groove (Fig. 8–9). The Hyrax®/Hyrax® neo screw should be “suspended” at least 1–2 mm above the palatal vault/gingiva (Fig. 6–7) and its dorsal side should be in line with the mesio-palatal cusps of the first molars. In order to avoid leverage, the screws must be situated parallel to the occlusal plane. In order to facilitate the task of inserting the safety key (Fig. 13) and activating the screw, the Hyrax®/Hyrax® neo screw is fitted with the direction of rotation facing towards dorsal.

5. Tips for working with Hyrax®/Hyrax® neo screws without retention legs

For the connection to be durable, the individualized retention legs must be welded onto the groove of the body of the Hyrax®/Hyrax® neo screw (Fig. 8–9), for example using the welding unit desktop Compact (REF 090-578-50). As an average value for NiCr, the initial laser setting on the unit is as follows: 240 volt, 4 ms, 0.8 mm. The use of a shielding gas is indispensable during the welding process. When using or adding new material to reinforce the welding area, we recommend the NiCr welding wire, Ø 0.50 mm (REF 528-220-00, Fig. 12).

6. Composition

The composition is included in the materials list; please refer to the catalog or www.dentaurum.com.

7. Safety instructions

Always use only the blue safety key (Fig. 13) supplied to prevent swallowing!

The product should not be used if there is a known allergic reaction to one or more of the material components.

8. Information for single use products

The Hyrax®/Hyrax® neo screw without retention legs is intended for single use only. Reconditioning a used Hyrax®/Hyrax® neo screw (recycling) and its reuse on a patient are not permitted.

9. Additional information

- There are no particular precautions to be taken when disposing of Hyrax®/Hyrax® neo screws after treatment.
- Should you have any questions, please contact your local representative.

10. Product range

Please refer to our Orthodontics catalog for the complete range of Hyrax®/Hyrax® neo screws.

11. Quality

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

12. Explanation of symbols used on the label

⚠️ Refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

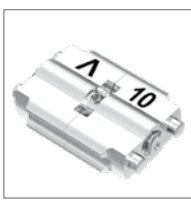
Fig. 1 Hyrax®

Fig. 2 Hyrax® neo

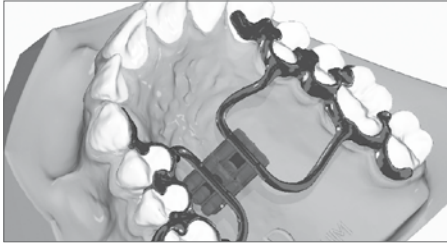
Fig. 3 Hyrax® neo

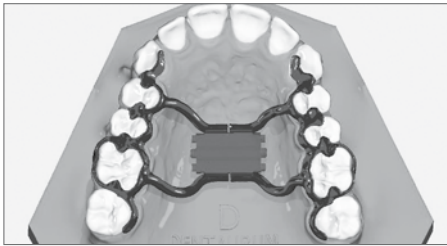
Fig. 4 Hyrax®

Fig. 5 Hyrax® neo

Fig. 6 Hyrax®

Fig. 7 Hyrax® neo

Fig. 8 Hyrax®

Fig. 9 Hyrax® neo

Fig. 10 Hyrax®

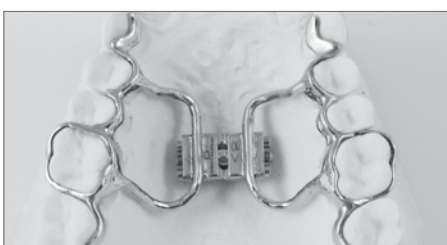
Fig. 11 Hyrax® neo

Fig. 12

Fig. 13

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Information for the doctor/patient

Before cementing the Hyrax®/Hyrax® neo appliance into position, open the spindle (screw) by making 2–3 quarter turns in the direction of the arrow to test mobility. Then turn back to the original position.

Turn the Hyrax®/Hyrax® neo screw only in the direction of the arrow. Opening and closing the screw several times during treatment can cause it to loosen and no longer function properly.

The finished Hyrax®/Hyrax® neo appliance is cemented into position in the usual manner.

Afterwards, give the spindle one full turn (4 x 90°) with the safety key supplied. One full 360° turn of the spindle = 0.8 mm expansion.

On the basis of your instructions, the patient should complete two quarter turns (2 x 90°) in the morning and two quarter turns in the evening until the desired expansion is achieved.

Note: Always use only the blue safety key supplied to prevent swallowing!

The safety key is designed to take a maximum force of 35 N ± 3.5 kg. Please note that the force exerted on the safety key wire should always be exerted on the closed side of the wire clasp so that the clasp cannot loosen from the axis. Improper use may lead to overload and consequently to failure of the safety key, thus potentially excluding any guarantee and liability.

Regular visits are recommended to monitor progress.

The Hyrax® neo screw has, due to its construction, no active stop (indent in pin offers passive stop). If the Hyrax® neo screw is turned too far, the spindle loses its function and the screw is held in position by the pins. It is then no longer possible to turn the Hyrax® neo screw back.

The Hyrax®/Hyrax® neo appliance is secured against unwanted reversing, after the desired degree of expansion is achieved, by fitting a ligature or blocking with acrylic. The appliance should remain in the mouth for approximately 2-3 months for stabilization purposes.

A retainer should be worn following removal of the appliance.

⚠️ The product should not be used if there is a known allergic reaction to one or more of the material components.

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident is informed accordingly.

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