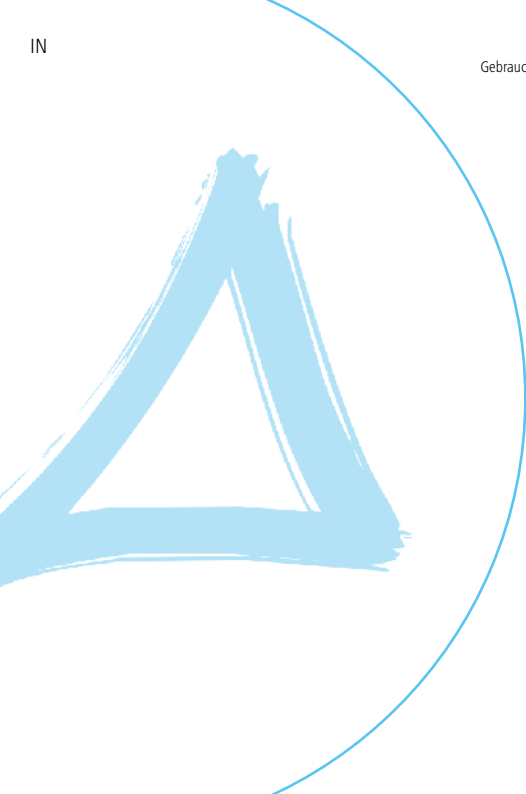
Triceram[®]

Magic Paste

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Dentaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

1. Composition (% by mass)

Colored glass ceramic pastes, made from

50%-60%	SiO ₂
6%-11%	Al ₂ O ₃
0-4%	K ₂ O
0-13%	Na ₂ O
0-3%	Li ₂ O
0-3%	CaO
0-2%	MgO
0-11%	B ₂ O ₃
0-2%	BaO
0-5%	Pigments from inorganic oxide-ceramics

Fluids used to layer ceramic masses, to adjust viscosity and to reinforce contrasts consist mainly of alcohols.

2. Technical data

The technical data was determined on the basis of DIN EN ISO 6872.

Triceram® Magic Paste	Type	Class	Coefficient of thermal expansion α (25 °C to 500 °C/77 °F to 932 °F or T_g) in $1 \cdot 10^{-6} \text{ K}^{-1}$	Glass transition temperature T_g in °C (°F)	Chemical solubility L in $\mu\text{g}/\text{cm}^2$	Flexural strength σ in MPa
Paste Glaze	I	1	9.0	490	40	120
Magic Paste 2D	I	1	9.1	490	40	120
Magic Paste 3D	I	1	9.0	480	40	140

3. Intended purpose

Dental ceramic semi-finished products are intended for the production of dental restorations to restore both the chewing function and aesthetics.

4. Indication

Bonding ceramic for the micro-layering technique, staining technique, cut-back technique, to correct, characterize and glaze fully anatomical restorations and restorations made of zirconium oxide with a coefficient of thermal expansion ranging from $9.7 \cdot 10^{-6} \text{ K}^{-1}$ to $10.6 \cdot 10^{-6} \text{ K}^{-1}$ (25 °C to 500 °C/77 °F to 932 °F) that have been fully or partially veneered with Triceram®, or made of lithium disilicate.

5. Contraindications and adverse reactions

There is no evidence of incompatibility with these materials when manufacturing or processing in accordance with the Instructions for use.

The product should not be used if there is a known allergic reaction to one or more of the material components.

Ceramic restorations are not recommended or are only recommended to a limited extent in the case of bruxism or other parafunctions. Consultation with the attending dentist is strongly recommended.

The following are generally contraindicated: inadequate oral hygiene, extreme dystrophy of the teeth, non-suitable or self-adhesive cementation, use or combination of the product with materials which are not included in the indications.

6. Patient population

Patients with defects in tooth structure for whom a restoration is necessary and indicated.

7. Intended users

Processing must be carried out only by dental professionals with knowledge of the production of dental restorations. These Instructions for use alone are insufficient for personnel without adequate training to professionally and correctly process prosthetic restorations and related components. Non-observance of this advice may lead to failure of the restoration and other complications.

8. Warnings and precautions to be taken

Dust may also be generated during mechanical processing. The mechanical process will also create heat and possibly burs. Processed material may therefore be hot and/or have sharp edges. Vapors may also form if liquid substances are used during processing. Exposure to dust and vapors may cause irritation of the eyes and/or respiratory tract. Equip the workplace with a suitable suction or ventilation system. We generally recommend that gloves, protective clothing, safety glasses and respiratory protection equipment are worn. Should there be contact with the skin, wash the affected area with soap and water for a few minutes. Should the product come into contact with the eye, rinse the eye with a gentle stream of water or saline solution for at least 15 minutes. Consult a doctor if you are exposed to critical substances.

Adhere to warnings and safety data sheets.

If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

The prosthetic restoration should not be ground or polished in the patient's mouth. There is no scientific evidence available on the safety or efficacy of treatment of pregnant women or nursing mothers, or children.

9. Reuse

Finished restorations which have been produced using semi-finished products are intended for single use only.

10. Storage and/or handling conditions

The product properties are not affected by normal changes in environmental conditions (such as temperature, pressure, humidity or light). Close the original packaging after use to prevent contamination. In general, the product should be kept dry and at room temperature in the tightly closed original packaging. Do not return used pastes to the jar.

11. Disposal

Adhere strictly to the rules that apply for your region, bearing in mind the details outlined in the safety data sheets.

12. Characteristics of the products

Glaze Paste

Paste Glaze
Fluorescent glaze

2D pastes

Paste 2D Body A
Body paste for Vita classical® A shades

Paste 2D Body B
Body paste for Vita classical® B shades

Paste 2D Body C
Body paste for Vita classical® C shades

Paste 2D Body D
Body paste for Vita classical® D shades

2D pastes

Paste 2D honey
Characterization: orange

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Paste 2D white
Characterization: white

Paste 2D grey
Characterization: grey

Paste 2D blue
Characterization: blue

Paste 2D violet
Characterization: violet

3D pastes

Paste 3D lumin
bright incisal for bright shades

Paste 3D I2
medium incisal for medium shades

Paste 3D opal grey
Modifier – grey opalescent

Paste 3D opal honey
Modifier – orange opalescent

3D pastes

Paste 3D opal blue
Modifier – blue opalescent

Paste 3D neutral
reduces the intensity of opal pastes

Liquids

Diluting Liquid
Adjusting the viscosity of the pastes

Refreshing Liquid
Adjusting the viscosity and maintaining
the stability of 3D pastes

13. Using the products

13.1 Wax-up

Triceram® Magic Paste products have been specially developed for the micro-layering technique. The 2D pastes are applied in thin layers. The 3D pastes can be applied more thickly to correct the shape and to achieve an incisal effect with micro-layering cut back.

Before each use, the pastes must be mixed thoroughly with a metal-free spatula and refreshed with corresponding liquids if required.

13.2 Processing steps for build-up

Step 1 (Framework preparation): The framework is conditioned in accordance with the manufacturer's instructions.

Step 2: Apply the Paste Glaze thinly and evenly over the entire object. Apply 2D Body pastes in the tooth shade / intensity required. Place individual 2D incisal or other effect pastes.

Step 3: Fire in the ceramic furnace in accordance with the firing chart parameters for "1st firing".

Step 4: Apply the 3D pastes where required on the object (final shaping). The 2D pastes can be used to help optimize the shade.

Step 5: Fire in the ceramic furnace in accordance with the firing chart parameters for "2nd firing". Glaze firing is not necessary. The 2D and 3D pastes can also be fired in one firing together.

13.3 Firing Chart

	Temperature at start °C (°F)	Drying time min	Heating rate* °C (°F)/min	Vacuum start °C (°F)	Vacuum end °C (°F)	Firing temperature °C (°F)	Holding time**
1st firing 2D and 3D pastes	450 (842)	8	55 (131)	450 (842)	730 (1346)	730 (1346)	1 min (without vacuum)
2nd firing Correction 2D and 3D pastes	450 (842)	8	55 (131)	450 (842)	720 (1328)	720 (1328)	1 min (without vacuum)

* Reduce the heating rate for large zirconium oxide restorations and cool more slowly.

** Extend the hold-time in the case of large restorations to compensate for the poor thermal conductivity of ZrO₂.

Work with long-term cooling for large restorations.

The firing temperatures indicated were determined in a calibrated furnace. Use only calibrated furnaces to ensure the required product quality. Firing temperatures may deviate nonetheless depending on the furnace. It may be necessary to individually adapt the specifications given. A test firing is recommended, whereby Triceram Neutral transparent for example is mixed with Modelling Liquid. Place the sample on a platinum foil and fire on the basis of the firing parameters for Triceram in the dentin firing 1. Visually assess and determine the correct furnace temperature by increasing or decreasing the temperature incrementally in 10 °C/50 °F steps. The furnace temperature is optimal if the sample is translucent and has sharp edges. If the temperature is too high, the sample has no sharp edges and appears to be very shiny. If the temperature is too low, the sample is white and milky.

14. Shade in accordance with the manufacturer's shade guide

Triceram® Magic pastes are based on the shade systems VITA Classical A1-D4® and VITA SYSTEM 3D-MASTER® (VITA Classical A1-D4® and VITA SYSTEM 3D-MASTER® are registered trademarks of Vita Zahnfabrik H.Rauter GmbH & Co KG, 79704 Bad Säckingen, Germany).

15. Information on combining with other products

The dental ceramic may only be combined with other products if recommended by Dentaureum. Only frameworks may be veneered where the coefficient of thermal expansion lies within the given range for veneering ceramics.

16. General information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, inform the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident.

The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaureum.com.

For more information on our products and services, visit www.dentaureum.com. You will find these Instructions for use and information on the current status at www.dentaureum.com/ifu.

In case of questions or ideas, please contact your local representative.

17. Explanation of symbols used on the label

Refer to the label. Additional information can be found at www.dentaureum.com/ifu (Explanation of symbols REF 989-313-00).

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