

ceraMotion® Z Hybrid – Nesting information


ceraMotion® Z Hybrid allows you to reliably achieve reproducible results regarding shade and mechanical properties.


The multi-shaded, multi-layered zirconium oxide **ceraMotion® Z Hybrid** is a truly multi-purpose material. In one blank, it combines the aesthetics of a cubic zirconium oxide with the strength of a highly translucent zirconium oxide.

ceraMotion® Z Hybrid offers pure aesthetics and is suitable for all restorations in the anterior and posterior region. By means of a special high-tech process, two different zirconium oxide materials are joined in one blank. **ceraMotion® Z Hybrid** is available in the 8 most popular V-shades and 2 bleach shades, and offers reliable shade reproduction.

Product advantages:

- Reliable reproducibility for high-level aesthetics
- Multiple indications due to high strength in the dentin area
- Natural and smooth shade transitions from cervical to incisal
- High resistance to warping in long-span bridges
- Available in the 8 most popular V-shades + 2 bleach shades
- Excellent machinability
- Recommended sintering temperature 1480 °C/ 2696 °F
- Translucency 44 % - 47 % (non-shaded, 1.0 mm)
- Economical finishing with ceraMotion® One Touch
- Reliable veneering with ceraMotion® Zr

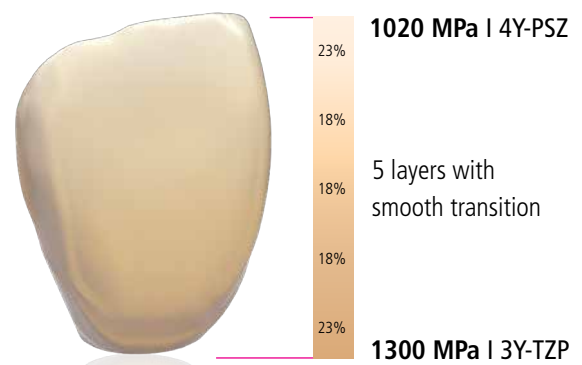

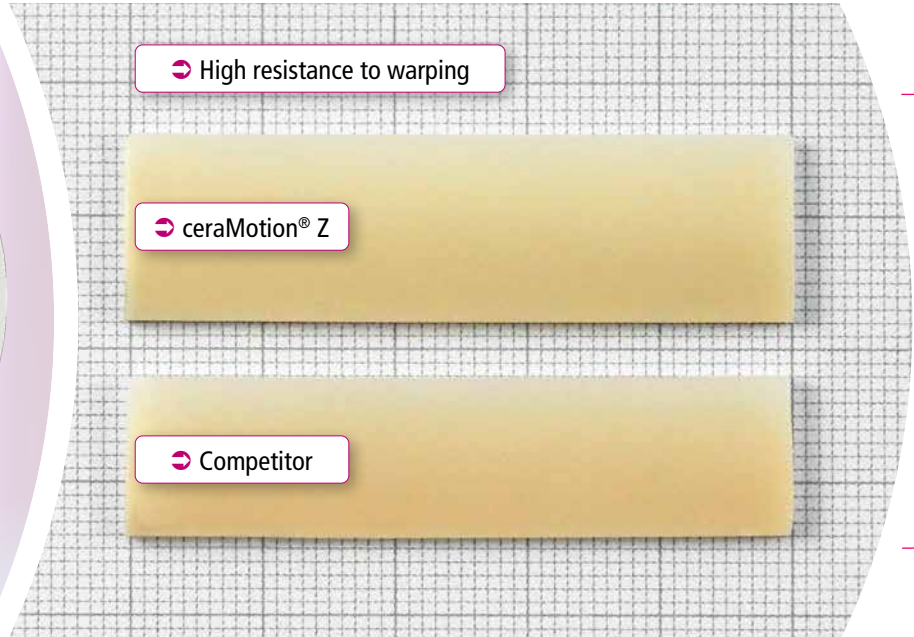

Getting the blank height right!

The blank height should be as close as possible to the enlarged, CAD-constructed crown.

How do I choose the right blank height?

CAD-constructed crown up to 10 mm → blank height 14 mm

CAD-constructed crown up to 13.4 mm → blank height 18 mm

CAD-constructed crown up to 16 mm → blank height 22 mm

Overview

Shade	REF (14 mm)	REF (18 mm)	REF (22 mm)	Qty
A1	272-411-14	272-411-18	272-411-22	1 piece
A2	272-412-14	272-412-18	272-412-22	1 piece
A3	272-413-14	272-413-18	272-413-22	1 piece
A3.5	272-415-14	272-415-18	272-415-22	1 piece
A4	272-414-14	272-414-18	272-414-22	1 piece
B1	272-421-14	272-421-18	272-421-22	1 piece
B2	272-422-14	272-422-18	272-422-22	1 piece
C2	272-432-14	272-432-18	272-432-22	1 piece
Bleach 1	272-300-14	272-300-18	272-300-22	1 piece
Bleach 2	272-400-14	272-400-18	272-400-22	1 piece

