D
DENTAURUM
1886

tiologic[®]
TWINFIT

tiologic[®] ST

CITO *mini*[®]

DE **Gebrauchsanweisung** · *tiologic*[®] ST, *tiologic*[®] TWINFIT, CITO *mini*[®] Rotierende Instrumente

EN **Instructions for use** · *tiologic*[®] ST, *tiologic*[®] TWINFIT, CITO *mini*[®] Rotary instruments

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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

The ADVANCED for tioLogic® drills are intended for the preparation of the implant site for tioLogic® ST and tioLogic® TWINFIT implants. The ADVANCED for tioLogic® drills are supplied non-sterile and packaged individually or non-sterile as part of the tioLogic® ST surgical tray or tioLogic® TWINFIT surgical tray. The ADVANCED for tioLogic® drills do not feature an integrated depth stop. Appropriate depth markings (7.0 mm, 9.0 mm, 11.0 mm, 13.0 mm and 15.0 mm) on the cutting edges of the drills will indicate whether the planned implant length has been reached. The diameters of the ADVANCED for tioLogic® drills are laser-marked and color-coded on the ISO shank.

Implant diameters in mm	3.3	3.7	4.2	4.8	5.5
Color coding	brown	yellow	red	blue	green
Implant lengths in mm	9/11/13/15	7/9/11/13/15			9/11/13/15

The CITO mini® stepped drills are intended for the preparation of the implant site for CITO mini® implants. The CITO mini® stepped drills are supplied non-sterile and packaged individually or non-sterile as part of the CITO mini® surgical tray. The CITO mini® stepped drills do not feature an integrated depth stop. Appropriate depth markings (6.0 mm, 11.0 mm, 13.0 mm and 15.0 mm) on the cutting edges of the drills will indicate whether the planned implant length has been reached. The diameters of the CITO mini® stepped drills are laser-marked on the ISO shank.

Implant diameters in mm	1.8	2.2	2.5
Implant lengths in mm	11/13		

tioLogic® ST and tioLogic® TWINFIT surgical trays contain all the rotary instruments and components needed for implantation of tioLogic® ST and tioLogic® TWINFIT. They are intended for safe storage and preparation.

CITO mini® surgical tray contains all the rotary instruments and components needed for the implantation of CITO mini® dental implants. They are intended for safe storage and preparation.

3. Intended purpose

Drills are intended for atraumatic preparation of the implant site for the implants in the maxilla and mandible.

Equipped trays for storage and preparation.

Non-equipped trays for storage and preparation.

4. Intended users

The tioLogic® ST, tioLogic® TWINFIT and CITO mini® product ranges are supplied exclusively to qualified personnel such as surgeons, dentists and dental technicians. It should only be used by qualified personnel who are familiar with dental implantology, including diagnosis, preoperative planning, surgical technique and prosthetic treatment.

5. Medical indications / contraindications

5.1 Indications

5.1.1 Marking drills

- Placing a marking
- Marking the insertion point
- Can be reused 15 to 20 times in hard bone

5.1.2 ADVANCED depth drill

- Determining the planned implant length and alignment of the implant site
- Can be reused 15 to 20 times in hard bone

5.1.3 ADVANCED surface cutters

- Marking of bone based on diameter to monitor crestal bone availability around the implant
- Preparing a flat bone surface cervically
- Can be reused 15 to 20 times in hard bone

5.1.4 ADVANCED stepped countersink

- Gradual expansion of the bone according to the implant contour in ascending diameter and implant length depending on the bone quality and observing the drilling protocol.
- Collection of bone chips as autologous augmentation material
- Can be reused 15 to 20 times in hard bone

5.1.5 ADVANCED 5 expander

- Gradual expansion of the bone according to the implant contour in ascending diameter and implant length depending on the bone quality and observing the drilling protocol.
- Collection of bone chips as autologous augmentation material
- Can be reused 15 to 20 times in hard bone

5.1.6 ADVANCED thread tap

- Pre-cutting of the threads at high bone density to reduce the implant insertion torque and the resulting radial bone compression
- Can be reused 15 to 20 times in hard bone

5.1.7 CITO mini® stepped drill

- Determining the planned implant length and alignment of the implant site according to the drilling protocol and bone quality
- Can be reused 15 to 20 times in hard bone

5.1.8 tioLogic® ST surgical tray and tioLogic® TWINFIT surgical tray

- Container for the secure storage of the rotary instruments and components ADVANCED for tioLogic®
- Container for disinfecting, cleaning, and sterilizing rotary instruments and components ADVANCED for tioLogic®

5.1.9 CITO mini® surgical tray

- Container for the secure storage of the rotary instruments and components CITO mini®
- Container for sterilizing rotary instruments and components CITO mini®

5.2 Contraindications

5.2.1 Marking drills

- Use for implant site preparation
- Reuse exceeding 15 to 20 times in hard bone

5.2.2 ADVANCED depth drill

- Exceeding the specified implant length
- Reuse exceeding 15 to 20 times in hard bone

5.2.3 ADVANCED surface cutters

- Use for full implant site preparation
- Use of an incorrect diameter
- Reuse exceeding 15 to 20 times in hard bone

5.2.4 ADVANCED stepped countersink

- Exceeding the maximum speed
- Use of an incorrect diameter
- Reuse exceeding 15 to 20 times in hard bone

5.2.5 ADVANCED expander

- Exceeding the maximum speed
- Use of an incorrect diameter
- Reuse exceeding 15 to 20 times in hard bone

5.2.6 ADVANCED thread tap

- Exceeding the maximum speed
- Use of an incorrect diameter
- Reuse exceeding 15 to 20 times in hard bone

5.2.7 CITO mini® stepped drill

- Exceeding the specified implant length
- Use of an incorrect diameter
- Reuse exceeding 15 to 20 times in hard bone

5.2.8 tioLogic® ST surgical tray and tioLogic® TWINFIT surgical tray

- Sterilization and storage without approved sterilization film

5.2.9 CITO mini® surgical tray

- Used as cleaning container
- Sterilization and storage without approved sterilization film

6. Patient target group

All skeletally mature patients undergoing implant treatment. Please consider the indications / contraindications.

7. Information on use, processing, fitting and insertion

Prior to use, dental professionals should ensure that they have carefully read and understood the corresponding tioLogic® ST and tioLogic® TWINFIT or CITO mini® Instructions for use.

We strongly recommend that, prior to using Dentaaurum products, dental professionals attend a Dentaaurum training course to familiarize themselves with the Dentaaurum products.

Carry out thorough radiological, psychological and physical examinations before implantation and inform patients of the process in detail. Dentaaurum recommends full clinical, radiological, photographic and statistical documentation.

Precautions:

Before surgical implantation, it is important to carry out a careful patient anamnesis and, if required, a third-party anamnesis by the patient's doctor (GP/PCP) in order to clarify if 1) implantation may be difficult due to an anatomical situation, 2) there is the risk of a serious surgical problem or a general risk, 3) wound healing/osseointegration may be difficult, or 4) good hygiene and care of the implant, abutment and restoration may worsen. Do not fit dental implants if any of the above cases or a combination hereof is severe or uncontrollable.

Prosthetic restoration of the implant must not be performed until the soft tissue has healed without irritation. Furthermore, a radiographic check-up should be performed before the prosthetic restoration.

Important:

The implantologist should ensure the products cannot be aspirated during intra-oral use.

7.1 Preparation for surgery

Ensure careful and validated hygiene measures throughout the entire operation and document these. Prepare the treatment room, instruments and patient under these aspects. Check all surgical instruments required for the operation to ensure they are complete, functional and sterile. We recommend having several implants and surgical instruments available as a precaution.

All rotary ADVANCED drills and thread taps and CITO mini® drills are supplied non-sterile. They must be thoroughly cleaned, disinfected, conditioned and sterilized before being used for the first time (factory new) and immediately after each use. Rotary instruments should always be checked to ensure that they are sharp, in good condition and the markings are legible, as they have a limited service life. Instruments can become blunt as a result of use and cleaning. Only instruments that are sharp and free from defects should be used. Rotary instruments – used with proper care and provided that they are not damaged or contaminated – can be reused in dense bone 15 to 20 times. Any further reuse or the use of damaged and/or contaminated instruments must be avoided. The dental professional bears responsibility. No liability is accepted if these instructions are disregarded.

7.2 Notes on operation planning

Implants need a distance of at least 1.5 mm to the neighboring teeth. The distance between the implants must be at least 3.0 mm. Select the implant diameter and length so that there is adequate bone around the implant (min. 1.0 mm).

For technical reasons, the ADVANCED for tioLogic® depth drills and the ADVANCED for tioLogic® stepped countersinks and the CITO mini® stepped drills are 1.0 mm longer apically than the given preparation length. This should be taken into account during diagnostics, planning and preparation. The distance to the mandibular nerve and inferior alveolar nerve should be at least 1.5 mm!

7.3 General information on implant site preparation

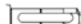
7.3.1 ADVANCED for tioLogic® rotary instruments

Careful preparation of the hard and soft tissue in the bone is important for optimum healing of the implant site. Avoid thermal or mechanical trauma at all costs. To this end, keep the temperature as low as possible when preparing the site and do not exceed the maximum implant insertion torque (max. 40 Ncm).

Low-speed preparation has a positive effect on temperature build-up. This applies specifically in conjunction with the collection of bone chips as autologous augmentation material.

For this reason, low-speed preparation is recommended for certain drills without external cooling.

As a general rule, the speed should be reduced as the drill diameter increases.

Article		Low-speed preparation	Maximum torque
Marking drill		max. 800 rpm	max. 800 rpm
Depth drill		max. 800 rpm	max. 800 rpm
Surface cutter		50-80 rpm	max. 500 rpm
Stepped countersink		50-80 rpm	max. 500 rpm
Expander		50-80 rpm	max. 500 rpm
Thread tap		max. 10 rpm or manually	max. 10 rpm or manually

7.3.2 CITO mini® rotary instruments

Careful preparation of the hard and soft tissue in the bone is important for optimum healing of the implant site. Avoid thermal or mechanical trauma at all costs. To this end, keep the temperature as low as possible when preparing the site and do not exceed the maximum implant insertion torque (max. 40 Ncm).

Article		Maximum torque
Marking drill		max. 800 rpm
Stepped drill ø 1.8 mm		max. 500 rpm
Stepped drill ø 2.2 mm		max. 500 rpm
Stepped drill ø 2.5 mm		max. 500 rpm

7.4 Implant site preparation


7.4.1 ADVANCED for tioLogic® rotary instruments

The handpiece instruments can also be used manually using an adapter (max. permitted torque 40 Ncm). Insert the instruments rotationally secure and check the fit. The manual hex keys and adapter can be used with the torque ratchet set at the relevant torque. The respective components should be secured with a sterile safety cord to prevent aspiration during use.

Select the required rotary instruments and their drilling sequence and depth for the preparation of the implant site depending on the bone quality. A drilling protocol for different bone qualities (soft, medium, hard) is available for the implantologist. Determination of the bone quality rests on the implantologist. Use the drills with intermittent drilling and ensure sufficient cooling using a sterile, cooled physiological saline solution (5 °C / 41 °F) as the diameter increases.

Cut the mucosa through to the bone with a ridge incision and raise the mucoperiosteal flap. The buccal flap section should be adequately mobilized and slightly retracted towards lingual. This exposes the actual contour of the alveolar process. It is generally necessary to make relief incisions mesially and distally. The position of the mental foramen should be clarified before placing implants in the mandible.


7.4.1.1 Drillpoint marking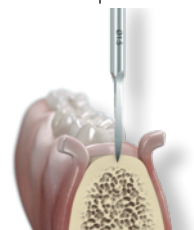

Drillpoint marking


Thin crestal bone in the region of implant insertion can be smoothed slightly with a round bur. The insertion point of the implant can be marked using the marking drill. This serves to center subsequent depth drilling. The green contra angle (800 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F).

7.4.1.2 ADVANCED depth drill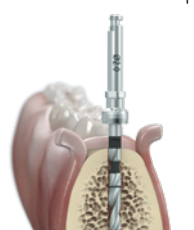

Depth drilling ADVANCED

The ADVANCED depth drill prepares the definitive depth and direction of the implant site independent of the intended implant diameter. It does not feature an integrated depth stop, but instead has depth markings (7.0 mm, 9.0 mm, 11.0 mm, 13.0 mm, and 15.0 mm) corresponding to the respective implant length. The depth markings on the ADVANCED depth drill indicate when the previously determined implant length has been reached. This guarantees that the exact insertion depth determined in the treatment planning is kept to.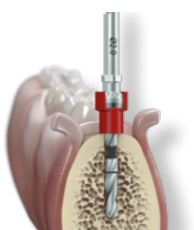

optional:
incl. depth-stop sleeve

As an option, the sterile depth-stop sleeves that are supplied with every tioLogic® TWINFIT implant type can be used. These enable depth-guided drilling corresponding to the implant length. The depth-stop sleeves are coordinated for the use and sequence of the rotary instruments.

Should there be a deviation of the bone quality for the case in hand, the user should proceed in accordance with the drilling protocol.


The depth drill has a diameter of 2.0 mm.

For technical reasons, the depth drill is 1.0 mm longer than the given preparation length. Take this into account during diagnosis and preparation.

In addition, the ADVANCED depth drill has a hexagon chucking system for the transmission of high torques. The green contra angle (800 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F). To allow the tip of the drill to cool down, drill intermittently without applying pressure.

The paralleling pin can be used following depth drilling. This is used as orientation for subsequent depth drilling and as a depth gauge of the depth drilling made. In addition, a paralleling pin with a short thread can be fixed in an implant that is already in place. Secure the paralleling pin with a cord.

7.4.1.3 ADVANCED surface cutters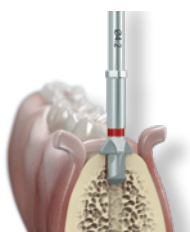


Surface cutting ADVANCED

The ADVANCED surface cutter is designed specifically for drilling through the compact bone and for creating a plane cervical bone level. It features a color-coded groove, which indicates the planned final implant diameter. It is used for all implant lengths. The implant diameter is also laser-marked on the surface cutter. Drill the surface cutter into the bone until a circumferential impression of the cutting cylinder is visible on the compact bone. This ensures that the implant is surrounded by bone crestally.

If the surface cutter is drilled deeper into the compact bone when there is adequate bone availability, the whole implant will be positioned deeper (adhere to the preoperative length measurement).

Depending on the implant diameter, surface cutting can be omitted if there is enough flat bone surface in the crestal region. The green contra angle (500 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F). The surface cutter should be used at low speed (50-80 rpm) to keep the temperature as low as possible and avoid thermal trauma. To allow the tip of the drill to cool down, drill intermittently without applying pressure.

7.4.1.4 ADVANCED stepped countersinks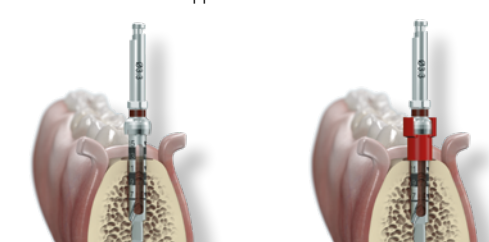

Stepped countersinking ADVANCED optional: incl. depth-stop sleeve

The ADVANCED stepped countersink enlarges the implant site according to the implant contour. It does not feature an integrated depth stop, but instead has depth markings (7.0 mm, 9.0 mm, 11.0 mm, 13.0 mm, and 15.0 mm) corresponding to the respective implant length. The implant diameter is indicated by a color-coded groove. The ADVANCED stepped countersink has laser-inscriptions of the implant diameter on the shank. In addition, the ADVANCED stepped countersink has a hexagon chucking system for the transmission of high torques. All ADVANCED stepped countersinks have a special hollow space for collecting bone chips, which can be used as an autologous transplant. The corresponding depth markings on the ADVANCED stepped countersink indicate when the predefined implant length has been reached. Expansion with the ADVANCED stepped countersink is performed in steps according to the drilling protocol, starting with the smallest available diameter until the specified implant diameter has been reached.

For a specified implant diameter, e.g. 4.2 mm and 13.0 mm length, widen the implant site stepwise after surface cutting with the ADVANCED stepped countersink \varnothing 3.3 mm and ADVANCED stepped countersink \varnothing 4.2 mm, each to the depth mark 13.0 mm.

As an option, the sterile depth-stop sleeves that are supplied with every tioLogic® TWINFIT implant type can be used. These enable depth-guided drilling corresponding to the implant length. The depth-stop sleeves are coordinated for the use and sequence of the rotary instruments for medium bone quality.

Should there be a deviation of the bone quality for the case in hand, the user should proceed in accordance with the drilling protocol.

For technical reasons the stepped countersink is 1.0 mm longer than the given preparation length. Take this into account during diagnostics and preparation.

Manual stepped countersinking using a torque ratchet adapter or sure-grip wheel may be helpful in very soft or narrow bone conditions.

The green contra angle (max. 500 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5°C/41°F). The stepped countersink should be used at low speed (50 rpm – 80 rpm) to keep the temperature as low as possible and avoid thermal trauma. To prevent bone chips from being washed out, low-speed preparation can be performed without cooling. To allow the tip of the drill to cool down, drill intermittently without applying pressure.


Drilling protocol - tiologic® TWINFIT

Adapt preparation protocol according to the indication and situation of each patient if required.

- ⊗ = max. implant depth
- ⑦ = min. depth marking 7
- ⊕ = optional use
- ☐ = refer to Instructions for use

View surgical tray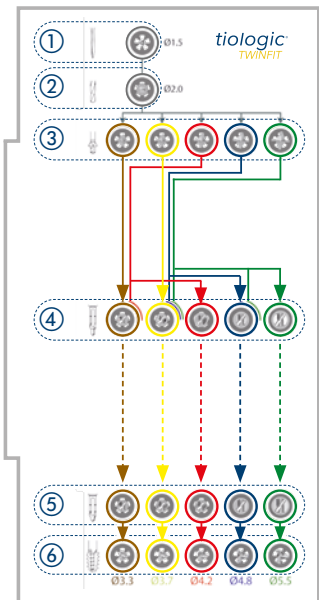

		∅ 3.3	∅ 3.7	∅ 4.2	∅ 4.8	∅ 5.5
①	Marking drill					
②	Depth drill					
③	Surface cutter					
④ a	Preceding stepped countersink					
④	Stepped countersink					
Preparation according to bone quality		Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard
⑤	Expanders					
⑥	Thread tap					
		∅ 3.3	∅ 3.7	∅ 4.2	∅ 4.8	∅ 5.5


Drilling protocol - tiologic® ST

Adapt preparation protocol according to the indication and situation of each patient if required.


- ⊗ = max. implant depth
- ⑦ = min. depth marking 7
- ⊕ = optional use
- ☐ = refer to Instructions for use

View surgical tray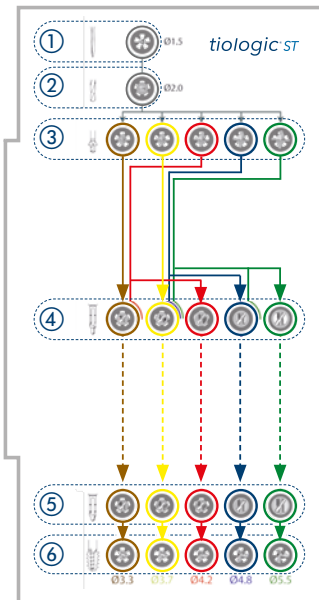

		∅ 3.3	∅ 3.7	∅ 4.2	∅ 4.8	∅ 5.5
①	Marking drill					
②	Depth drill					
③	Surface cutter					
④ a	Preceding stepped countersink					
④	Stepped countersink					
Preparation according to bone quality		Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard	Preparation according to bone quality D4-D3 D3-D2 D2-D1 soft medium hard
⑤	Expanders					
⑥	Thread tap					
		∅ 3.3	∅ 3.7	∅ 4.2	∅ 4.8	∅ 5.5

7.4.1.5 ADVANCED expanders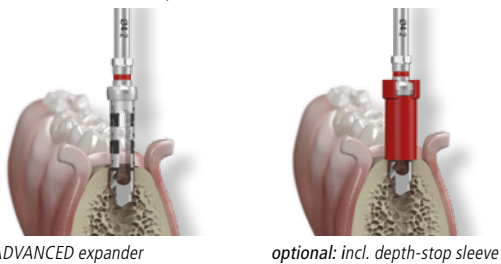

ADVANCED expander

optional: incl. depth-stop sleeve

After use of the ADVANCED stepped countersink, the fine thread section of the implant site is prepared to the diameter of the implant using the ADVANCED expander. It does not feature an integrated depth stop, but instead has depth markings (7.0 mm, 9.0 mm, 11.0 mm, 13.0 mm, and 15.0 mm) corresponding to the respective implant length. The implant diameter is indicated by a color-coded groove. The implant diameter is laser-marked on the shank of the ADVANCED expander. In addition, the ADVANCED expander has a hexagon chucking system for the transmission of high torques. All ADVANCED expanders are provided with a special hollow space for storing bone chips, which can be used as an autologous transplant. Expansion with the ADVANCED expander is performed according to the drilling protocol. Subsequent cutting of the thread may prove necessary in hard bone. The respective insertion depth of the ADVANCED expander allows individual adjustment to achieve maximum primary stability.


As an option, the sterile depth-stop sleeves that are supplied with every tiologic® TWINFIT implant type can be used. The depth-stop sleeves are coordinated for the use and sequence of the rotary instruments for medium bone quality.

Should there be a deviation of the bone quality for the case in hand, the user should proceed in accordance with the drilling protocol.

Manual preparation using a torque ratchet adapter or sure-grip wheel may be helpful in very narrow bone conditions.

The green contra angle (max. 500 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F). The stepped countersink should be used at low speed (50 rpm – 80 rpm) to keep the temperature as low as possible and avoid thermal trauma. To prevent bone chips from being washed out, low-speed preparation can be performed without cooling. To allow the tip of the drill to cool down, drill intermittently without applying pressure.

7.4.1.6 ADVANCED thread taps

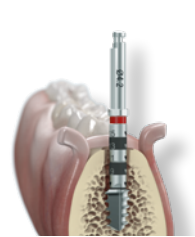ADVANCED Thread tap

Depending on the bone quality, finishing the preparation with a thread tap is recommended as an option. It is available in the same diameters as the implants, and features a color-coded groove and an additional marking on the shank. Corresponding depth markings (7.0 mm, 9.0 mm, 11.0 mm, 13.0 mm and 15.0 mm) on the thread tap indicate when the planned implant length has been attained. In addition, the ADVANCED thread tap features a hexagon chucking system for the transmission of high torques.

Manual preparation using a torque ratchet adapter or sure-grip wheel may be helpful in very narrow bone conditions. Alternatively, the thread tap can also be used with a handpiece (max 10 min⁻¹) with the relevant attachments in the same way as for the manual technique.

7.4.2 CITO mini® rotary instruments

The handpiece instruments can also be used manually as required using an adapter (max. permitted torque 40 Ncm). Insert the instruments rotationally secure and check the fit. The manual hex keys and adapter can be used with the torque ratchet set at the relevant torque. The respective components should be secured with a sterile safety cord to prevent aspiration during use.

Select the required rotary instruments and their drilling sequence and depth for the preparation of the implant site depending on the bone quality. A drilling protocol for different bone qualities (soft, medium, hard) is available for the implantologist. Determination of the bone quality rests on the implantologist. Use the drills with intermittent drilling and ensure sufficient cooling using a sterile, cooled physiological saline solution (5 °C / 41 °F) as the diameter increases.

Carry out implant site preparation using the CITO mini® drilling system, bearing in mind the drilling protocol, as follows.

The following describes preparation regardless of the bone quality in order to show exactly how each of the rotary instruments work. The template-guided preparation and implant insertion with pOPosition for tiologic® are described in the Instructions for use pOPosition for tiologic® rotary instruments and components for guided surgery (REF 989-805-01).


Drilling protocol.

Drilling protocol with CITO mini® drills.
Taking different bone qualities into account.

The drilling protocol may need to be adapted depending on the indication and the individual situation of the patient. The insertion of CITO mini® implants in soft bone is **not** indicated.


	Soft bone quality			Medium bone quality			Hard bone quality		
	Ø 1.8 ³	Ø 2.2	Ø 2.5	Ø 1.8 ³	Ø 2.2	Ø 2.5	Ø 1.8 ³	Ø 2.2	Ø 2.5
Marking drill ²				X	X	X	X	X	X
Stepped drill for Ø 1.8 mm ¹				min. 6.0 mm			X ⁴		
Stepped drill for Ø 2.2 mm ¹					min. 6.0 mm			X ⁴	
Stepped drill for Ø 2.5 mm ¹						min. 6.0 mm			X ⁴

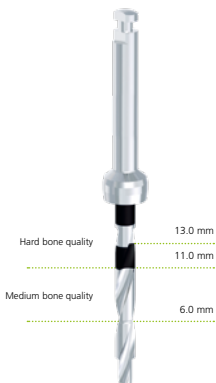¹ The insertion depth / length of the stepped drill depends on the desired primary stability and the bone quality. The depth markings / scaling help to avoid damaging neighboring structures (e.g. nerves) due to perforations that are too deep.
² Optional use.
³ Not intended for restorations in the maxilla.
⁴ Should the bone be very hard, it may be necessary to use a stepped drill one size larger to ream the cortical bone.


7.4.2.1 Drillpoint marking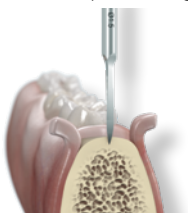

Drillpoint marking

Thin crestal bone in the region of implant insertion can be smoothed slightly with a round bur. The insertion point of the implant can be marked using the marking drill. This serves to center subsequent depth drilling. The green contra angle (max. 800 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F).

7.4.2.2 CITO mini® stepped drill

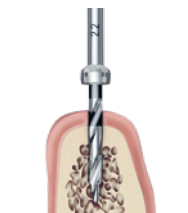CITO mini® stepped drill

The CITO mini® stepped drills prepare the definitive depth and direction of the implant site dependent of the intended implant diameter. The implant diameter is laser-marked on the shank of the CITO mini® stepped drill. It does not feature an integrated depth stop, but instead has depth markings (6.0 mm, 11.0 mm and 13.0 mm) corresponding to the respective implant length and the prevailing bone quality. The depth markings on the CITO mini® stepped drill indicate when the previously determined implant length has been reached. This guarantees that the exact insertion depth determined in the treatment planning is kept to.

Should there be a deviation of the bone quality for the case in hand, the user should proceed in accordance with the drilling protocol.

For technical reasons, the stepped drills are 0.6 mm longer than the given preparation length. This should be taken into account during diagnosis and preparation.

The green contra angle (max. 500 rpm) is used for drilling with external cooling using a sterile, cooled physiological saline solution (5 °C/41°F). To allow the tip of the drill to cool down, drill intermittently without applying pressure.

The paralleling pin can be used following depth drilling. This acts as orientation for further depth drilling. The paralleling pin should be secured with a cord.

Important:

Should the bone be very hard, it may be necessary to use a stepped drill one size larger to ream the cortical bone.

8. Technical specification

Composition

Product	Composition
Marking drill	1.4197
ADVANCED depth drill	1.4197
ADVANCED surface cutters	1.4197
ADVANCED stepped countersink	1.4197
ADVANCED expander	1.4197
ADVANCED thread tap	1.4197
CITO mini® stepped drill	1.4197
tioLogic® ST and tioLogic® TWINFIT surgical tray	PPSU
CITO mini® surgical tray	PPSU

Information about the composition is included in the material list for our implantology products (REF 989-801-06), or see www.dentaurum.com/ifu. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

9. Adverse reactions and residual risks

Possible complications and risks during surgical treatment (temporary adverse reactions) are:

Pain, swellings, difficulties speaking, inflammation of the gums.

Possible complications and risks during prosthetic treatment (permanent adverse reactions) are:

Chronic pain in connection with the implant, permanent paresthesia, dysesthesia, nerve damage, exfoliation, hyperplasia, local or systemic infections, oroantral or oronasal fistulas, alveolar bone loss in maxilla/mandible, irreversible damage to adjacent teeth, fractures to the implant, jaw, bone or restoration, aesthetical problems.

10. Safety information and precautions

- Do not use the product if there is a known allergic reaction to one or more of the material components.
- Due to the small size, the article could be swallowed or aspirated. Aspiration could lead to difficulty in breathing or death due to asphyxiation. Therefore, secure all articles used intra-orally against swallowing and/or aspiration.
- Different types of alloy in the oral cavity can lead to galvanic reactions.
- If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.
- All ADVANCED for tioLogic® drills may only be used in conjunction with tioLogic® ST and tioLogic® TWINFIT implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaurum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- All CITO mini® stepped drills may only be used in conjunction with CITO mini® implants! Use of third-party components can negatively impact the function and safety. In such cases, Dentaurum GmbH & Co. KG accepts no liability for replacement products or guarantees.
- Observe the indications for the individual products. Consider all indications that are not listed explicitly to be contraindications.
- Disinfection, cleaning and sterilizing should not be done in the original packaging.
- Products must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.
- Rotary instruments – used with proper care and provided that they are not damaged or contaminated – can be reused in dense bone 15 to 20 times. Any further reuse or the use of damaged and/or contaminated instruments must be avoided. The dental professional bears responsibility. No liability is accepted if these instructions are disregarded.
- Technical advice (oral and written) is based on the scientific and technical knowledge available at the time the product is put on the market. It does not release the dental professional from the responsibility of personally checking the suitability of the products for the intended indications and applications. Advice is only given as a non-binding recommendation and cannot be assumed to provide any form of assurance, liability or guarantee.

Severe incidents:

Report all severe incidents arising from the use of the product to the manufacturer and the competent authority in the country in which the dental professional and/or the patient are resident.

Severe incidents are identified as malfunctions and a drop in performance or deterioration of the properties of a medical device which resulted or may have resulted in a direct or indirect temporary or permanent deterioration in a patient's state of health, in the death of a patient, a dental professional or other persons or as posing a serious danger to public health.

11. Storage and shelf life

Always store the tioLogic® ST, tioLogic® TWINFIT and CITO mini® implants, abutments and components in a dry, dark place at room temperature in the original packaging. If the packaging shows damage, the product must be checked for integrity and cleanliness before use, and must be disposed of if necessary.

12. Disposal

Adhere strictly to the rules that apply for your region and the details outlined in the safety data sheets, if available. Products to be disposed of should be treated as surgical waste and be decontaminated in compliance with the relevant regulations for medical waste disposal.

13. Information for multiple use products

The ADVANCED for tioLogic® drills and the CITO mini® stepped drills are supplied non-sterile and are intended for multiple use. The product must be disinfected, cleaned and sterilized before being used initially and before being used again. ADVANCED for tioLogic® drills and the CITO mini® stepped drills must be replaced if they cannot be clearly identified or if the function is impaired due, for example, to poor readability of the markings or labels.

14. Information on disinfection / cleaning / sterilizing

The ADVANCED for tioLogic® drills and the CITO mini® stepped drills are supplied non-sterile. They are intended for multiple use. They must be cleaned, disinfected and sterilized before and after being used on a patient (e.g. transfer to dental laboratory). For sterilization, the loaded tioLogic® ST, tioLogic® TWINFIT or CITO mini® surgical tray is packed in disposable packaging dedicated to the sterilization process in accordance with ISO 11607-1/2. The materials used to manufacture the tioLogic® ST and tioLogic® TWINFIT surgical trays can withstand 300 sterilization cycles.

15. Product overview and scope of delivery

Please refer to the tioLogic® ST product catalog for the tioLogic® ST product range.

Please refer to the tioLogic® TWINFIT product catalog for the tioLogic® TWINFIT product range.

Please refer to the CITO mini® order form for the CITO mini® product range.

16. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

17. Explanation of symbols

Please refer to the label. See chapter "Additional information".

18. Additional information

Additional information can be found at www.dentaurum.com/ifu

Instructions for use	REF
tioLogic® ST Additional components	989-805-87
tioLogic® TWINFIT Additional components	989-805-88
CITO mini® matrices and components	989-804-97
Torque ratchet	989-719-00
Preparation instructions for the implant system components	989-801-07
Explanation of symbols used on the label	989-313-00

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